

MedicalWear Glove

pre-sized gloves

MedicalWear pre-sized gloves are designed to provide compression therapy in edema reduction.

MedicalWear pre-sized gloves provide gradient pressure. Contracture seams provide intimate contact to dorsal web spaces.

Features and Benefits:

- Convenient ready-to-wear range with six adult sizes available
- Choose from 2 styles standard and wrap-around closure
- · Ambidextrous fits right and left hand
- Flattened seams for added comfort minimizes pressure over newly formed sensitive skin
- Cool and light, the open weave fabric allows the skin to breathe
- Soft VELFOAM® wrist closure on the wrap-around closure glove

Indications:

- Interim therapy for the control and reduction of edema
- · Provides relief for arthritic symptoms of the hand





Ordering Information

JOBST GLOVE SIZE (COMMON TO BOTH RIGHT & LEFT HANDS)	STANDARD STYLE	WITH WRAP-AROUND VELCRO® BRAND CLOSURE
SR-Small Reg.	100580	100586
SL-Small Long	100581	100587
MR-Med. Reg.	100582	100588
ML-Med. Long	100583	100589
LR-Large Reg.	100584	100590
LL-Large Long	100585	100591

NOTE: Please refer to product label and /or package insert for full instructions on safe use of these products.





MedicalWear Glove


pre-sized gloves

Measuring Information

- 1. Select RIGHT or LEFT hand measuring template.
- 2. Center pencil (or similar post) at point (P) to act as stop. Place patient's hand, palm down, on the illustrated hand template and butt web space between middle and index fingers up against stop at (P).
- 3. Press patient's fingers together.
- 4. Trace horizontal line from tip of middle finger (T) to glove size diagram.
- 5. Trace vertical line up from base of little finger (B) to glove size diagram.
- 6. Find exact glove size at intersection of vertical and horizontal lines in glove size diagram.
- 7. Reference glove size and style to Ordering Information.

NOTE: Although right and left hands must be measured on individual templates, Pre-sized Gloves are common to both right and left hands.

VELFOAM is a registered trademark of VELCRO U.S.A., Inc.

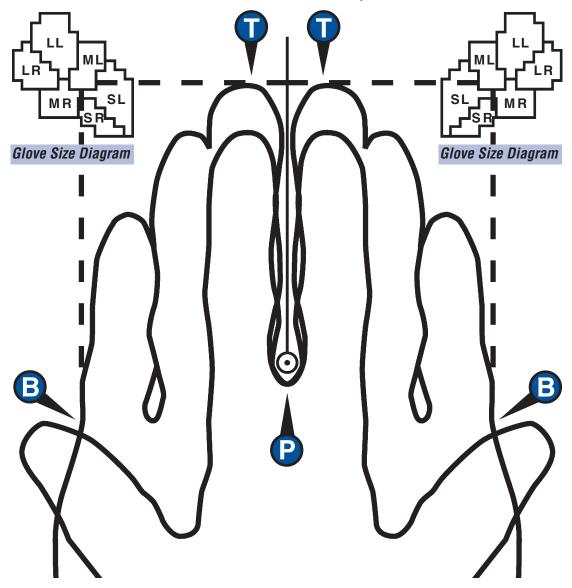

BSN-JOBST, Inc.

5825 Carnegie Blvd., Charlotte, NC 28209-4633 PO Box 471048, Charlotte, NC 28247-1048 U.S.A. To order toll-free 1-800-537-1063

Tel. 704-554-9933 Fax: 1-800-835-4325 Fax for international use: 704-551-7173

BSN-JOBST

www.jobst.com