


ponyTM

English user manual

GB CONTENTS

Pony	3
Safety	4
Garantee.....	4
Tools	4
Maintenance	4
Ready for use	5
Height adjustment of seat and body support.....	6
Angle adjustment.....	6
Directional stabilizer, rear wheel.....	7
Directional stabilizer, front wheels	7
Drag wheels	8
Abdominal pad.....	8
Rear support.....	8
Head support.....	9
Handbrace	10
Armrest.....	10
Side supports.....	11
Helper's handle	11
Vertical hand grips, mounted at the ends	12
Vertical hand grips, mounted in the middle.....	13
Product identification	14
Measurements.....	15
Technical data.....	16
Producer	16
Distributor	16

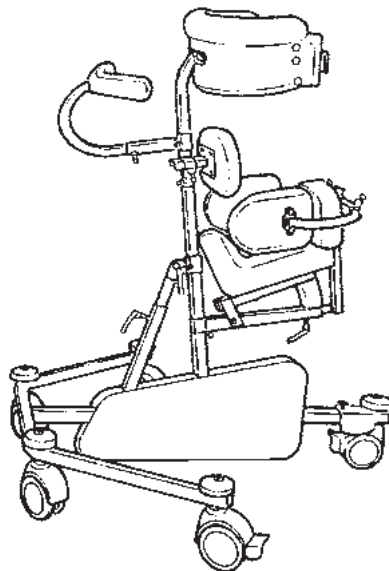

Pat. pend.

GB PONY

Congratulations on choosing the PONY. We are convinced that it will fully live up to your expectations.

The PONY is a walking aid from R82 for children from 18 months to eight years of age needing supplementary assistance with their daily walking. It comes in 4 different sizes.

This manual describe and illustrates all functions, and will enable you to take maximum advantage of your PONY.



GB SAFETY

Pony is CE-marked. This is your guarantee that the product fulfills the required European health- and safety demands. Pony is CE-marked according to the European Council Directive 93/42/EØF, class I, of 14. june 1993 regarding medical arrangements.

The durability of this product correspond to a life cycle of 7 years at normal use. After this we recommend to have the product renovated (by R82 personel) to extend the life time.



Pony is solely for indoor use.



Never leave your child unattended in this product. Ensure permanent supervision by an adjult - not even if all the wheels are locked.



Remove the CE-mark, when re-building the product or when using other than original R82 spare parts and fittings.

GB GARANTEE

Notification of defects will be received at R82 A/S within two years.

The Pony is guaranteed for 12 months. The guarantee is only valid when using original R82 spare parts and fittings. All special adjustments must be made by R82. R82 will not be held responsible for damage or injury caused by use of non-original parts or repairs made by a non-authorized R82 person.

GB TOOLS

Behind the rear support you will find a 4 mm Allan key.

This key is used to perform some of the adjustments described in this manual.



GB MAINTENANCE

The Pony functions without any maintenance, but it is recommended to clean the frame frequently.



Do not use any cleaning materials containing chlorine.



Adjustments and changes not described in this manual, must be made by authorized R82 personel.

GB READY FOR USE

Lift up the two bars on the frame. Except Pony 0

NOTE: Remove black lever first, then:
A: Place the schroud over the bars.
B: Fasten the schroud with the four Phillips headscrews.
Except Pony 0




Place the seat on the large vertical bar. Fasten with the attached Allen key.


Install the angle adjustment bar:
A: Insert the bar into the smaller vertical bar. Fasten with handle.
B: Place the pivoting end over the larger vertical bar. Fasten with the Allen key.

Place the hand grip on the body support bar and slide it to the top. Fasten with the Allen key.

Insert the body support into the larger vertical bar and fasten with the Allen key.

Your PONY is assembled.

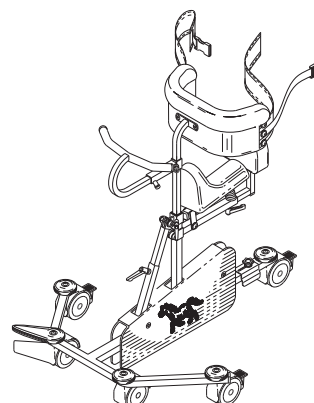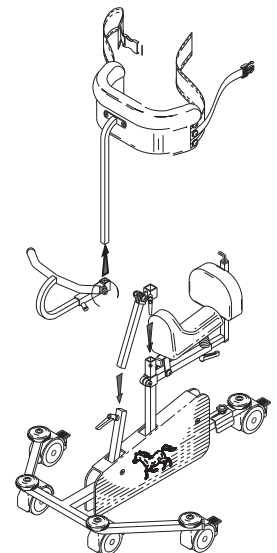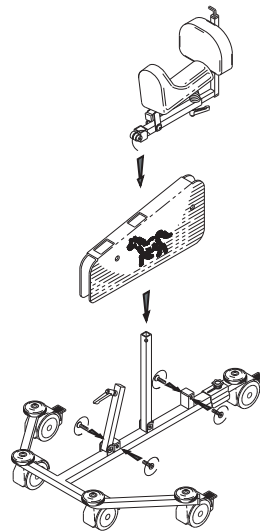


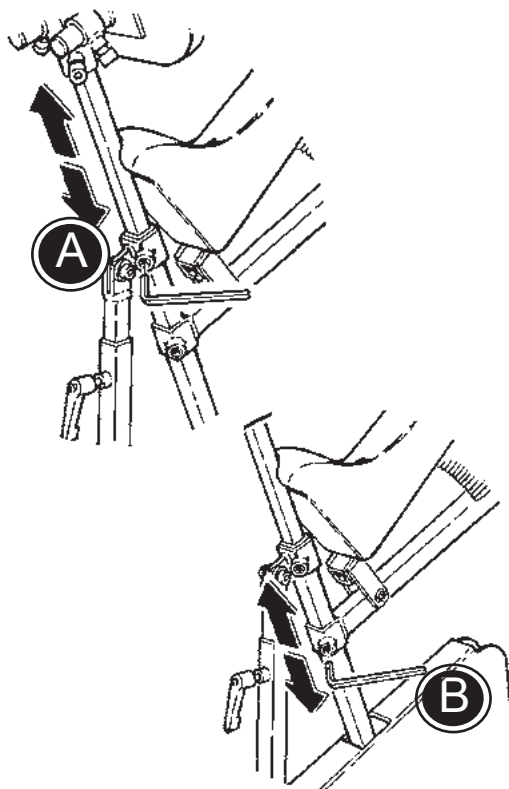

GB HEIGHT ADJUSTMENT OF SEAT AND BODY SUPPORT

Adjust the height or remove the body support by using the Allen key. (A)

Use the Allen key as shown to adjust the seat height. (B)

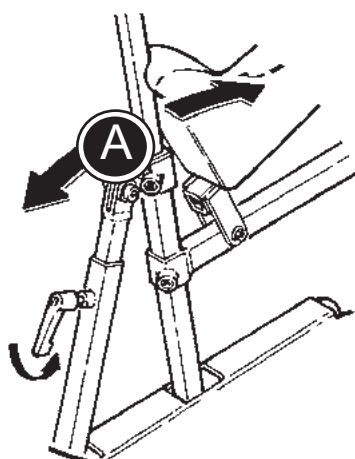


GB ANGLE ADJUSTMENT

Loosen the black lever (A) as shown to adjust the angle of the PONY. Tighten the lever when the desired angle is attained.

GB DIRECTIONAL STABILIZER, REAR WHEEL


The directional stabilizer prevents the rear wheel from turning. To lock the rear wheel, loosen the knob securing the stabilizer, turn the wheel as shown, slide the stabilizer over the wheel, and tighten the knob. (A)

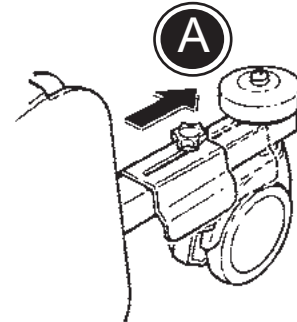

GB DIRECTIONAL STABILIZER, FRONT WHEELS

Follow the instructions below to mount the directional stabiliser (B) on the Pony.


The directional stabilisers are mounted by the 2 front wheels (A) on the Pony.

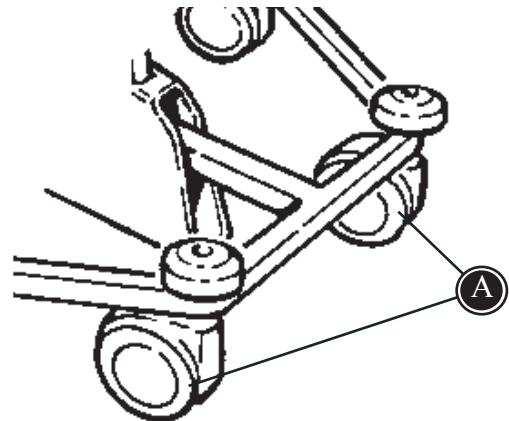


- * Activate the stabilisers by swinging down the 'legs' (E).
- * Make sure that the wheels (A) are pointing in the right direction as shown on the drawing.
- * Place the stabiliser (B) over the frame behind the wheels, as shown on the drawing.
- * Tighten the screw (C)

When the directional stabilisers are mounted, these are easily activated or de-activated. This is done by swinging the 'legs' (E) upwards or downwards.


For exact directional stabilising adjustment, the screw underneath (D) can be loosened and the stabiliser adjusted and then retightened.

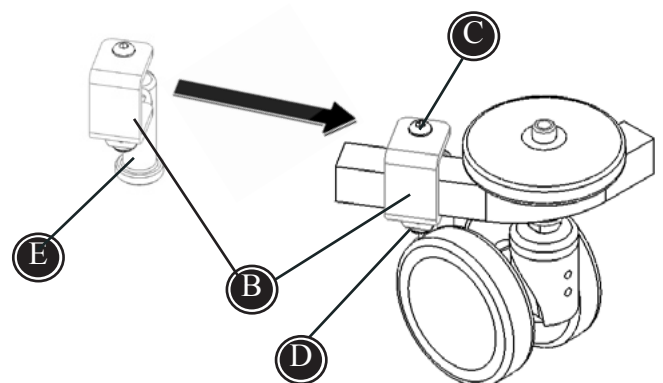

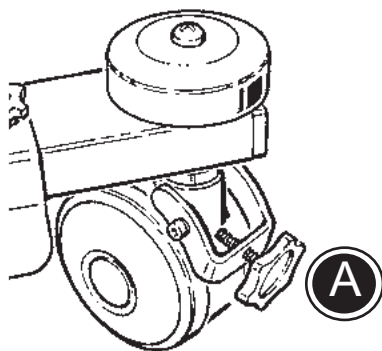

GB DRAG WHEELS

The Pony is as standard equipped with one drag wheel. Mount the enclosed screw on the rear wheel and fasten as needed (A).

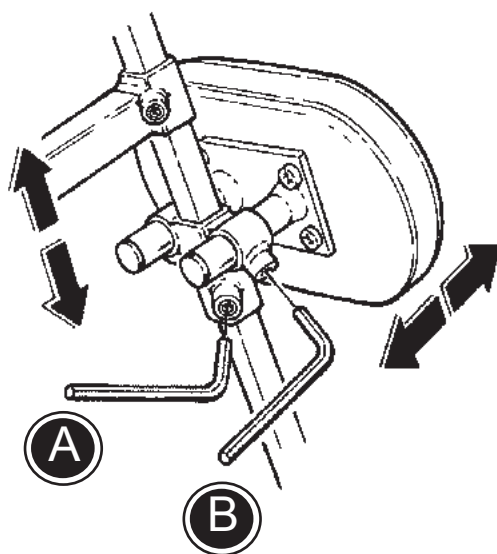


GB ABDOMINAL PAD

Height adjustment of the abdominal pad is performed with the Allen key. (A)

Depth adjustment of the abdominal pad is performed with the Allen key. (B)



Only for size 1+2+3

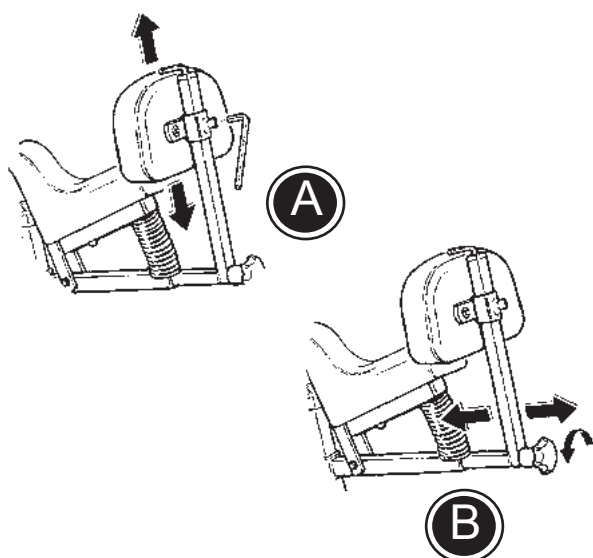

GB REAR SUPPORT

Use the Allen key to adjust the height of the rear support. (A)

Remove or adjust the depth of the rear support by utilizing the handle behind the seat. (B)



GB HEAD SUPPORT


Follow the below to mount the head support correctly on the Pony:

- 1) Loosen the screw (A) to dismount the rear support and the side supports.
- 2) Mount the head support beneath the seat and fasten the screw (B).
- 3) Loosen the handles (C), (D) and (E) to height, width and angle adjust the head support.



When both head support and side supports are needed, the side supports have to be mounted on the body support bar. Please notice that Pony size 2+3 can not be equipped with both head support and side supports.

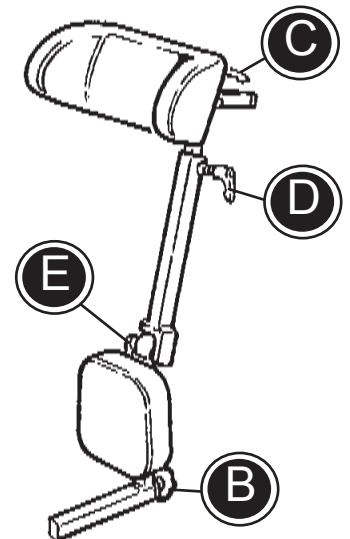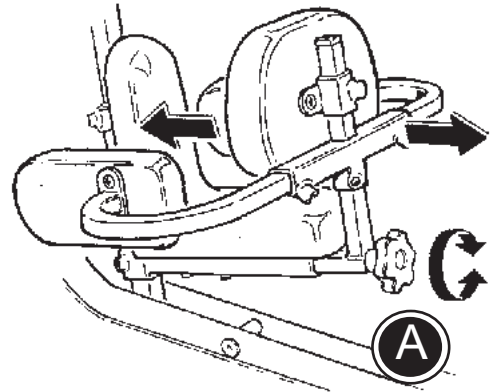


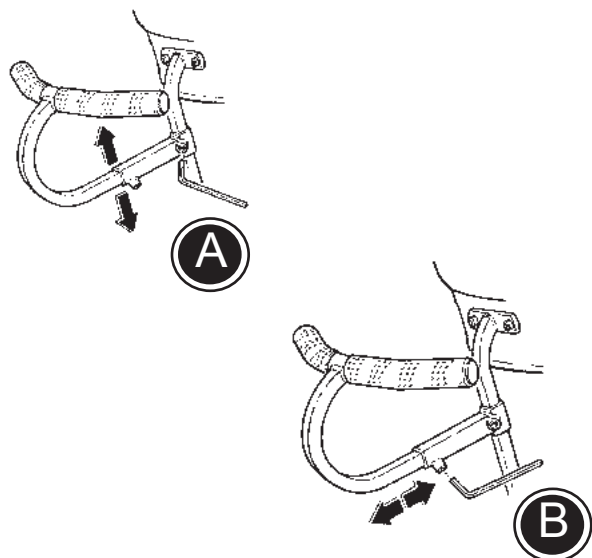

GB HANDBRACE

Height adjustment of the handbrace is done with the Allen key (A).

Depth adjustment of the handbrace is done with the Allen key (B).

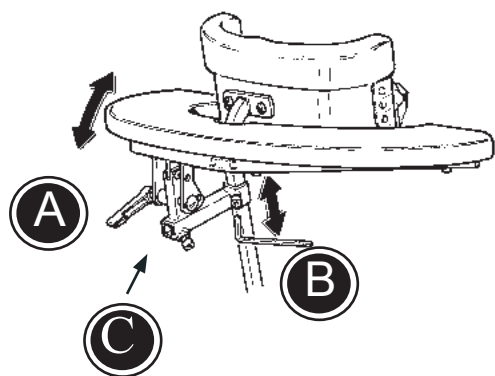

GB ARMREST

An armrest can be mounted together with the handbrace. Dismount the body support (see page 10) to switch the fittings.

Angle the armrest by using the black lever (A). Adjust the height with the Allen key. (B)


Insert the hand brace into the cross bar of the armrest fittings. (C)

GB SIDE SUPPORTS

The side supports are mounted on the rear support bar.

Adjust the height with the attached Allen key (A).

The side supports are individually adjustable in width as shown. (B)

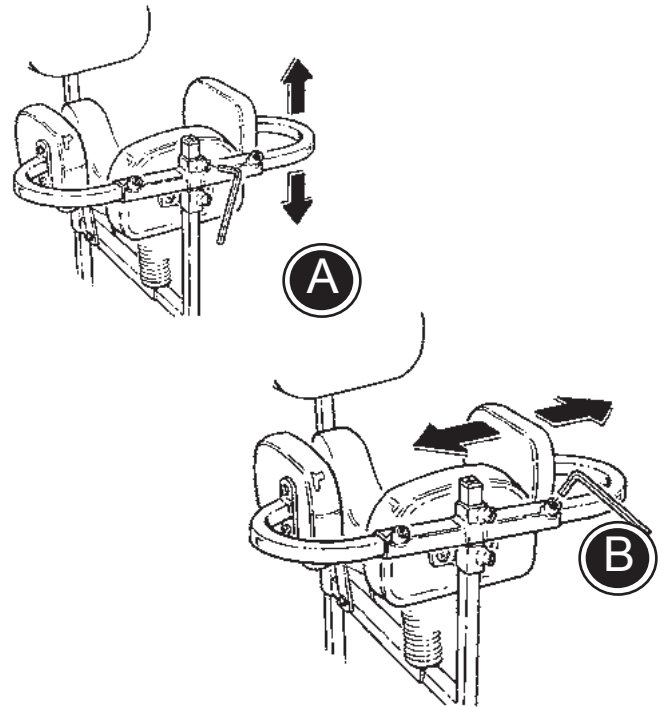



GB HELPER'S HANDLE

1. To be able to place the helper's handle, mount the fittings first. Start by dismounting the two screws as shown (A). Then place the fittings and tighten the screws again. (B)
2. When the bar clicks into place tighten the black lever as shown.
3. Height adjustment of the handle.



Do not lift the Pony by using the helper's handle.

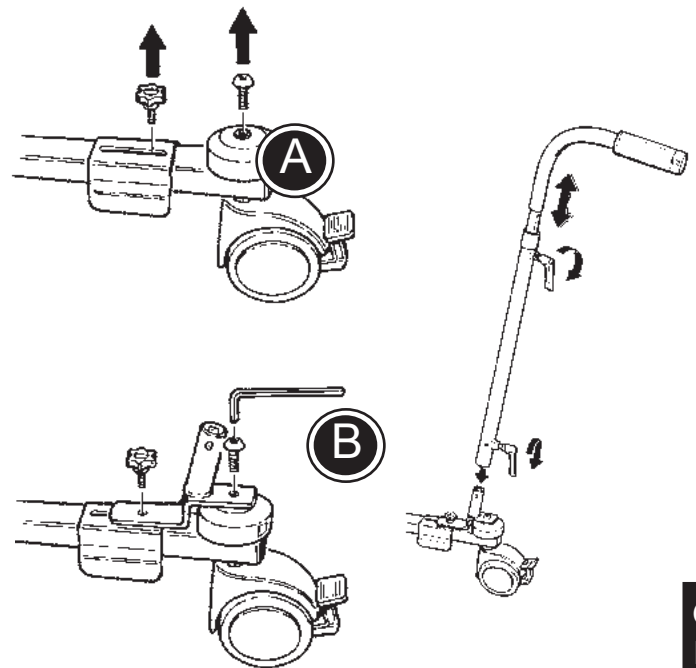

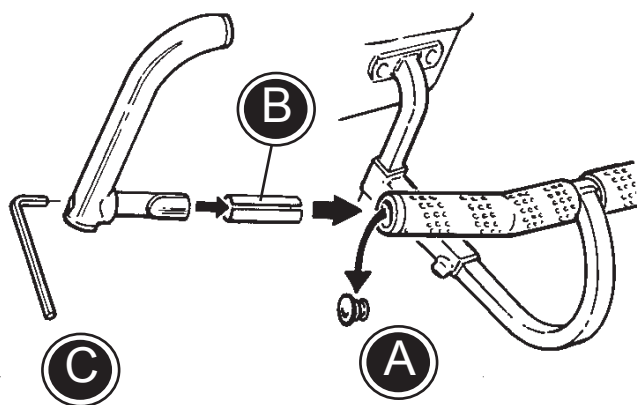





GB VERTICAL HAND GRIPS, MOUNTED AT THE ENDS


Mount the vertical hand grips at the end of the hand brace in the following manner:

- 1) Pop out the stopper (A) with a screwdriver.
- 2) Insert the grip and the supplied bushing (B) into the hand brace at the desired angle.
- 3) Tighten with the Allen key (C).

GB VERTICAL HAND GRIPS, MOUNTED IN THE MIDDLE

1. Cut the grip material (A) and remove it from the handbrace. Remove the stopper.
2. Determine placement of the hand grip (B) and take measurements to determine length of new grip material required for both sides of the hand grip. Cut grip material (C) to fit handle.
3. You can notch the grip material as shown (D) for complete coverage of the handle.
4. Mount the inside piece of the grip material. (Pouring some ethyl alcohol into the grip helps it slide onto the hand brace). Mount the hand grip (E) and tighten with the Allen key. Mount the outer grip material and replace the stopper.

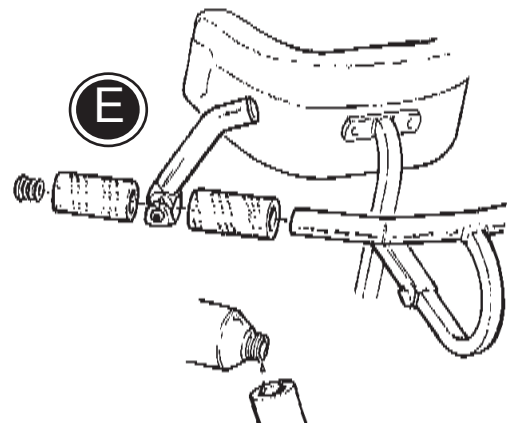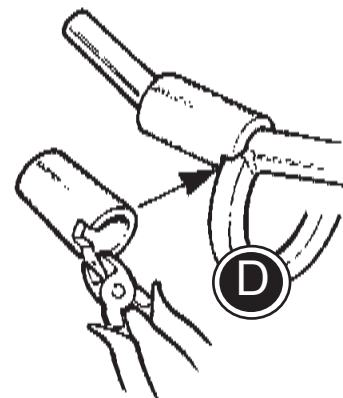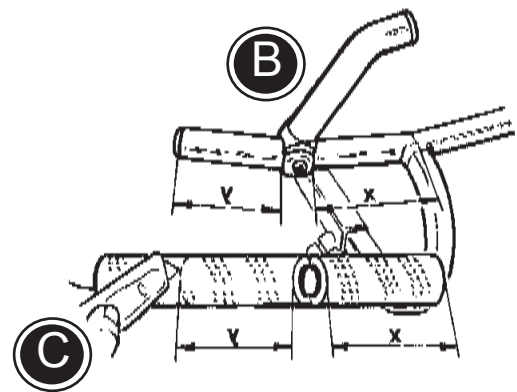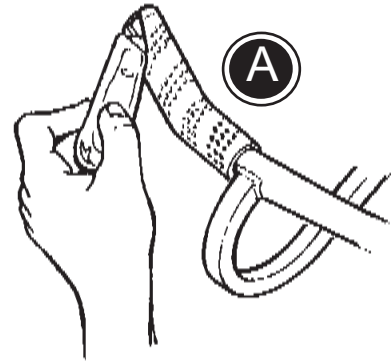

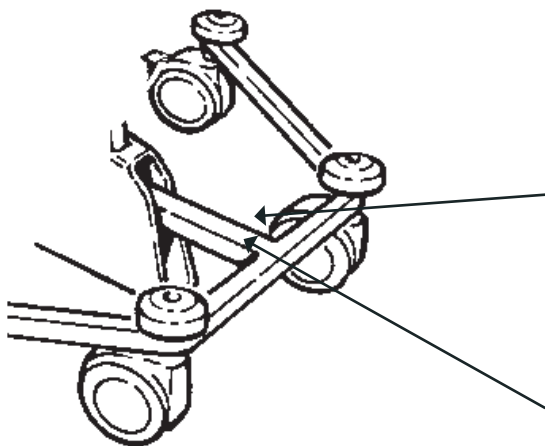

GB PRODUCT IDENTIFICATION

A) Serial number

The label is placed between the front wheels under the frame.

B) Manufacturer


The label is placed between the front wheels on top of the frame.

A




B



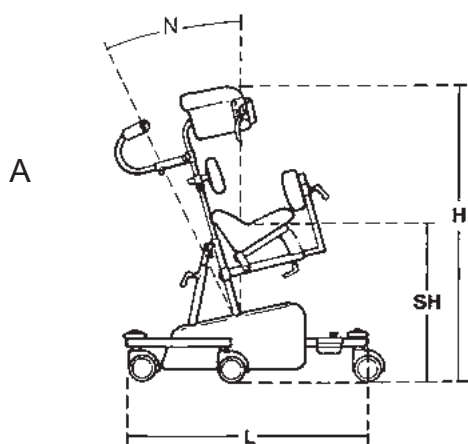

L

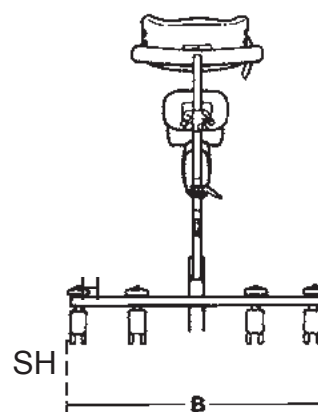

W

GB MEASUREMENTS

	Size 0	Size 1	Size 2	Size 3
Length (L)	67 cm	72 cm	104 cm	125 cm
Width (W)	59 cm	69 cm	77 cm	103 cm
Height (H)	52 - 69 cm	60 - 82 cm	75 - 104 cm	100 - 156 cm
Seat height (SH)	24 - 38 cm	38 - 50 cm	48 - 64 cm	55 - 90 cm
Angle frequency (A)	32°	32°	32°	32°
Weight	7 kg	10 kg	13,5 kg	17 kg
Max load	30 kg	40 kg	50 kg	75 kg

GB TECHNICAL DATA

Frame: Chrome or powder lacquered Steel

Upholstery: PUR (Polyurethane foam)

GB PRODUCER

R82 A/S
Parallelsvej 3
8751 Gedved
Denmark

GB DISTRIBUTOR

Please find your distributor on www.r82.com