

Wiring Diagram

Basic Connection Instructions

For connecting the audio plugs in your Somatron Audio Equipment.

Disconnect the main speakers from your stereo amplifier.

Connect the cable from the SOMATRON product to your stereo amplifier according to the following color codes:

CABLE STEREO

- Red Right Plus or R +
- Black Right Minus or R -
- Green Left Plus or L +
- White Left Minus or L-

This will get you started, please contact us and we will gladly send additional information to you for your specific Somatron products requirements.

General Instructions

Stereo and/or portable players are REQUIRED for use with Somatron products!

Stereo Requirements:

- 10-100 watts of power required for Mat, Recliner Cushion, Wedge, Therapy Lounge, Exercise Rocker, Play floor, and Crawl Space.
- 20-200 watts of power required for Cloud Chair, Learning Chair, Body Pillow, Abbotsford Chair, Clinical and EZ Access Recliners, Music Mattress, Tubby Ball Pool, Waterbed, Multi-sensory Room.
- Portable CD player or MP3 music input required for Neurotron, Headrest and Snugums (amplifier included).
- Bookshelf stereo unit available for purchase. Item #A300
- Amplifier (portable music input required) available for purchase. Item #A400

Additional Charges:

There are additional charges for any color other than the stock color for all products. Prices quoted are for continental U.S. only.

Approximate Manufacturing Times

- Two weeks for portables and soft furniture
- Four weeks for Recliners, Exercise Rocker, Therapy Lounge and Abbotsford
- Six weeks for Ball Pool, Multi-sensory Room, Crawl Space, Waterbeds, Play floor and special orders.

Shipping and Handling

- In the continental U.S.: Shipping and handling will be quoted and charges are based on the product; quantity of items and destination.
- Alaska, Hawaii and Puerto Rico shipping and handling will require more information and will be quoted separately.
- Outside of the U.S. shipment will require the purchaser to provide a freight forwarder and purchaser is responsible for payment to the freight forwarder.

Warranty

Somatron products are warranted for one year, from the shipping date, against defects in materials and workmanship.

Patent Information

Protected by U.S. and International patents and patents pending, including Canada, Britain, Germany, France and Japan.

Medical Disclaimer

Somatron are not manufactured or sold as medical devices by the Somatron Corporation.

Cleaning Instructions

Day-to-day soil-remove ordinary dirt and smudges with a mild soap and warm water solution. Dry with a soft, lint-free cloth or towel. For more difficult stains, use a stronger detergent; however, follow the detergent manufacturer's instructions closely.

Special cleaning problems - Many stains will be removed when these cleaning agents are used in the following order:

Step 1: Use nonabrasive household cleaners such as Formula 409 All Purpose Spray Cleaner or Fantastik Spray Cleaner with water and a soft cloth.

Step 2: Use a solvent-type cleaner such as rubbing alcohol (isopropyl alcohol) liberally applied with a soft cloth. Dry area with another cloth, rinse with clean water and dry.

Step 3: Use strong, active solvent cleaners such as nail polish remover (acetone / water) applied with a soft cloth. Stain should be removed with less than six (6) rubs; if stain persists after six rubs, stain has probably set** and may not be capable of being removed. Dry area with another cloth, rinse with clean water and dry.

***Caution:** These solvents are highly flammable. Exercise proper care in cleaning and notify personnel in area of danger. Wear rubber gloves during all cleaning activity. Use caution in cleaning around buttons, stitching and wooden or decorative trim, since these solvents could easily damage such areas.

**Although vinyls protected with PreFixx are resistant to most common stains, some fabric dyes have the ability to create a permanent stain. To minimize any problems from these and other stains, it is recommended that as soon as they are noticed all stains should be cleaned, making sure the recommended cleaning steps are followed in order.