

Champion Recliner Operation



1
LOCK CASTERS



2
EXTEND PULL-OUT STEP



3
ENTER AND SIT BACK



4
UPRIGHT POSITION



5
RELEASE HAND LEVER FOR "TV" POSITION. Foot rest will rise.



6
"TV" POSITION



7
PUSH BACK WITH HEAD AND SHOULDERS. To assist, grasp recliner arms and pull hips forward.



8
FULL RECLINE



9
SIT UP TO RETURN TO "TV" POSITION. To assist, grasp recliner arms and push hips back.



10
RETURN TO UPRIGHT POSITION. Foot rest will "click" when latched.



11
UPRIGHT POSITION FOR EXITING CHAIR



12
EXITING THE CHAIR

Exiting with Attendant Assistance



1
Locate and move hands and feet to safe location.



2
Move to the rear of the chair and use push handle to lift back to "TV" position.



3
Move to front of the chair and push the foot rest closed.



4
Allow exit once foot rest is latched.



NEVER exit without closing foot rest.



NEVER exit while reclined or if foot rest is extended.



NEVER exit from the side of the chair.



NEVER put your hands into the mechanism area.



NEVER put your hands into the mechanism area.



NEVER force the foot rest closed.