

Setting the Trendelenburg Feature

The Trendelenburg function should be operated by the caregiver. Your chair must be purchased with the Trendelenburg option when it was ordered. If your chair has this option, follow these directions for operation.

1 ADJUST TO FULL RECLINE POSITION

Before operating the Trendelenburg function, you must move the chair to the full-recline position. Pull the side lever to release the foot rest. Then push the chair backrest as far as it will go into the full-recline position.



RELEASE FOOT REST WITH SIDE LEVER



TV POSITION



FULL RECLINE POSITION

2 PLACE THE RECLINER IN THE TRENDELENBURG POSITION

Locate the Trendelenburg lever on the chair back (shown to the right). Pull the lever all the way up and hold as you push the backrest down as far as it will go to the Trendelenburg position (shown to the right bottom). You may stop at any position going downward by releasing the lever.

Please note: The caregiver is shown at the back of the chair in these instructions for clarity. You may also perform this function from the side of the chair.



LOCATE BACK LEVER



RAISE AND HOLD LEVER



PUSH RECLINER BACK DOWN



FULL TRENDELENBURG POSITION

3 RETURN TO UPRIGHT POSITION

To return the chair to upright position, locate the Trendelenburg lever on the chair back. Pull the lever all the way up and hold as you lift the backrest and return it to the fully-upright/TV position (shown at bottom right). It is recommended that this step be done from the back of the chair.



LOCATE BACK LEVER



RAISE AND HOLD LEVER



LIFT RECLINER BACK UP

4 RAISE TO FULL UPRIGHT POSITION

Once the backrest is returned to the fully-upright/TV position, push the foot rest down until you hear it click and lock in place. To avoid damage to the mechanism, the foot rest should never be pushed down until the backrest is returned to the fully-upright/TV position.



RAISE BACK OF RECLINER



PRESS FOOT REST DOWN



FINAL UPRIGHT POSITION

CHAMPIONTM
Quality Healthcare Seating Products