



Store these instructions in a safe place or with the enclosed maintenance checklist

Instruction Sheet

Tube Swing Harness 1500 / Tube Swing Kit 1501



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

Warning: Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

MATERIALS LIST:

Tube Swing Harness 1500

- 1 - Tube Swing Harness
- 1 - Valve Stem Cover
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

Tube Swing Kit

- 1 - Tube Swing Harness
- 1 - Valve Stem Cover
- 1 - Truck Tube
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

MINIMUM HEIGHT REQUIREMENT AND SPACE NEEDED:

Minimum Height required is 8 ft.
Keep a clear distance of 6 ft. from walls and other obstacles.

HEIGHT ADJUSTER AND SAFETY ROTATIONAL DEVICE NEEDED:

Use of the Southpaw Height Adjuster (# 5000) is recommended for ease of adjusting equipment.

A Southpaw Safety Rotational Device (# 2000) **must** be used to prevent damage to your harness.

WORKING LOAD: 250 lbs. when used with our tube.

ASSEMBLY PROCEDURES:

- Inflate tube to desired size. Slip valve cover over the valve stem.
- Wrap harness around the tube with the foam padding on the inside of the cover covering the valve stem. The Blue Webbing should go around the tube completely as in Fig. 3.
- Put the hook strip of the hook and loop fastener through the wire loop. Pull the end of the white web through the wire loop until the folded section of the webbing is past the wire loop (Fig. 1).
- Fold the webbing over and fasten the hook to the loop of the hook and loop fastening strip (Fig. 2).
- To hang your tube swing put both long blue loops through a Southpaw Safety Snap (# 2000) and hang the safety snap from your suspension system. You must use a Southpaw Safety Rotational Device (# 2000) with your tube swing. This will give you free movement and take stress off of your support system.

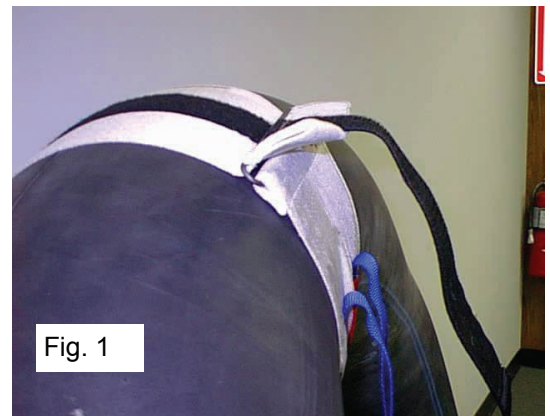


Fig. 1



Fig. 2

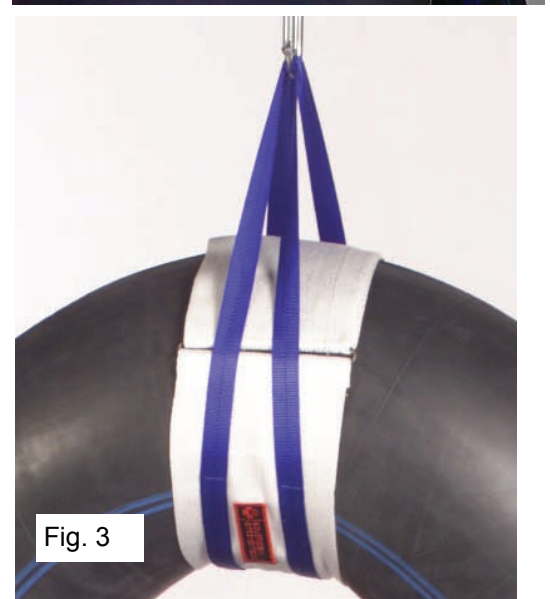


Fig. 3

MATTING PRECAUTIONS:

- Always use appropriate matting.

MAINTENANCE CHECKLIST

All of the following should be checked on a monthly basis and recorded on the checklist enclosed.

- Check for excessive wear on the blue loops where they contact the safety snap. If excessive wear or fraying is found, discontinue use until repaired.
- Check that valve stem is not poking through cover. If valve stem pokes through, discontinue use until repaired.

STORAGE PROCEDURES

- Store out of direct sunlight.
- Keep away from chemicals and household cleaners.
- Store where not accessible by clients without proper supervision.

CLEANING INSTRUCTIONS

- Check with your infectious disease department before using these procedures.
- Check for client allergies first.
- Clean tube and harness with a damp cloth using soap and water or disinfectant.
- Dry before using.

CAUTION

- ◆ **Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ◆ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401 -1047
Toll Free - (800) 228-1698
Come see us on the World Wide Web at <http://southpawenterprises.com>



southpaw enterprises, inc.®

Instruction Sheet Tube Wrap Single 0185

Store these instructions with the enclosed maintenance checklist in a safe place. You may also access them on our website.



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

Warning: Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

MATERIALS LIST:

TUBE WRAP SINGLE (0185)
1 - Tube Wrap
1 - Valve Stem Sleeve
1 - Instruction Sheet
1 - Maintenance Checklist

TUBE WRAP SET OF TWO (0186)
2 - Tube Wraps
2 - Valve Stem Sleeves
1 - Instruction Sheet
1 - Maintenance Checklist

ASSEMBLY PROCEDURES:

Inflate tube to desired size.

Place the hole in the Tube Wrap over the valve stem with the valve stem pointing toward the short end of the wrap.

Slip the valve stem sleeve over the valve stem.

Wrap the Tube Wrap around the tube and press the hook and loop fastener together securely. Check to make sure the valve stem is under the foam padding and between the two strips of hook and loop fastener. If not, readjust the wrap before using.

MAINTENANCE CHECKLIST:

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed.

Check Tube Wrap for loose or missing stitches. If loose or missing stitches are found, discontinue use until repaired.

Check hook fastener for lint or debris. If lint is found in the hook fasteners, it may be removed with a brush or pick.

MATTING PRECAUTIONS:

Always use appropriate matting.

As with all suspended equipment we suggest the client wear a helmet.

STORAGE PROCEDURES:

Store out of direct sunlight.

Store away from all chemicals and household cleaners.

Store where not accessible by clients without proper supervision.

CAUTION: Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.

Floor mats of proper thickness should be positioned properly under and around all suspended equipment.

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



Southpaw Enterprises Inc.®

If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.

P.O. Box 1047, Dayton, Ohio 45401 -1047

Toll Free - (800) 228-1698

Come see us on the World Wide Web at <http://southpawenterprises.com>