



PREMIER SERIES STOOLS

MODELS: 2155 & 2155-21



ASSEMBLY INSTRUCTIONS

ASSEMBLY INSTRUCTIONS KEY SERIES & STANDARD SERIES STOOLS

MODELS: 2155 & 2155-21

MAXIMUM WEIGHT CAPACITY 250 LBS 

Thank you for counting on Clinton to meet your medical equipment needs by purchasing this quality equipment. All Clinton equipment is designed and manufactured to provide years of dependable service because we want you as a customer now and in the future. Below are easy to follow instructions on how to properly assemble your equipment. Please contact us at 800-441-9131 if you experience any difficulty with the assembly or have questions.

MODELS: 2155



MODELS: 2155-21



Contents

- A. Seat
- B. Cylinder
- C. Base
- D. Casters (5)

- E. Height Adjustment Lever
- F. Backrest
- G. J-Bar

Tools

- May require a Rubber Mallet

Time Required

- 10 – 15 Minutes

INTENDED USE. Clinton stools are specifically designed and engineered for professional use. Stools are intended for seated tasks. **DO NOT STAND ON STOOL.** Standing on the stool could result in serious injury. Combination of weight and force exerted on the stool is not to exceed a total of 250 lbs. Using the stool for unintended purposes could result in damage to the equipment and serious injury.

IMPORTANT: Stools with casters move. Hold the stool in a stable position before attempting to sit. Sitting without holding the stool could result in the stool moving which could cause a fall and potential injury for the person attempting to sit on the stool.

1

Firmly grip each caster and push into place on bottom of the 5-leg, aluminum base.



2

Invert seat and place on surface that will not damage the vinyl top. Place the button end of the adjustable cylinder into the opening of the seat tube. Press firmly. If you are having difficulty inserting the cylinder, use a rubber mallet to tap it gently into position. The cylinder should not fall out when the seat is inverted.



3 Insert the seat and cylinder assembly into the base and press firmly. Base should not detach from cylinder when stool is lifted. Then carefully sit down on the seat until the cylinder locks in place. The seat will be tipped 2 degrees toward the front. It is designed this way to relieve pressure on the thighs.



4 Using the lever under the seat, adjust the seat to the height that's best for you.



FOR MODEL WITH BACKREST (-21)

5 Turn the stool on its side and insert the J-bar into the backrest flange on the seat plate. Position at desired depth and insert the knob through the slot and into the threaded hole in the seat. Tighten the knob firmly.



FOR MODEL WITH 26 EXTRA TALL CYLINDER OPTION AND 22 FOOT RING OPTION

6 Before placing the optional 26 Lift Cylinder into the base, place the Foot Ring on the cylinder (making sure the chrome ring is facing the stool top) 1/2 the way up the cylinder and lock in place. To adjust the height position, loosen knob, raise or lower foot ring and then secure by tightening locking knob.



TROUBLESHOOTING

PROBLEM: Height adjustment, lever is not operating.

SOLUTION: The button on the adjustable cylinder is not pushed into the seat tube completely. Push down on the padded seat of the stool to seat the cylinder more firmly in the seat tube.

NOTE: Person on the stool needs to weigh at least 90 lbs. to adjust the height of the seat.

IMPORTANT

FOR SAFETY PURPOSES, TAKE TIME TO CHECK STOOL TO INSURE THAT THE STOOL HAS BEEN PROPERLY ASSEMBLED ACCORDING TO THESE INSTRUCTIONS AND THAT ALL PARTS ARE SECURE.

Warranty

Clinton Industries warrants manufactured products to be free of defects in materials, workmanship or design under normal use for five years on all mechanical products and all electrical components from the date of purchase. This warranty does not include damage resulting from accident, abuse or misuse, alteration, or negligence of the product and expressly excludes normal wear of parts (including but not limited to vinyl). Contact Clinton for full warranty details. **ANY ALTERATIONS TO OUR PRODUCTS VOIDS ALL WARRANTIES AND CLAIMS.**



Care & Cleaning

Clinton products are built to be durable. With proper care, your Clinton equipment will provide years of superior looks and performance. Clean stool with a soft cloth and any mild cleaning agent that may be required for sanitation. **NEVER** use abrasive scrubbing agents that may scratch the surfaces. **Before using any cleaning agents, try them on a small inconspicuous area to insure that they will not harm the any surface.** Damage from cleaning is not covered under the warranty. **Do not use alcohol based cleaning agents.**

