




CLASSIC SERIES STAIRCASES

MODELS 4-5501-30 & 4-5501-36

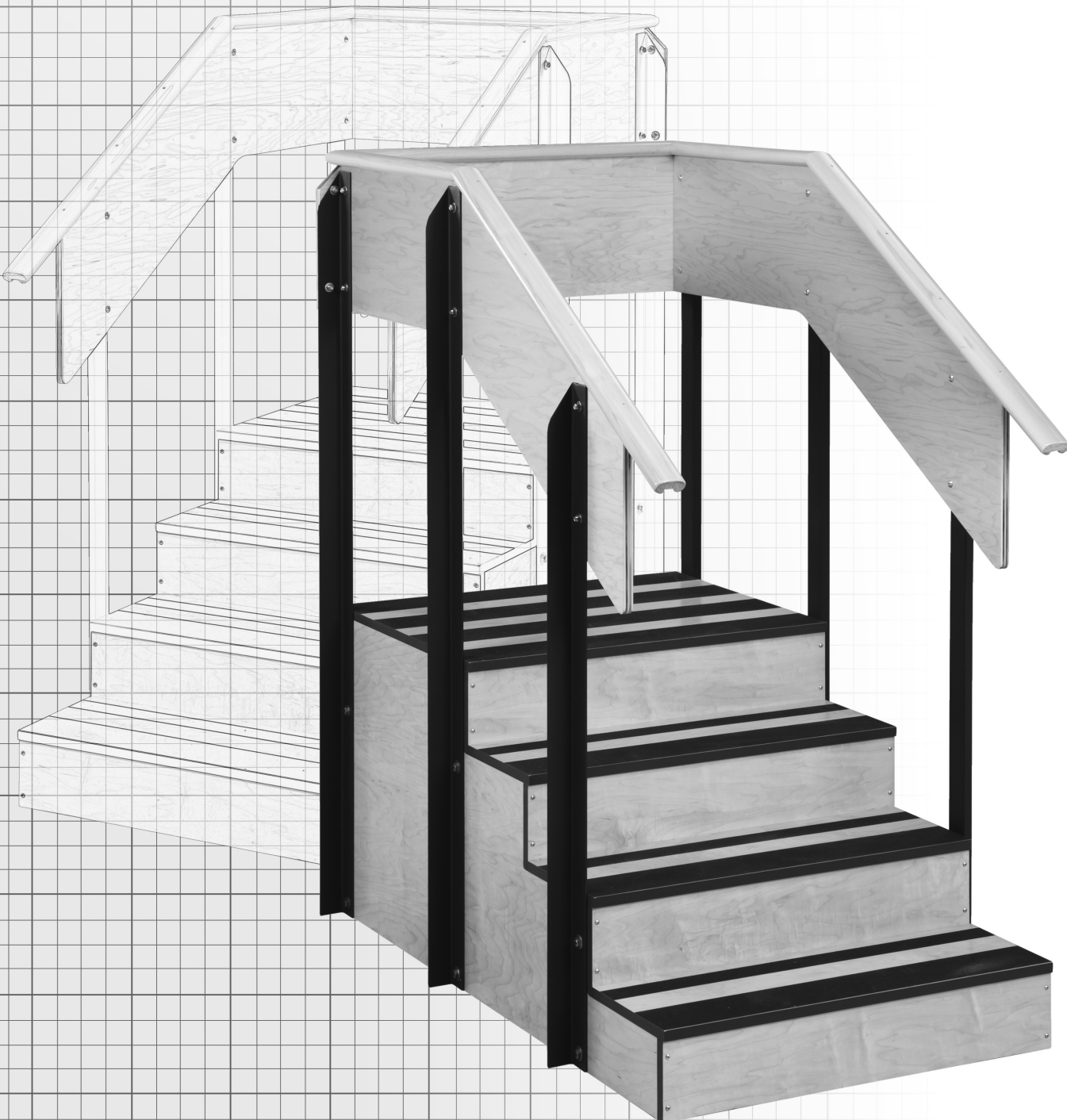

ASSEMBLY INSTRUCTIONS

ASSEMBLY INSTRUCTIONS FOR CLASSIC SERIES, ONE-SIDED STAIRCASES

MODELS 4-5501-30 & 4-5501-36 

MAXIMUM WEIGHT CAPACITY 500 LBS

Thank you for counting on Clinton to meet your physical therapy needs by purchasing this fine equipment. All Clinton equipment is designed and manufactured to provide years of dependable service because we want you as a customer now and in the future. Below are easy to follow instructions on how to properly assemble your equipment. Please contact us at 800-441-9131 if you experience any difficulty with the assembly or have questions.

INTENDED USE. Clinton Classic Series, One-Sided Staircases are specifically designed and engineered for physical therapy use only. Persons using the staircases should be observed by trained medical or physical therapy personnel. Never allow anyone to use the staircases unattended. Combination of weight and force exerted on the equipment is not to exceed a total of 500 lbs. Using this equipment for unintended purposes could result in damage to the equipment and/or serious injury to persons or persons using the equipment.



Contents

- A. Upright Posts
(consisting of lettered posts A through F)
- B. Platform
- C. Left Handrail
- D. Right Handrail
- E. Rear Handrail

Hardware

- (4) 1 1/4" Phillips Head Bolts
- (16) 7/16" Washer Nuts
- (12) 9/16" Washer Nuts

Tools Required


- Adjustable Wrench or Ratchet
- Phillips Screwdriver

Time Required

- 20 – 30 Minutes

1

Remove protective black bolt covers from both sides of the platform and both right and left handrails

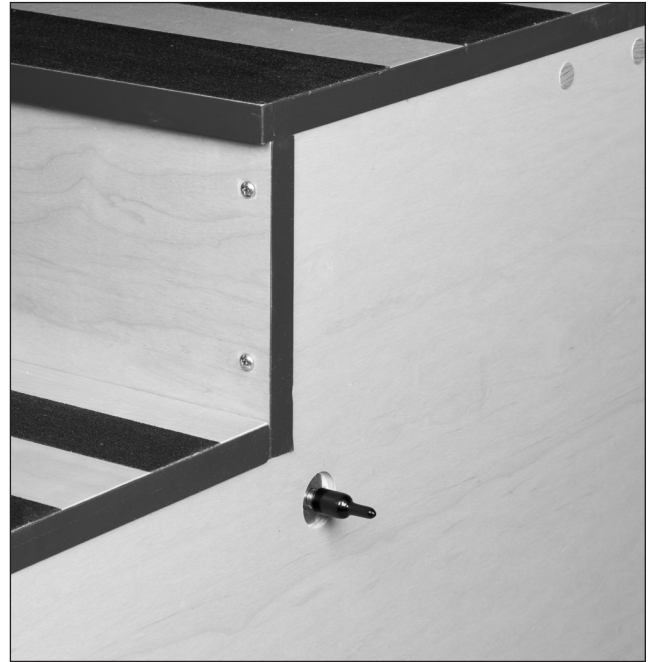


2


Match lettered steel uprights with the letters on the platform and secure with the 9/16" washer nuts—but do not tighten all the way. Repeat the procedure for the opposite side of the platform.



3 Once all the uprights are in place, it is important that you attach the rear handrail **FIRST**. Position the rear handrail on the outer side of the rear uprights and put the four (4) 1-1/4" Phillips head bolts through the wooden handrail and the steel uprights.

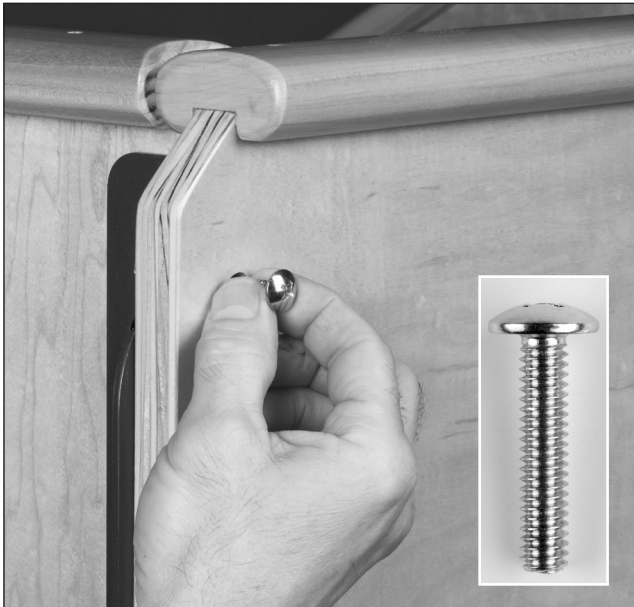


4 Secure the rear handrail with four (4) 7/16" washer nuts. **DO NOT OVER TIGHTEN.**

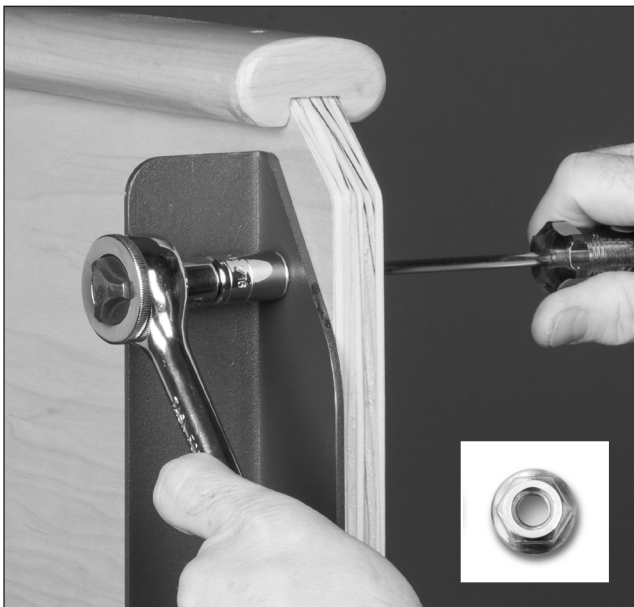

5 After the rear handrail has been secured, position the bolts on the right and left handrails so they line up with the holes on the uprights and secure with the remaining twelve (12) 7/16" washer nuts.



6 After the right and left handrails have been fully tightened, go back and fully tighten all of the uprights at the base.

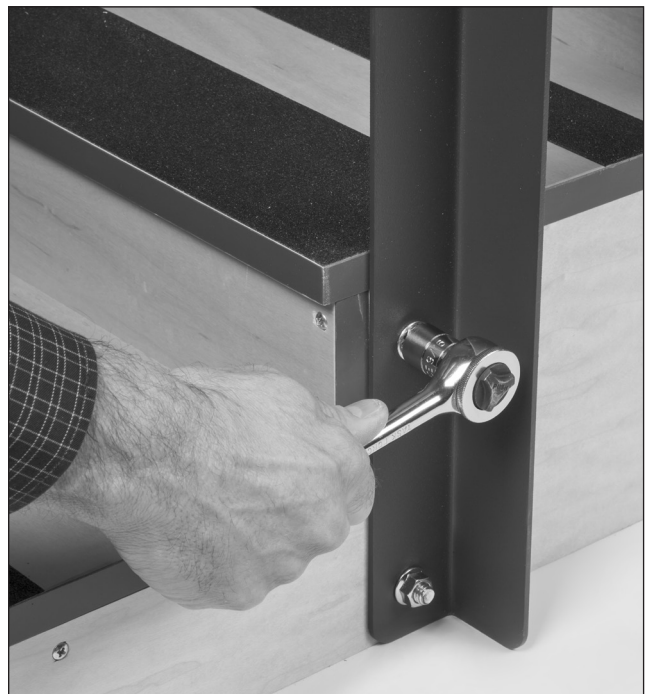

FOR SAFETY PURPOSES, TAKE TIME TO CHECK ALL BOLTS AND NUTS TO INSURE THAT THEY ARE SECURELY TIGHTENED AND THAT THE EQUIPMENT HAS BEEN PROPERLY ASSEMBLED ACCORDING TO THESE INSTRUCTIONS. CLEAN ACCORDING TO YOUR FACILITY'S STANDARDS BEFORE USE.
(SEE CLEANING INSTRUCTIONS ON BACK.)

Warranty

Clinton Industries warrants its equipment to be free of defects in workmanship for five years from the date of purchase. This warranty does not include damage resulting from accident, abuse or misuse, alteration, or negligence of the product and expressly excludes normal wear of parts. Contact Clinton for full warranty details.



Care & Cleaning

Clinton physical therapy staircases are built to be durable. With proper care, your Clinton equipment will provide years of superior looks and performance. Before using any cleaning agents, try them on a small inconspicuous area to insure that they will not harm the surface. Damage from cleaning is not covered under the warranty. **Do not use alcohol based cleaning agents.**

Periodically check that all fasteners are secure and operating properly.

