




# Spine Bracing System

## PATIENT APPLICATION INSTRUCTIONS

### EXOS FORM™ II 626 / 627

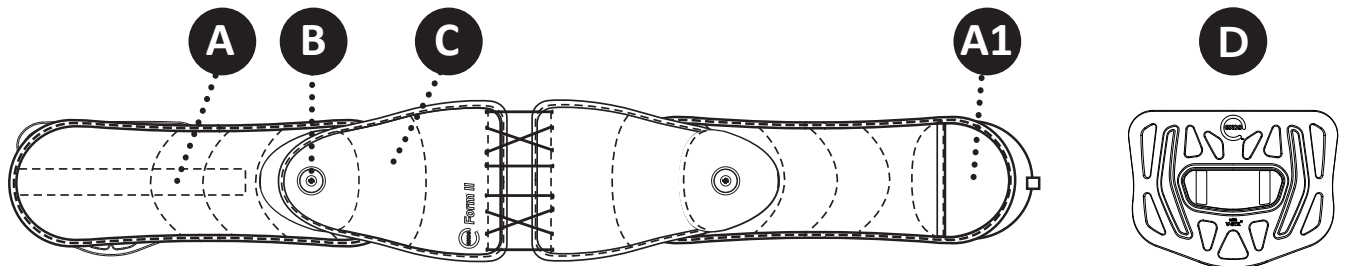

### Application of Brace:

**1 LOOSEN BRACE** by pulling out on both Boa Knobs (B) and pull Rear Panel Section (C) in opposite directions.



**2a APPLY BRACE** by inserting hand into the Donning Mitt pocket (A1) and wrapping the Belt Wings (A) around the waist.



**2b NOTE:** Center Rear Panel Section on back. If applicable, the Exos logo on the Front Panel (D) (included with model 627) should be facing upright and away.



**3 TIGHTEN BRACE** by pushing in both Boa Knobs and turn knobs clockwise.



**NOTE:** Left Boa Knob controls top half of brace. Right Boa Knob controls bottom half.

**OPTIMAL COMPRESSION:** When sitting down, prior to sitting, release compression by pulling-out on Boa Knobs. Once seated, tighten compression by pushing in on Boa Knobs. When standing up, tighten compression if necessary.

**BRACE REMOVAL:** Loosen brace compression by pulling-out on Boa Knobs. Detach hook and loop connection on Belt Wings and remove brace.

**USE AND CARE (CLEANING & STORAGE INSTRUCTIONS):** Hand wash in cold water, less than 86°F (30°C), using mild soap. Air dry. **NOTE:** If not rinsed thoroughly, residual soap may cause irritation and deteriorate material. Do NOT iron. Do NOT expose to heat over 120°F (49°C). Do NOT tumble dry. Do NOT bleach.

**MATERIAL CONTENTS:** Nylon, Polyethylene, Polyester, Antimicrobial, Polyurethane, Stainless Steel, Polycarbonate, Thermoplastic Elastomer, Polyoxymethylene, Carbon Steel, Acrylic Adhesive. **FOR SINGLE PATIENT USE ONLY. NOT MADE WITH NATURAL RUBBER LATEX.**

**CONTRAINDICATIONS:** Pregnancy and circulation, pulmonary, cardiovascular or skeletal conditions which have risk to be made worse as a result of compression and/or pressure.



**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

### Next Appointment:

Date: \_\_\_\_\_

Time: \_\_\_\_\_