1 Check all parts • Vérifier toutes les pièces • Revise todas las piezas

EN

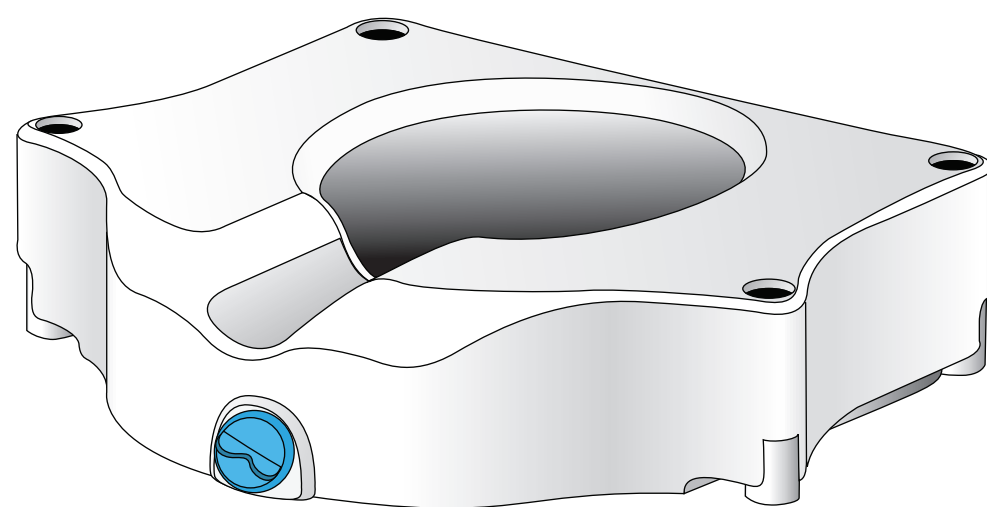
- Before setup and use, verify that all parts illustrated are present and examine them for any damages that may have occurred during shipping. Do not assemble this product if there is apparent damage.

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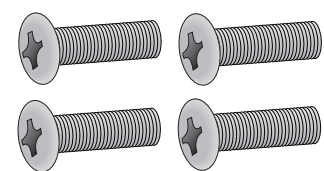
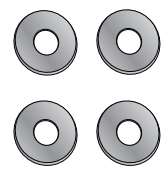
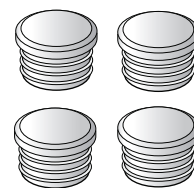
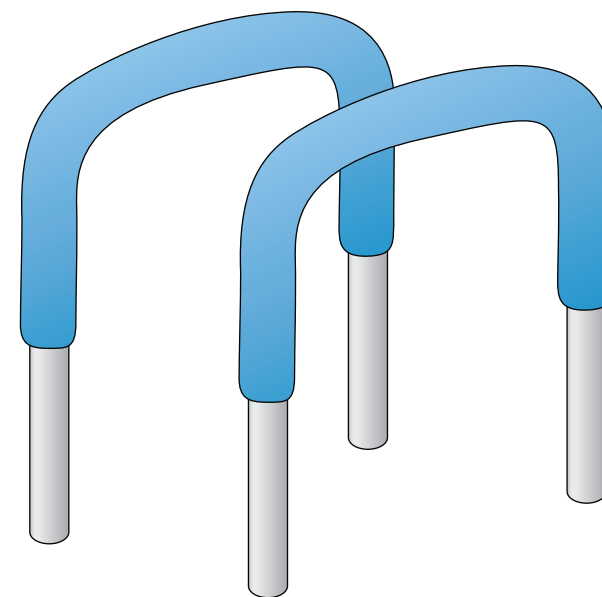
- 2 Armrests
- 2 Accoudoirs
- 2 Apoyabrazos

PREPARATION

Before installing your raised toilet seat, make sure your toilet is dry and dirt free.

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EN

This AquaSense® product is guaranteed to be free from defects in materials and workmanship for the life of the original user. This warranty does not extend to non-durable components which are subject to normal wear and replacement. Any alterations, abuse, misuse, or accidental damage voids this warranty. For replacement or repair under warranty, please call:

CANADA & INTL: 1-800-363-2381 (Monday to Friday, 8 am to 5 pm EST)
USA: 1-866-820-2086 (Monday to Friday, 8 am to 6 pm EST)

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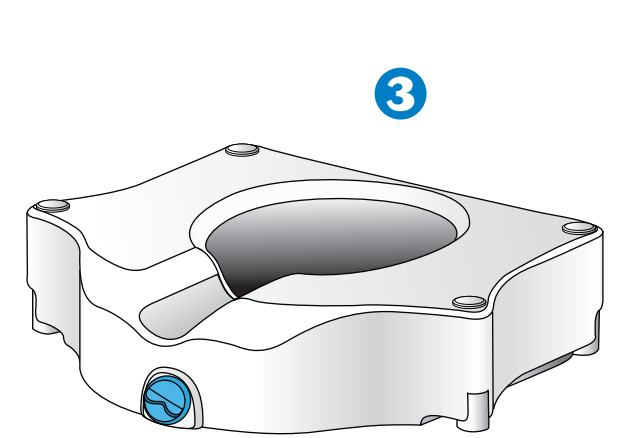
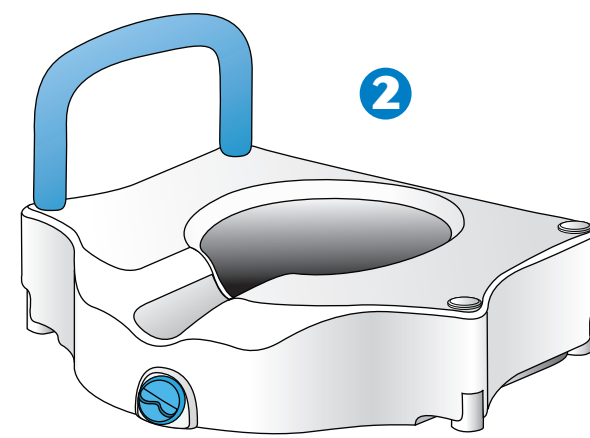
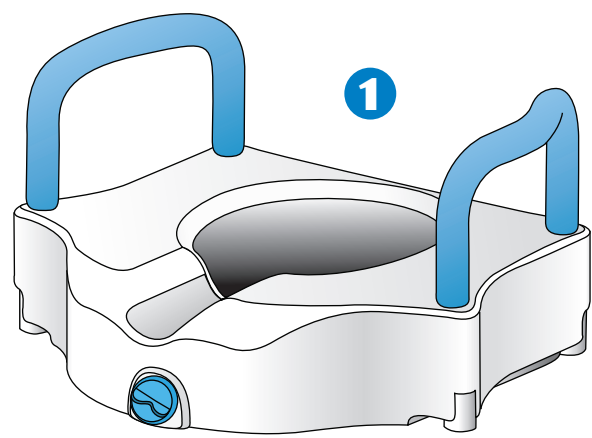
B

ARM ASSEMBLY (IF DESIRED)

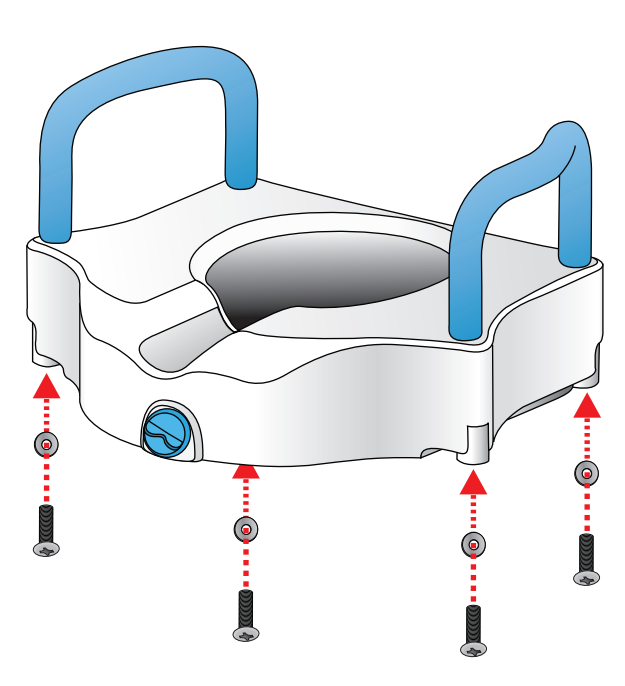
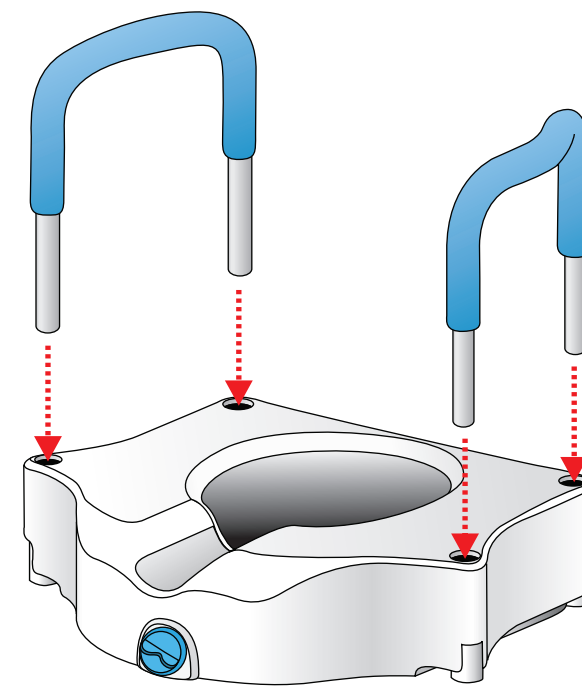
- Slide armrest into the holes in the Raised Toilet Seat.
**Make sure to slide the handles in evenly so that the foam is flush with the seat.*
- Insert screws with washers and tighten by turning clockwise until snug. Do not over tighten.

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C

TO INSTALL SEAT PLUGS

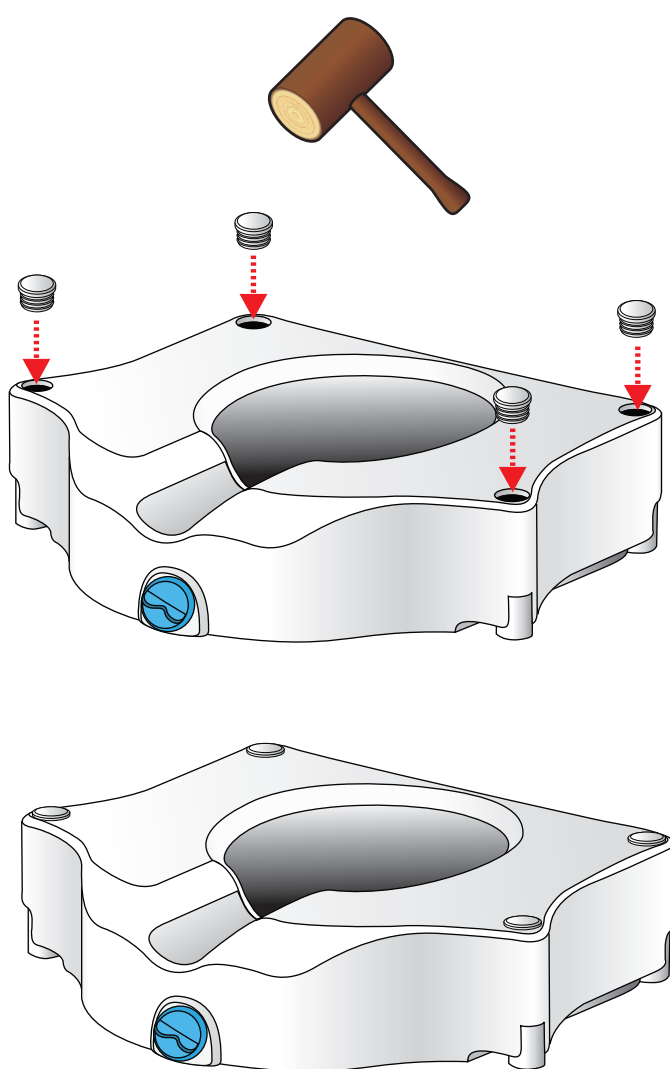
- Align seat plugs with the holes in the Raised Toilet Seat.
- Tap completely into hole using a mallet or hammer.

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D

SECURING THE SEAT TO THE TOILET

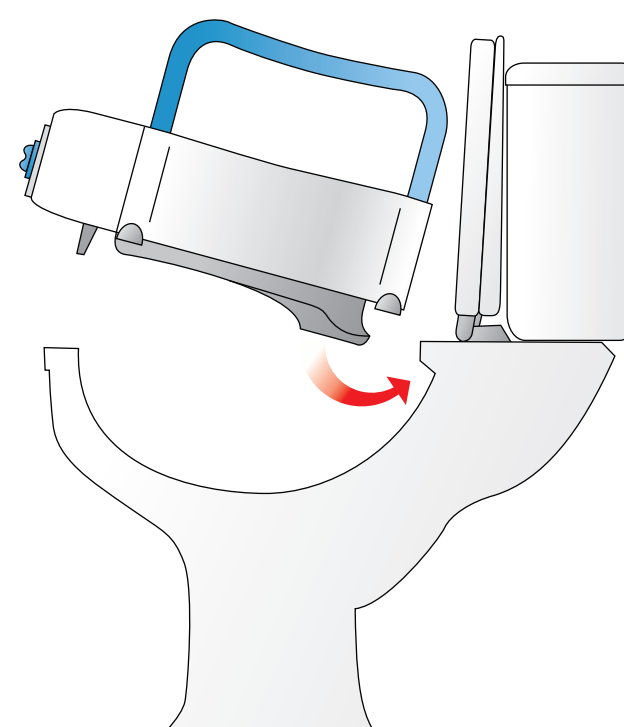
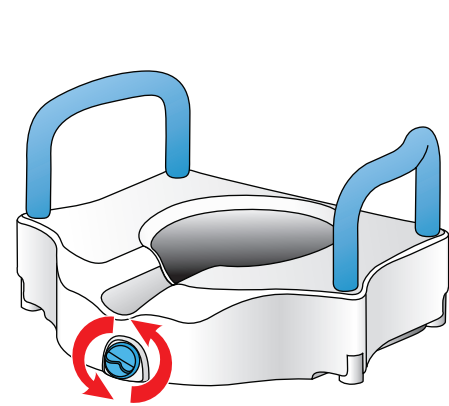
- Turn the blue adjustment knob counter-clockwise until it will no longer turn.
- Lift the lid and seat of the toilet. (Some toilet bowls may require you to remove the lid and/or seat entirely to allow proper installation)
- Place the Raised Toilet Seat onto the rim of the bowl with the knob facing forwards. Make sure the rear flanges are under the rear edge of the rim.
- Secure the Raised Toilet Seat by turning the blue adjustment knob clockwise until the seat is secure. Do not over tighten.

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E

REMOVAL

- Rotate the blue adjustment knob counter clockwise until the Raised Toilet Seat is loose.
- Lift the front of the Raised Toilet Seat until it is released from the bowl rim, then pull forward and remove.

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Precautions

EN

- Before using, make sure the Raised Toilet Seat is securely fixed to the toilet bowl.

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