# Etac Hi-Loo

med kantstopp  
with brackets

etac®

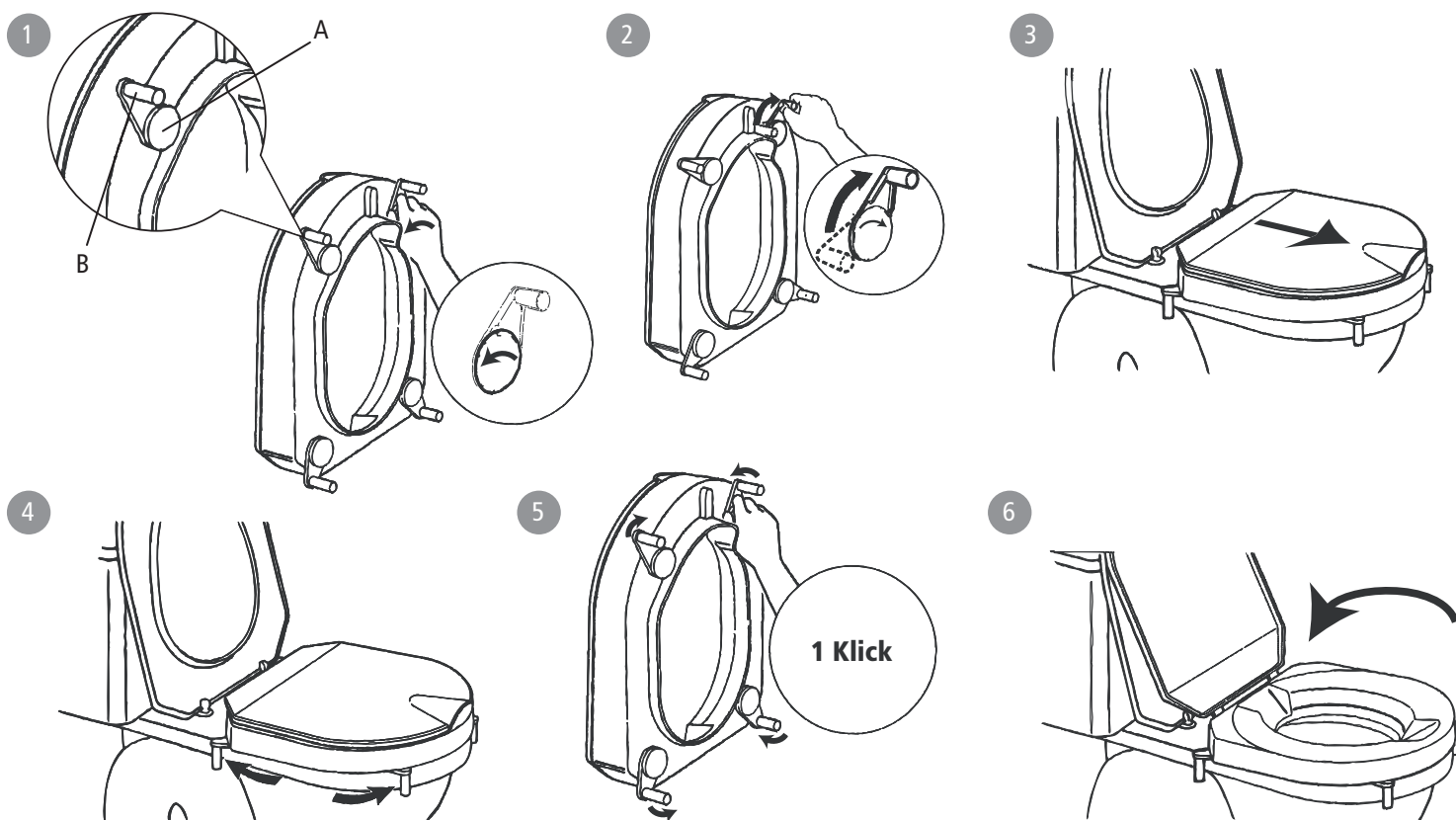
Manual

78271F 17-01-10


sv no da en de fi fr nl it es pt


## A

## B
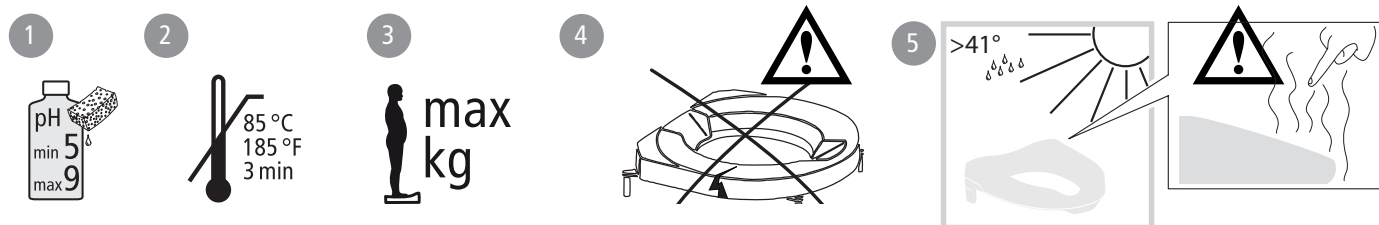
## C

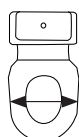6 cm	360 mm	390 mm	200 mm	270 mm	1.1 kg	150 kg
10 cm	360 mm	390 mm	200 mm	270 mm	1.5 kg	150 kg

**sv Svenska**

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**en English**

**General**

The Etac Hi-Loo with brackets is a toilet seat raiser designed for use on the toilet bowl. It has been developed to facilitate toilet use by persons who have restricted mobility and who need a raised toilet seat.

**Installation and use ..... Figur A**

1. Undo the four fixing screws. The screws should be loosened only. Do not remove completely!
2. Move the four brackets towards the outside of the seat. Tighten the screws so that it still will be possible to turn the brackets one more click.
3. Place the Hi-Loo on the toilet bowl and adjust it until there is contact at the front of the bowl.
4. Turn the rear and front brackets at the side towards the bowl.
5. Slowly lift the Hi-Loo away from the bowl. Turn all four brackets one click inwards. Now tighten the screws. Important: The brackets must remain in position!
6. Attach the Hi-Loo to the bowl, front brackets first. Then use downward pressure to secure the rest of the Hi-Loo to the bowl. Check it is securely attached.

**Maintenance and safety ..... Figur B**

1. Clean/disinfect the chair with normal non-abrasive cleaning agents with a pH value of between 5-9, or with 70% disinfectant solution. Rinse and dry. The product is otherwise maintenance-free.
2. The product can withstand cleaning in a sterilizer at 85°C for 3 minutes. This applies for reconditioning.
3. Maximum user weight (see technical data).
4. A defective product must not be used.
5. Heated chair can cause burns.

The materials used are corrosion-resistant. The materials used are resistant to all general disinfectants.

**Guarantee, service life**

5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com.

Service life 10 years. For complete information regarding the service life of the product, see www.etac.com.

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R82 UK Limited.
Snug Seat, Inc.
Etac Supply Center AB