
Etac Swift Mobil-2 @etac®


78691.13 23-01-18



Α

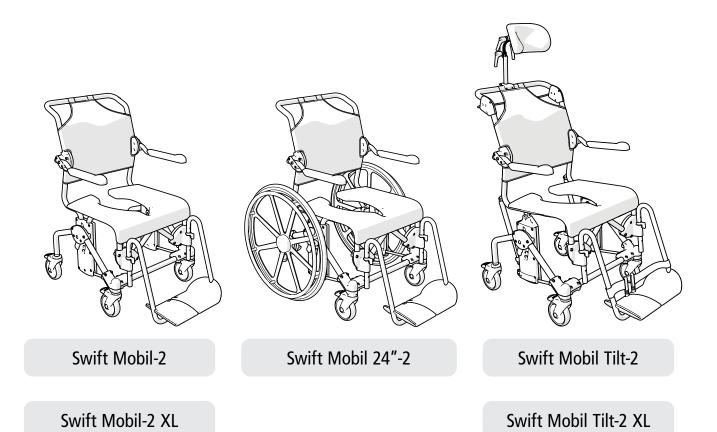


Swift Mobil-2

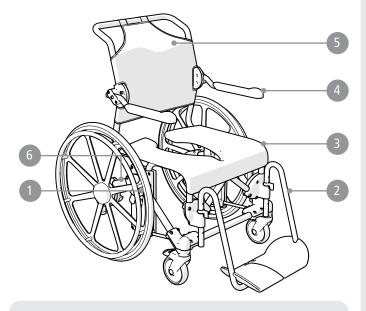

Swift Mobil 24"-2


Swift Mobil-2 XL

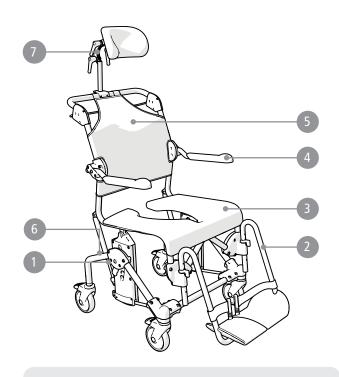


Swift Mobil Tilt-2

Swift Mobil Tilt-2 XL

В

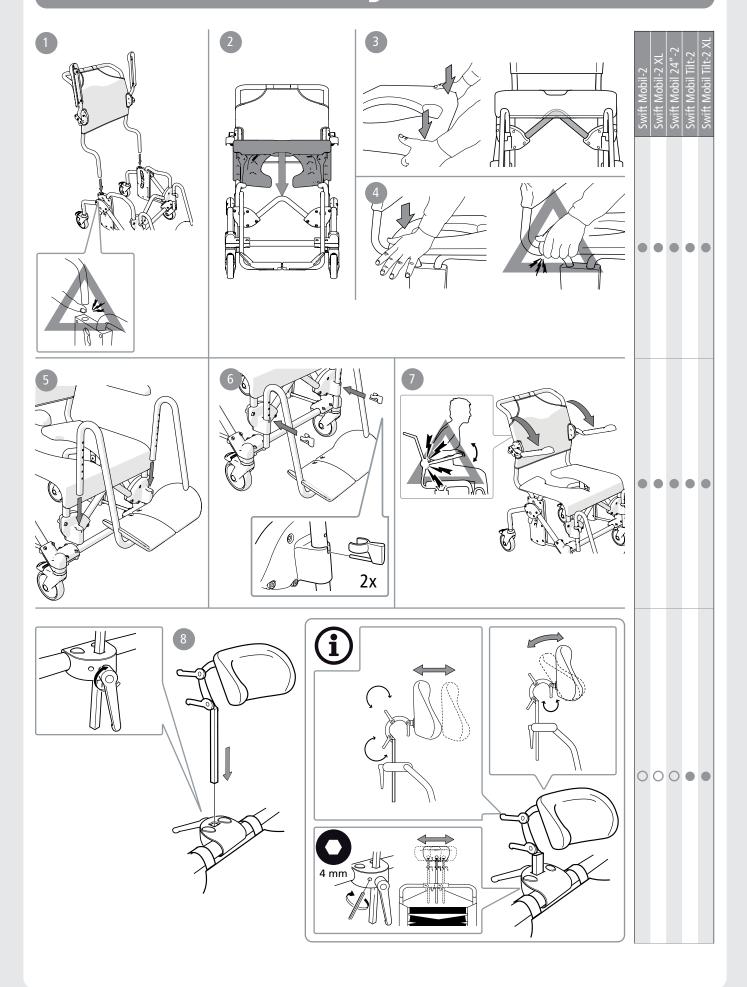

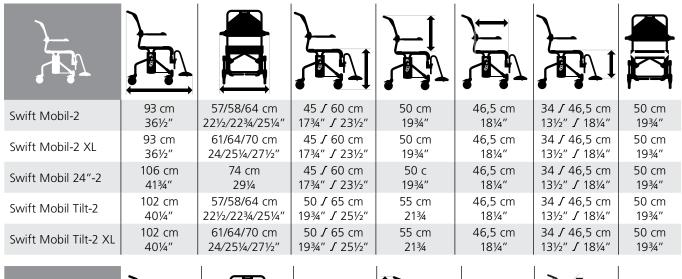


							KG
Swift Mobil-2	22/25/28 cm 8¾/9¾/11"	42/48/54 cm 16½/19/21¼"	54 cm (47 cm*) 21¼" (18½"*)	95-110 cm 37½"-43¼"	125 mm 5"	-	16,6 kg 37 lbs
Swift Mobil-2 XL	22/25/28 cm 8¾/9¾/11"	48/54/60 cm 19/ 21¼/23½"	54 cm 21¼"	95-110 cm 37½"-43¼"	125 mm 5"	-	17,3 kg 38 lbs
Swift Mobil 24"-2	22/25/28 cm 8 ³ ⁄⁄/9 ³ ⁄⁄/11"	42/48/54 cm 16½/19/21¼"	67 cm 26¼"	95-110 cm 37½"-43¼"	125 mm 5"	610 mm 24"	25,1 kg 55 lbs
Swift Mobil Tilt-2	22/25/28 cm 8 ³ ⁄⁄9 ³ ⁄⁄/11"	42/48/54 cm 16½/19/21¼"	60 cm 23½"	100-115 cm 39¼"-45¼"	125 mm 5"	-	20,4 kg 45 lbs
Swift Mobil Tilt-2 XL	22/25/28 cm 8 ³ ⁄ ₄ /9 ³ ⁄ ₄ /11"	48/54/60 cm 19/ 21½/23½"	60 cm 23½"	100-115 cm 39¼"-45¼"	125 mm 5"	-	21 kg 46 lbs

^{*}Art no 8022 9489, 8022 9490, 8022 9491

	<u></u>	max (g		nax os		
Swift Mobil-2	160 kg	135 kg**	352 lbs	297 lbs**	ø 990 mm	
Swift Mobil-2 XL	160 kg		352 lbs		ø 990 mm	
Swift Mobil 24"-2	160 kg	135 kg**	352 lbs	297 lbs**	ø 1210 mm	
Swift Mobil Tilt-2	160 kg	135 kg**	352 lbs	297 lbs**	ø 1040 mm	
Swift Mobil Tilt-2 XL	160 kg		352 lbs		ø 1040 mm	

^{**} Art No 8022 9477, 8022 9478, 8022 9479, 8022 9486, 8022 9487, 8022 9488, 8022 9491, 8022 9494

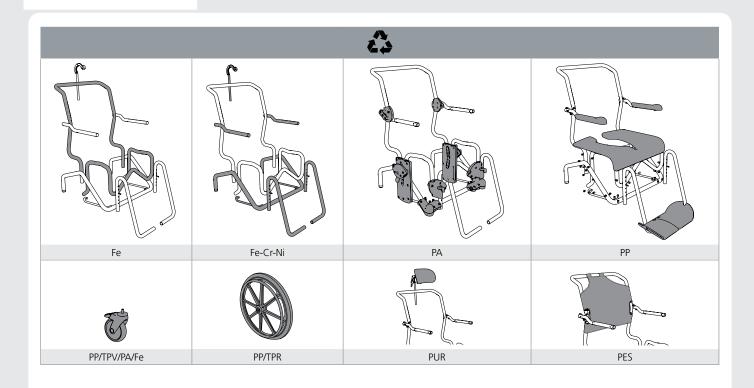

D









Ŀ






























F

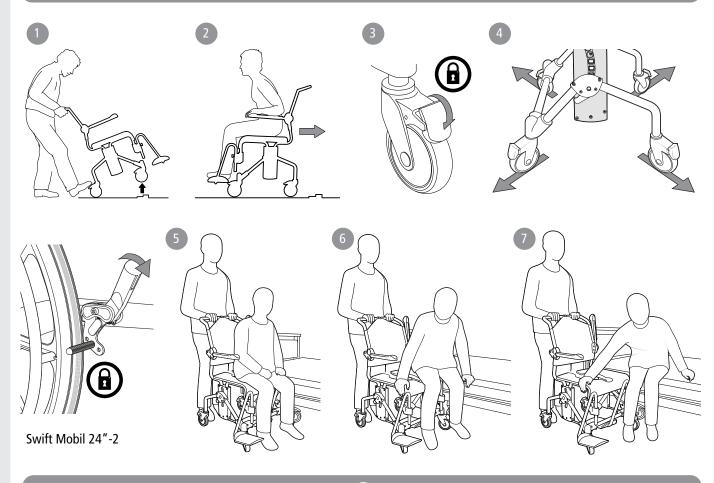



G

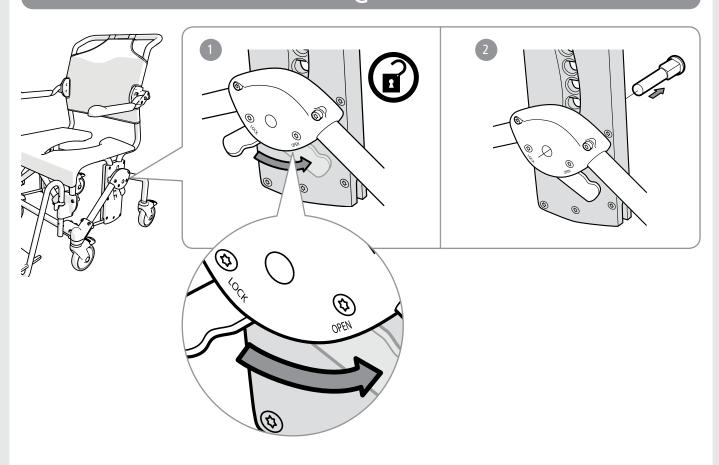


G

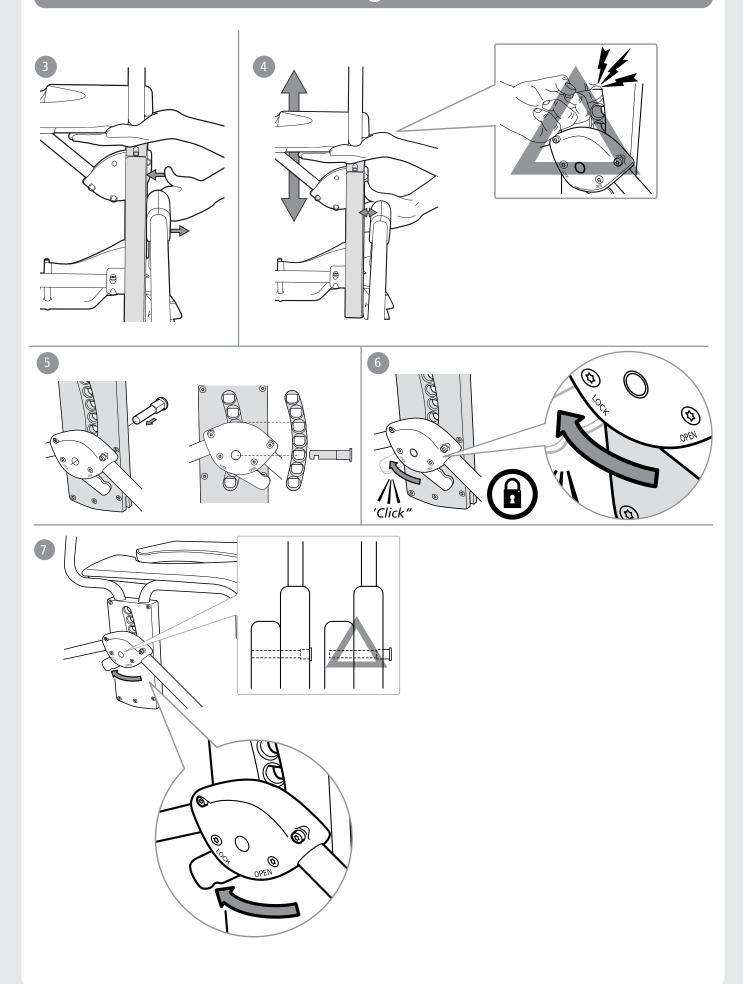

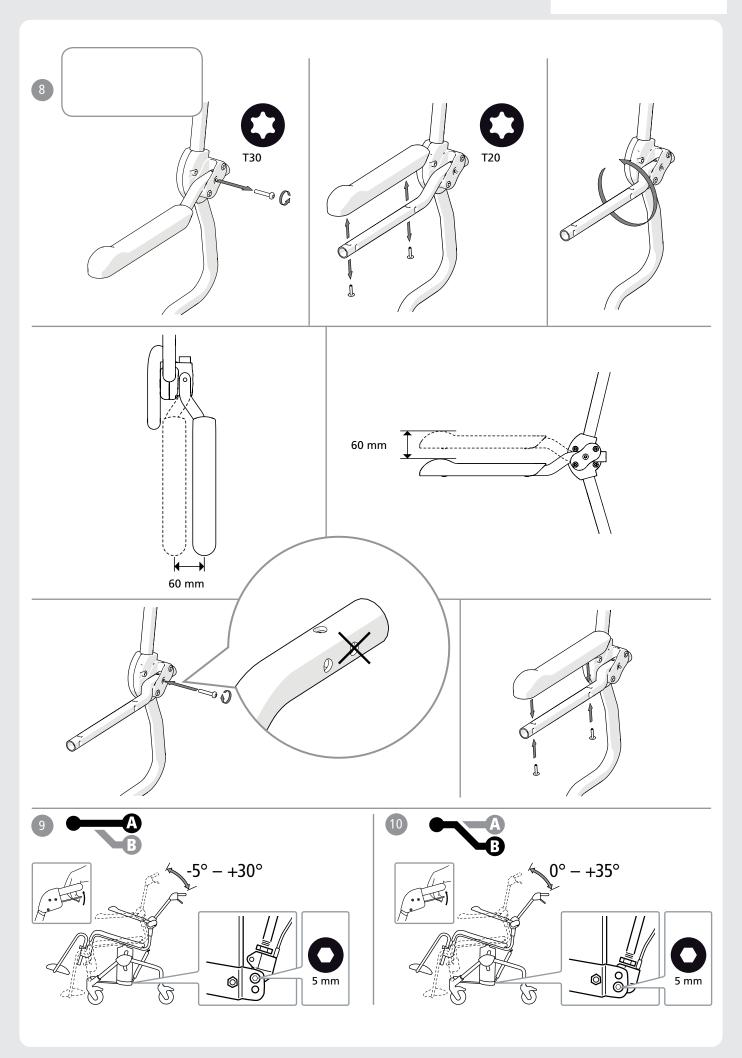

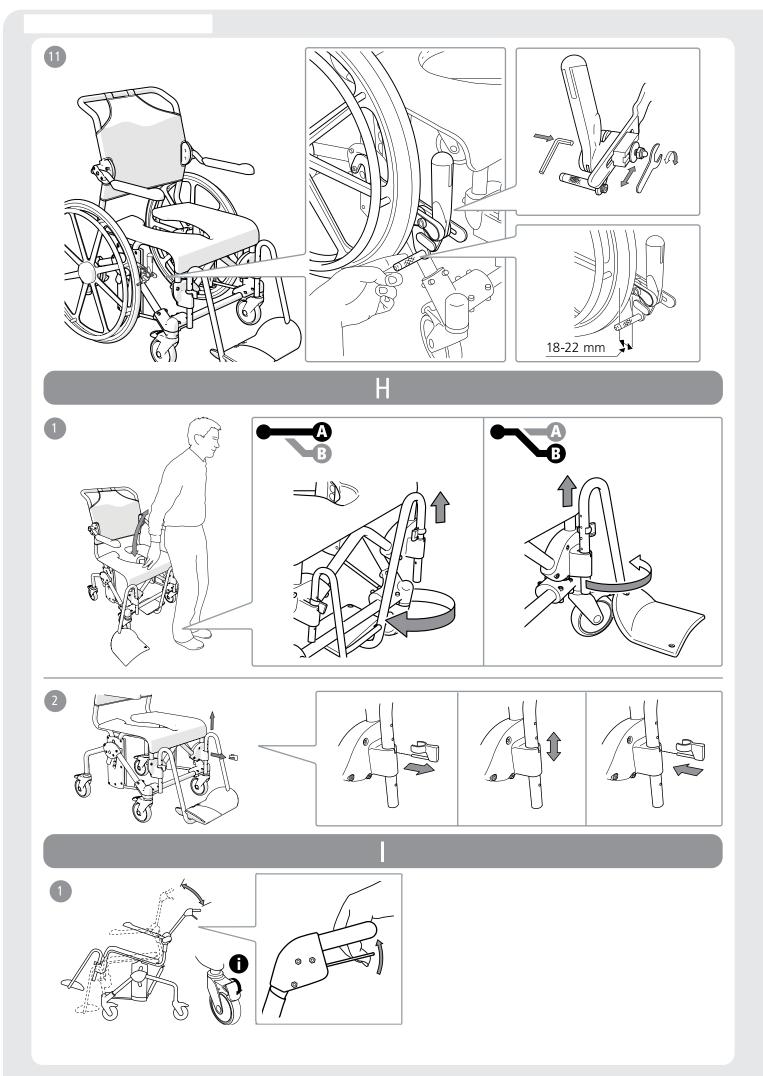

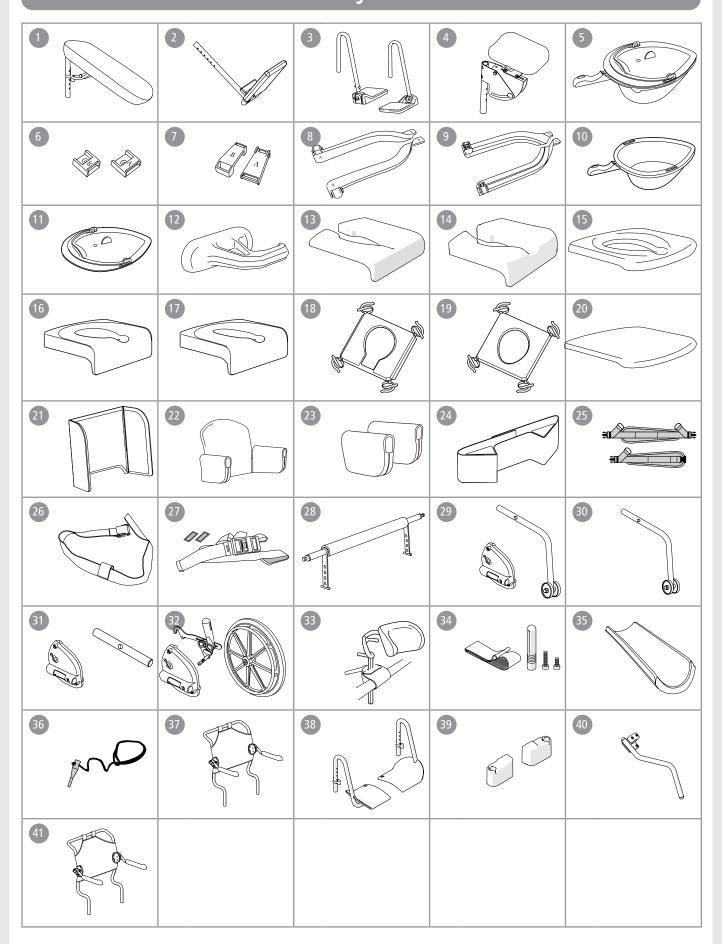

English

General

Thank you for choosing an Etac product. In order to avoid damage during assembly, handling and use, it is important to read this manual and save it for future reference. You can also find it at

You can select your language via the "International" and "Local websites" link. Here you will also find other product documentation, such as prescriber information, pre purchase guide, and reconditioning instructions.

In the manual, the user is the person sitting in the chair. The carer is the person helping the user.

Intended use:

The Swift Mobile 2 family is a range of mobile shower chairs and toilet chairs (hereafter called the product) intended for indoor use when performing hygiene tasks for people with restricted mobility. The product can be used in the shower, at a sink, or over the toilet, as well as for transfers to and from the hygiene room.

The product is intended for use in home environments and institutions

The product is suitable for use in wet areas, but not pools or similar corrosive environments.

The product is intended to compensate for various types of injuries or disabilities that limit the mobility of the user.

The product is intended for adult users, generally persons who are at least 146 cm tall and who weigh at least 40 kg.

The product is not intended for any other use or environment beyond those stated above.

The product has been tested and complies with the requirements set out in EN ISO 10993-1 and ISO 17966.

The product conforms to the requirements of the Medical Devices Regulation (EU) 2017/745.

Contra indications:

There are no known contraindications.



This symbol appears in the manual along with text. It indicates instances where the safety of the user or the assistant may be at risk.



The images which are referred to on each page are found at the start of the manual (pages 3 to 11). See Figures A to J for the corresponding text.



This product should be recycled as per national regulations.

Item list: = Applies to, $\bigcirc =$ Accessories

See the item list on the right which displays the information applicable to each product.

Product description

See Figure A

A1. Bolt, A2. Leg support, A3. Seat, A4. Arm support, A5. Back support upholstery, A6. Item no label (serial no, production date)

A7. Head support

*The product's date of manufacture can be read from the barcode on the product. The number 11 is shown below the barcode in brackets. The number combination after these brackets is the date of manufacture.

Assembly See Figure B

Technical data See Figure C

Warranty / maintenance

See Figure D

Reuse

The product is suitable for reuse. The number of times it can be reused depends on how often and in which way the product is used. Before euse, thoroughly clean and disinfect the product and check its safety features. If necessary, take remedial action.

Storage

The product should be stored indoors in a dry place at a temperature above 5°C. If the product has been stored for a long time (more than four nonths), its function must be checked by an expert before use.

English



🔨 Safety

00



- Defective chairs must not be used.
- **E6.** Risk of crushing with arm supports. **E7.** Do not use defective products.
- **E1.** Make sure the castors are locked when users sit down on or get up
- from the chair.
- An anti-tip attachment must be used if the user is an amputee. **E2.** Never stand on the foot supports as there is a risk of tipping!
- E3. The sitting position has an impact on stability.
- **E4.** Chair stability is improved if the castors point outward.
- **E5.** See the maximum user weight for your commode chair in the table (page 5).

In case of an adverse event occurred in relation to use of the device, it should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

Transfers

Over a threshold:

- F1. Chair only, put one foot on the wheel.
- F2. With user, pull the chair backwards.

In stationary position:

- F3. Always lock as many castors as possible when the chair is to be used in a stationary position.
- **F4.** Out-turned castors improve the chair's stability.

Transferring:

Assess the risks and take notes. You as a carer are responsible for the safety of the user. Aids should be used if the transfer is deemed as

- F3-F4. It is important to ensure that the castors are locked and pointing outward before carrying out transfers to and from Swift
- F5-F7. Fold up the arm support on the transfer side of the chair. Carry out the transfer.

Adjustment

Height:

G1-G7. The height is easily adjusted by inserting the bolt in the required position. See distances in the technical data section.



Check that both sides have been adjusted to the same height as otherwise stability can be reduced. G7. Do not use the chair if the bolt (orange) is not locked.

Arm supports:

G8. The height of the arm supports and/or width between the arm supports can be adjusted. Loosen the screws, twist the support to the desired position and retighten the screws. Then loosen and refit the arm support plate in the correct position. Swift Mobil Tilt-2 XL is delivered with the arm supports mounted in the most inward position (seat width 48 cm).

Tilt angle:

- **G9.** Fit the gas piston in position A to tilt the seat at angles from -5° to 30°
- **G10.** Fit the gas piston in position B to tilt the seat at angles from 0° to 35°

User brake:

G11. When adjusting the brake, the brake pin should be set 18 to 22 mm from the rear wheel for good grip.

Leg support

H1. The assistant can easily raise and swivel the leg supports to the side for the user to sit down in or get up out of the chair. The foot supports are detachable.

H2. Various height adjustment options are available by moving the locking pin. The leg supports have 6 different height settings.

Tilt function

22

11. The assistant can adjust the sitting angle with ease by holding in the handle and tilting to the required position. Make sure that the wheels are locked if the tilt function is used when the user is sitting in the chair.

English

Accessories	See Figure .

In the section "Accessories", you can see all accessory options available for your product. Variations may occur in different markets. There are restrictions on accessory combinations



See Figure J		Sw M2 Sw M2 XL Sw M. 24"-2 Sw M. Tilt-2 Sw M. Tilt-2	See Figure J		Sw M2 Sw M2 XL Sw M. 24"-2 Sw M. Tilt-2 Sw M. Tilt-2 XL
J1.	Leg and calf rest, angle-variable	00000	J23.	Armrest cushion, pair	00000
J2.	Footrest	00000	J24.	Trunk support	00000
J3.	Angle adj. footrest	00000	J25.	Chest and hip positioning belts	00000
J4.	Amputee legrest	00000	J26.	Chest belt	00000
J5.	Commode pan	00000	J27.	Positioning belt	00000
J6.	Pan-holder attachments	00000	J28.	Crossbar 605 mm	0 00
J7.	Attachments for low pan-holder mounting	00000	J28.	Crossbar 665 mm	00000
J8.	Commode pan holders	00000	J29.	Anti-tip stabilisers including bracket	00000
J9.	Bucket holders	00000	J30.	Anti-tip stabilisers excluding bracket	00000
J10.	Pan	00000	J31.	Tilter	00000
J11.	Lid with handle	00000	J32.	Rear wheel kit	00
J12.	Splash guard	00000	J33.	Headrest and bracket	0000
J13.	Soft Comfort seat, 2 cm high, open at back	00000	J34.	Heel straps	0000
J14.	Soft Comfort seat, 4 cm high, open at back	00000	J35.	Hemiplegic armrest	00000
J15.	Comfort seat	00000	J36.	Armrest lock	00000
J16.	Soft Comfort seat	00000	J37.	Extended back support	000
J17.	Soft Comfort seat - narrow opening	00000		Short leg support	00000
J18.	Soft cushion - standard opening	00000	J39.	Widening kit for leg support	00000
J19.	Soft cushion - oval opening	00000	J40.	Anti tip front	00000
J20.	Seat cover	00000	J41.	Back Swift Mobil-2, upright	000
J21.	Soft back pad	00000			
J22.	Comfort padding	0 00			