

---

---

**On/Off Jumbo Switch #7805**

---

---

**USER'S GUIDE**

### *Easy to use!*

This version of our popular Jumbo Switch works as an On/Off switch. Just press the switch to turn on a device and press it again to stop. Great for users who can not sustain their hand on a switch. Size: 5 1/8" D x 1/2 "H

#### **Operation:**

1. The Jumbo Switch does *not* require any batteries to operate.
2. Plug the Jumbo Switch into the jack on your switch-adapted toy/device.
3. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
4. Make sure that the connection between the Jumbo Switch and your toy/device is tight. There should be no gaps.
5. Simply press anywhere on the Jumbo Switch's colored top to activate your toy/device.
6. The toy/device will remain activated. Press on the Jumbo switch again, and the toy/device will turn off.

#### **Troubleshooting:**

**Problem:** The Jumbo Switch does not activate your toy/device.

**Action #1:** Make sure that the connection between the Jumbo Switch and your toy/device is tight. There should be *no* gaps. This is a common error and an easy fix.

**Action #2:** Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

**Action #3:** Try a different adapter (if applicable) to rule this out as the source of the problem.

#### **Care of Unit:**

The Jumbo Switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

**Do not submerge** the unit, as it will damage the contents and the electrical components.

**Do not use abrasive cleaners**, as they will scratch the surface of the unit.