



# Enterprise 9000X


## Attendant Control Panel (ACP)

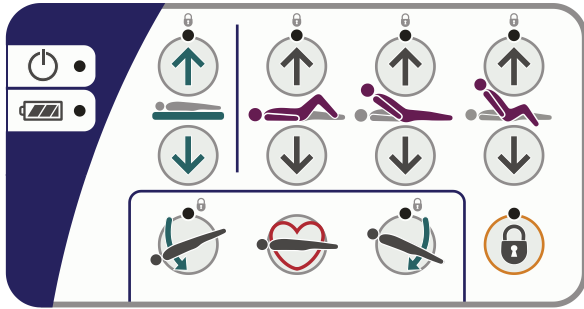


## Caregiver Controls

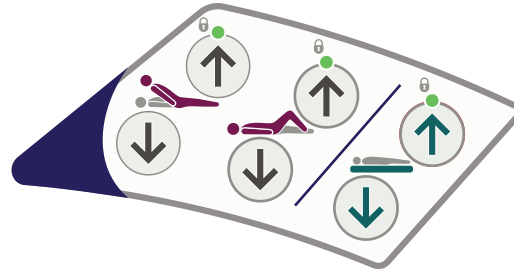

## Patient Handset

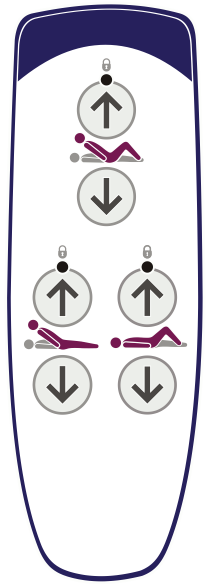

## Patient Controls

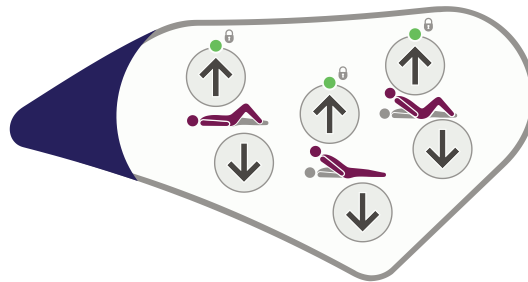

Power On indicator



Battery Status LED indicator



fully charged



charging



discharged



**CPR button** - flattens the mattress platform.



Lower head end (Trendelenburg position).



Lower foot end (reverse Trendelenburg position).



**Function Lock button** - press the button, then press selected Attendant Control Panel button(s) to lock/unlock function. Yellow light = locked



**Auto-Chair up button** - press and hold for chair position.



**Auto-Chair down button** - return mattress platform to flat and level position.



Raise/lower the mattress platform.



Raise/lower the thigh section.



Raise/lower the backrest.




**Bio-Contour up button** (patient control only) - raise backrest and thigh section.



**Bio-Contour down button** (patient control only) - return to flat position.

## Patient Weighing and VariZone Movement Detection Controls

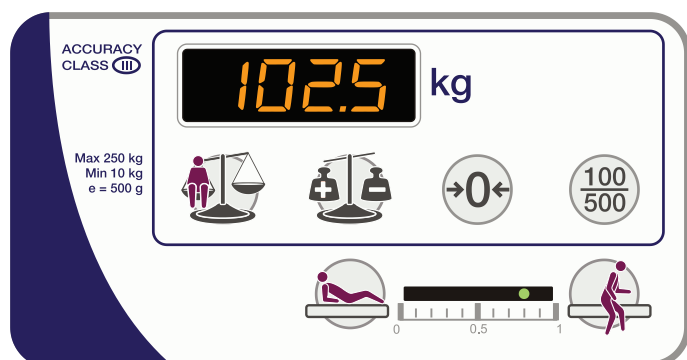

**Weigh** - calculate and display patient weight.



**Auto Compensation** - allows items to be added or removed without affecting indicated patient weight.




**Zero** - reset weighing system.



**Select Weight Display** - rounds the patient weight to nearest 100 g or 500 g.



**In bed** - activate/deactivate patient movement detection and increase the sensitivity of the system.

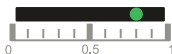

**Patient movement detection threshold display** - an indicator shows the current system status and the selected sensitivity of patient movement detection.



**Egress** - activate/deactivate patient movement detection and decrease the sensitivity of the system.