


Premium Pressure-Check™ Foam Psychiatric Mattress

Reorder #	Cover	Size
<input type="checkbox"/> 912308	Standard - Nylon	6" x 35" x 76"
<input type="checkbox"/> 912324	Perimeter-Guard - Nylon	6" x 35" x 76"
<input type="checkbox"/> 912310	Standard - Nylon	6" x 35" x 80"
<input type="checkbox"/> 912328	Perimeter-Guard - Nylon	6" x 35" x 80"
<input type="checkbox"/> 912311	Standard - Vinyl	6" x 35" x 80"
<input type="checkbox"/> 912314	Standard - Nylon	6" x 35" x 84"
<input type="checkbox"/> 912330	Perimeter-Guard - Nylon	6" x 35" x 84"

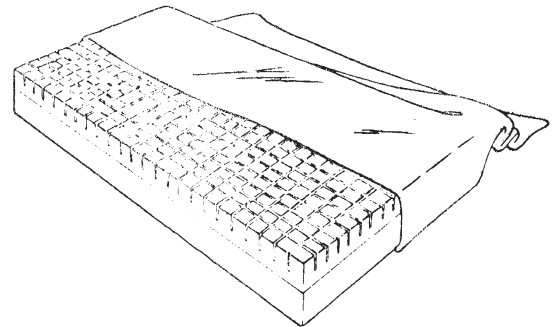

Purpose

Skil-Care's Psychiatric Mattresses are manufactured to meet the special needs of psychiatric facilities. The durable, tear-resistant nylon covers are made without zippers so that patients do not have access to the mattress foam. All seams are sewn with strong nylon thread. Skil-Care's wipe-clean nylon covers stand up to incontinence, cleaning solutions, and disinfectants.

Clinically-tested features

- Pressure-Check Foam includes nearly 400 separate foam sections that react individually to achieve the weight distribution patients require for reduced interface pressure.
- Ultra-Soft Heel Zone lowers pressure for sensitive heels.

Important Fire Code Information.

All Skil-Care mattresses sold in the United States meet the mandatory fire code 16 CFR.

Cover care

- Comply with OSHA regulations and CDC recommendations relating to blood borne pathogens. Follow accepted precautions.
- USE an EPA registered disinfectant according to manufacturer's instructions when performing disinfecting procedures.
- Before disinfecting, clean the cover surface using a soft cloth dampened with soap and water.
- Do not saturate the cover or allow cleaning solutions to pool on the cover surface.

Inspection

Inspect the cover before use. Do not use if the cover has cuts, rips, or tears.