Q chassis / ThevoTherapy

(DE)

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(GB)

Assembly instructions - Accessory

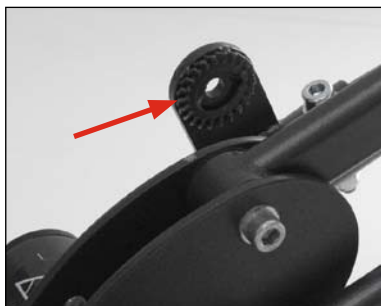
Push bar

1

2

3

4

5

6

7

8

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Push bar

Item code: 6660

The push bar facilitates indoor use. Mounting: Remove the eight tube plugs left and right at the rear of the chassis (two tube plugs each on top and bottom on either side). (Fig. 1)

Now, mount the push bar receivers left and right, using the screws, washers, and locknuts provided. (Fig. 2) Make sure the gear faces inwards. (Fig. 3)

Note:

Locknuts lose function when loosened. In case of dismantling the locknuts, we recommend using new ones when reattaching the push bar receivers.

Attach the push bar to the receivers using the release levers provided. Make sure the height adjustable part of the push bar points rearwards. (Fig. 4)

The angle of the push bar can be changed by slightly loosening both release levers right and left at the chassis. (Fig. 5)

Note:

For safety reasons, the release levers should always be positioned vertically, pointing downwards. Position can be changed by pressing the button and lifting the lever. (Fig. 6 to 8)

Caution:

Do not attach any items to the push bar! This increases the risk of tipping. The grip must only be used to push the chassis.

The logo for Thomashilfen features a stylized graphic of three curved lines above the text 'Thomashilfen'.

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