

KAYE ABDUCTORS, MODELS 106-A1 AND VS1-A3

(effective 10/15/02)

These are the user instructions for the 106-A1 and VS1-A3 abductors. All people who set-up, adjust, and use the unit should first read these entire instructions. Please store these instructions for future reference.

USE

The abductors are designed for children who cannot keep their knees separated when using a prone stander or vertical stander.

The 106-A1 abductor fits the 102, 103, 104, and 106 Kaye Prone Stander.

The VS1-A3 abductor fits the VS1 and VS1+C Kaye Vertical Stander.

ASSEMBLY AND INSTALLATION

Refer to the diagrams and install the abductors as follows:

1. Set the stander up for use.
2. Adjust the stander cushions so that there is at least 2.5 inches or 6.5 cms. of space between the cushions where you wish to attach the abductor.
3. Remove the attacher screw and then the spacer block from the bottom of the abductor.
4. From the underside of the stander (the side opposite the cushions) pass the attacher screw through the desired attachment/adjustment hole.
5. Slide the spacer block onto the projecting attacher screw. (For the 106-A1, the wide part of the block should rest against the stander. For VS1-A3, the larger slot should rest against the stander.)
6. Position the abductor so that the narrow end points upward and screw the attacher block into any of the three holes in the abductor assembly.
7. Tighten the attacher screw while ensuring that the stander and the abductor are recessed in the slots of the spacer block.

HEIGHT ADJUSTMENT

Position the abductor along the inside of the user's thighs and knees. Do not position it below the knees.

There are two options for changing the height or position of the abductor:

1. Remove the attacher screw and place it through an alternative adjustment hole in the stander.
2. Secure the attacher screw to any one of the three adjustment holes in the abductor.

WIDTH ADJUSTMENT

The abductor will align the legs and separate the knees best when the feet are

positioned 6 to 8 ins., or 15 to 20 cms., apart.

Refer to the diagrams and adjust the width of the abductor as follows:

1. Loosen the adjustment knobs.
2. Reposition the cushions.
3. Re-tighten the knobs.

To locate the abductor over the knee cushion, place the spacer block immediately below the knee cushion and use the bottom of the abductor's three adjustment holes.

If you have two knee cushions, one above and one below the knees, locate the spacer block between the two cushions and use the bottom of the abductor's three adjustment holes.

MAINTENANCE AND CARE

Inspect the unit regularly. Tighten the hardware as necessary.

If the unit needs service or spare parts, please contact Kaye Products, Inc. or the distributor from whom you purchased the stander.

If a problem is discovered that may impact the unit's function, immediately cease use and contact Kaye Products, Inc.

Do not expose the unit to rain or submerge it in water.

Use a slightly damp cloth to clean the unit, but wipe the dampness from the wood after cleaning it.

Avoid any undue stress to the unit while using, storing, or transporting it.

LIMITED WARRANTY

If an item proves defective within two years of the original purchase, we will provide replacement parts in order to correct that defect. Normal wear and tear is not covered by the warranty.

Kaye Products, Inc. makes no other warranty, expressed or implied, and does not warrant the product as being fit for a particular purpose. The purchaser, owner, and user assume all risk of personal and property injury due to the use of the equipment.

CAUTIONS

1. Do not adjust the unit while it is in use.
2. Watch traffic patterns around the stander to ensure that others do not bump or knock the stander.
3. Do not use the unit if there are broken or missing parts.
4. Do not alter the unit or use it in any way other than described in these instructions.
5. Before use, always ensure that all of the hardware is fully tightened.

6. Do not leave the user unattended.
7. Use qualified supervision.
8. Observe all cautions and size and weight limits for the Kaye Standers.

QUESTIONS?

If you have any questions, comments, or requests, please contact Kaye Products, Inc.

Our authorized representative for the European Medical Device Directives is Advena, Ltd., PO Box 30, Leominster, Herefordshire, HR6 0ZQ, UK.



10/15/02 I updated Advena's address. AWH.