## LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to nondurable components, such as rubber accessories, which are subject to normal wear and replacement.

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Medline Industries, Inc.

Three Lakes Drive, Northfield, IL 60093 USA

1-800-MEDLINE  
medline.com | info@medline.com

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### Medline United States

1-800-MEDLINE (633-5463)

www.medline.com | info@medline.com

### Medline Canada

1-800-268-2848

www.medline.ca | canada@medline.com

### Medline Mexico

01-800-831-0898

www.medlinemexico.com | mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this instruction sheet. ©2021 Medline Industries, Inc. Medline is a registered trademark of Medline Industries, Inc.

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Please read the instruction sheet carefully before installing on the tub. Your Personal safety will depend on following these instructions.

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PROTECTION THAT LIVES ON  
**MICROBAN**™

Inhibits the growth of stains and odors caused by bacteria, mold & mildew on the product. Keeps the deluxe tub bar cleaner for longer.

\*These antimicrobial properties are built-in to protect the product. This product does not protect users or others against bacteria, viruses, germs or other disease organisms.

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By June Wang at 2:31 pm, May 26, 2021

## **SAFETY PRECAUTIONS:**

- The bath tub wall, both inside and out, should be clean of soap or waxy residues and free of any obstructions. This is essential if the clamping action is to work correctly.
- Ensure all surfaces and clamping pads are dry and free of soapy residue. Clean and dry before each installation.
- On certain types of tub surfaces (i.e. fiberglass or thin steel tubs), use extreme care not to over tighten the knob. If the knob is over tightened, the clamp may crack the tub.
- ✓ The flat surface of the base of the tub grab bar should rest directly on the top of the tub wall. If a shower doortrack interferes with this contact, the unit should not be installed.
- ✓ Check the tub grab bar for tightness and stability before each use.

### **Installation Instructions:**

1. Open the clamp by turning the adjustment knob counter-clockwise.
2. Place the clamp over the sidewall of the tub with adjustment knob positioned on the outside of tub and secure tightly by turning adjustment knob clockwise. Improper installation can cause injury when exiting the tub.

### **Use Instructions:**

1. In order to assure the user maintains their balance when using the tub grab bar, always test for stability by vigorously attempting to lift and move unit sideways. If unit is clamped correctly, it should be extremely stable and rigid, even when substantial forces are applied.
2. If the grab bar becomes wet/soapy it may become slippery.
3. The tub's grab bar knob protrudes away from the tub. When entering or leaving the tub, use care to avoid injury from the protruding knob.

### **Cleaning:**

1. The tub grab bar features built-in Microban® antimicrobial protection to help inhibit the growth of stain and odor causing bacteria, mold and mildew to keep it cleaner for longer.
2. Periodic cleaning of the tub grab bar is recommended and will not harm the antimicrobial coating. Using soap and water is the recommended cleaning method for this product. Avoid the use of harsh cleaners.
3. Dry the tub grab bar after cleaning and before each use.

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