


BATHTUB RAIL

QUICK START GUIDE

ASSEMBLY INSTRUCTIONS

No Tools Required! ✂

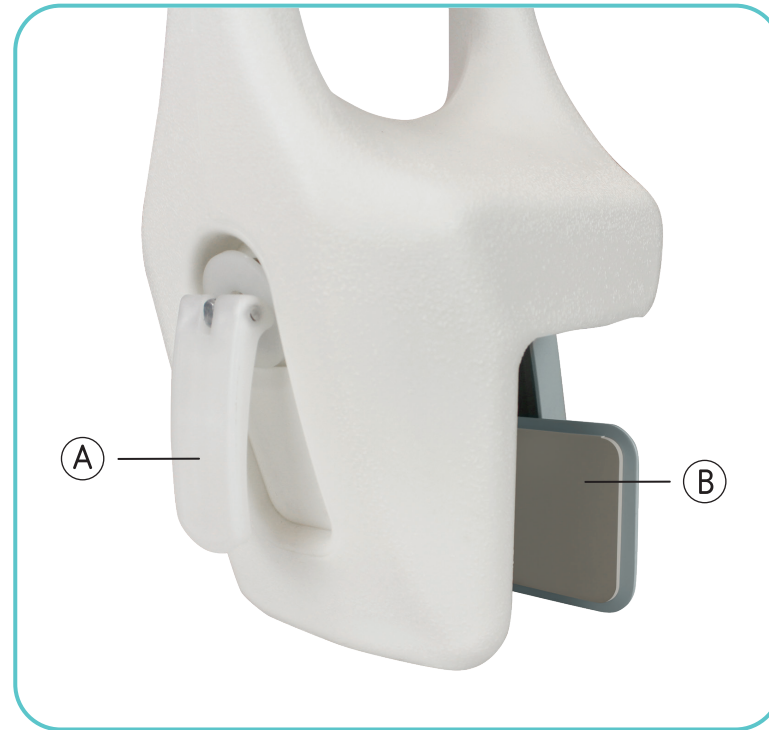

1. Clean bathtub thoroughly and wipe dry before applying rail. Surface must be dry and free of soapy residue for proper installation.
2. Lift up knob (A) and rotate counterclockwise to open. Pull out clamp (B) until rail is wide enough to slip on side of bathtub.
3. Place Safety Rail on bathtub with knob (A) facing outside of tub and tighten clockwise until rail feels tight and secure.
4. Push knob (A) down to original position.
5. Test installation of rail before using. Pull forward, back and side to side. If rail slips or moves, make sure surface is free of soapy film and is dry, then replace at desired location and re-tighten.

WARNING

- Check periodically before use and re-tighten as needed. Rail may require tightening from time to time.

SPECIFICATIONS

- Height: 21"
- Length: 7.5"
- Depth: 6.5"
- Adjustable Clamp: 3.75" - 6.5"
- Product Weight: 5.2 LBS
- Max User Weight: 300 LBS



vive®