

---

---

**Petite Pillow Switch #831-R**

---

---

**USER'S GUIDE**

Our soft, pliable Petite Pillow Switch is ideal for people with limited head and limb movements. The unique switch is encased in foam and covered with a soft bag which can be easily removed and washed.  
Size: 3" x 2".

### **Operation:**

1. The Petite Pillow switch does **not** require any batteries to operate.
2. Plug the Petite Pillow switch into the jack on your switch-adapted toy/device.
4. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
5. Make sure that the connection between the Petite Pillow switch and your toy/device is tight. There should be no gaps.
6. Simply press anywhere on the Petite Pillow switch to activate your toy/device.
7. The toy/device will only remain activated while the Petite Pillow switch continues to be actuated. Once you release the switch, the toy/device will turn off.

### **Troubleshooting:**

**Problem:** The Petite Pillow switch does not activate your toy/device.

**Action #1:** Make sure that the connection between the Petite Pillow switch and your toy/device is tight. There should be **no** gaps. This is a common error and an easy fix.

**Action #2:** Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

**Action #3:** Try a different adapter (if applicable) to rule this out as the source of the problem.

### **Care of Unit:**

The Petite Pillow switch's soft cover can be washed.

**Do not submerge** the unit, as it will damage the contents and the electrical components.

**Do not use abrasive cleaners**, as they will scratch the surface of the unit.