# DO NOT RETURN THIS PRODUCT TO STORE

We thank you for purchasing this Carex Health Brands Product. At Carex, we strive to bring you superior products that will satisfy the highest of standards for Quality, Durability and Performance. Any questions or concerns about any aspect of this product need to be directed to the toll free number below. **DO NOT RETURN THIS PRODUCT TO STORE.** We are available Monday – Friday, 8AM – 5PM CST, to address your needs.

**Carex Health Brands**  
© 2010  
PO Box 2526  
Sioux Falls, SD 57101-2526

**B316-00 and B317-00**

## toilet seat elevator

with removable safety handles

**Assembly and Installation Instructions**

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**1**

**Remove product from carton and READ ALL INSTRUCTIONS CAREFULLY before attempting to install product.**

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**2**

**To assemble handles:**

**Note: handles are marked for right and left**

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a. Place one handle at a time into the tubes on the raised toilet seat (the brass buttons on the handle should be facing out, away from the toilet seat.)

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b. Depress the brass buttons on the handle while sliding the handle into position. You will hear “clicks” when the buttons are positioned properly. (You will see the brass buttons pop through the position holes.)

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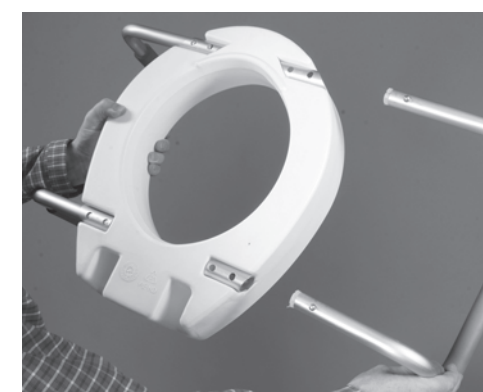
c. Repeat process for the other handle.

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**To remove handles:**

d. To remove the handles, depress the brass buttons while pulling out on the handles.

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**Note:** For most toilets you will need to reach behind and underneath toilet to unscrew the nut while holding screw with screwdriver.

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**5**

**4**

**Position toilet seat elevator on rim of toilet bowl and center.**

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**5**

**Place original toilet seat and lid on top of toilet seat elevator with holes lined up.**

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**6**

**6**

**Use screws provided to attach original seat, lid and toilet seat elevator to bowl. Secure with nuts provided.**

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**7**

**Check for secure fit routinely before using product.**

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**8**

**User's weight should be centered over toilet seat.**

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### Cleaning Instructions

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1. Use a detergent or household cleaner with warm water.

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2. Rinse with warm water and wipe dry.

Enjuague con agua tibia y seque.

3. **DO NOT USE AN ABRASIVE CLEANER** on handles, as it may cause damage.

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Advertencias e instrucciones

Caution / Precaución

1. Always be sure that the seat is correctly and securely in place before using.

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2. Users should be supervised or assisted when using this product.

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3. Be sure that the unit is correctly installed, fits properly and is stable before use.

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