
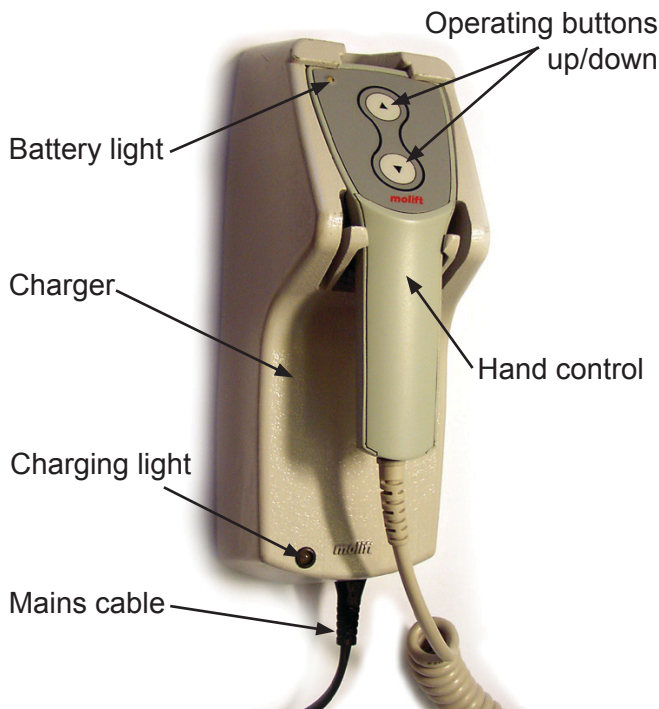


Charging

The charger can be placed on the wall or a table and connected to the power outlet. The Charger must be placed or installed in a way that makes it easy to disconnect mains cable. The charging light on the charger gives a yellow light when the charger is ready for use and there is no hand control placed in the charger.

 The battery will not achieve full capacity before it has been fully charged and discharged a couple of times.


The lifter can be operated without hand control. The lifter is then equipped with a special charger with a contact to be plugged in the hand control contact on the lifter when charging. Both chargers have the same functions.



Description of charging light

Charging light	Mode
Yellow	Ready for use, no battery connected
Yellow	Initialization
Orange	Fast charge
Green/yellow	Top-off charge
Green	Trickle charge
Orange/green	Error

Place the hand control in the charger, make sure the cable is connected to the lifter. After a few seconds the charger light will change from yellow to orange, indicating that the battery is charging. If the charging light gives a constant green light, this means the batteries are fully charged. The lifter can not be used while charging the batteries.

 If the emergency stop button is activated the batteries will not charge.