



When you use a patient lift, you want the security of knowing that the slings you use are durable and well-made.

At Tollos, we're diligent about you and your patients' safety.

So we make most of our slings right in our own facility. We take ownership of the materials and ensure the quality of the sling is up to our standards.

Slings are a Class I Medical Device. Tollos takes compliance with all applicable federal regulations very seriously.

We pride ourselves on our quality and your peace of mind.

Combi Sling

tollos

The Combi Sling is one of the most versatile slings on the market. This sling provides extra support to the hips and mid-back. It is equipped with handles on the back to allow for easy maneuvering of the patient.

The Combi Sling can be used for bed, chair, and floor lifting.

Combi Sling Features

- Seated sling with head support & padded legs
- Comfortable, yet durable poly-blended fabrics or mesh
- Color-coded binding allows for quick sizing
- Available in washable reusables or in single-patient-use varieties

General Washing Instructions:

Note: Please refer to the enclosed instructions-for-use packaged with all slings.

All Tollos reusable slings should be laundered according to the instructions on the tag that has been affixed to each sling. The following washing instructions are to be followed for optimum performance to preserve and maintain the quality of Tollos reusable slings:

- Washing
 - Sling should be inserted into a laundry bag prior to being placed in the washing machine. This prevents general wear and tear.
 - Slings should be washed separately.
 - Machine wash with laundry pods or other neutral (pH 7) detergents.
 - Slings should be washed at temperatures between 165°F/74°C (MIN wash temperature) and 185°F/85°C (MAX wash temperature) .
 - Avoid phenol-based disinfectants.
 - DO NOT USE CHLORINE BLEACH OR ALKALINE BUILDERS (The use of these products will void the warranty of the sling). Non-chlorine bleach may be used at concentrations less than 100 ppm (0.01%).
- Drying
 - Air dry or tumble dry at a maximum temperature of 120°F/50°C on air/fluff cycle ONLY. Exceeding this temperature may melt sling fibers and compromise the integrity of the sling.
 - Do not dry on steam pipes, heaters, ovens, or any other similar device.
 - DO NOT DRY CLEAN OR IRON.

Do not launder single-patient-use slings.



Sling Guide

Reusable	Reusable Mesh	Single Patient Use	Size	Suitable for:	Binding Color	Safe Working Load
T015CPPF	T015CPPMF	T015CPP-PSS	X-Small	49 - 75 lbs	White	600 lbs/ 272 kg
T015CPF	T015CPMF	T015CP-PSS	Small	75 - 125 lbs	Red	600 lbs/ 272 kg
T015CF	T015CMF	T015C-PSS	Medium	125 - 175 lbs	Yellow	600 lbs/ 272 kg
T015CXF	T015CXMF	T015CX-PSS	Large	174 - 250 lbs	Green	600 lbs/ 272 kg
T015CXXF	T015CXXMF	T015CXX-PSS	X-Large	250 - 400 lbs	Blue	1000 lbs/ 454 kg
T015CXXXF	T015CXXXMF	T015CXXX-PSS	XX-Large	400 - 600 lbs	Orange	1000 lbs/ 454 kg
T015CXXXXF	T015CXXXXMF	T015CXXXX-PSS	XXX-Large	600 - 1000 lbs	Gray	1000 lbs/ 454 kg