

## 2-in-1 LEG RELIEF WEDGE

Item #: CWEDG

Go to [www.contourliving.com/get-product-help/](http://www.contourliving.com/get-product-help/) to see inflation/deflation instructions.

### 3 Easy Steps to Elevated Comfort

#### 1 Inflate Your Wedge with the Hand Pump

Open valve cap and place pump into opening. Push / pull pump to fully inflate.



#### 2 Adjust Air Pressure to Suit Your Comfort

Lay on wedge and **press on the flat tab of the Air Release Switch** until wedge reaches desired comfort level.



#### 3 Close the Cap

Close the valve cap tightly to prevent air leaks.



### To Deflate

Gently twist the **Air Release Switch** a 1/4-turn until it clicks into open valve position. Let all air out. Tightly close the valve cap.



### Enjoy 2 Positions





## 2-IN-1 LEG RELIEF WEDGE

Item #: CWEDG

### To Clean

Spot clean only with soap and water. Do not use chemicals.



### Important Safety Information

Read and follow all warnings and instructions before using this product. Injury can result from improper use. Save these instructions.

#### **WARNING**

- **DO NOT USE AS A FLOTATION DEVICE. NOT INTENDED FOR USE IN WATER.**
- **THIS IS NOT A TOY. DO NOT ALLOW CHILDREN TO STAND OR JUMP ON BACK WEDGE.**
- **DO NOT USE AS A BED FOR INFANTS, CHILDREN OR PETS.**
- **USE ONLY FOR THE INTENDED PURPOSE - TO ELEVATE LEGS.**

#### **CAUTION**

- **INFLATE WITH HAND PUMP ONLY. DO NOT USE A COMPRESSOR WHICH COULD CAUSE THE WEDGE TO BURST.**
- **IT IS NORMAL FOR INFLATABLE ITEMS TO LOSE AIR OVER TIME. CHECK YOUR WEDGE PERIODICALLY TO SEE IF IT NEEDS FURTHER INFLATION.**
- **THE MATERIAL OF THIS ITEM IS SENSITIVE TO COLD AND MAY CRACK IF LEFT IN 40°F OR BELOW TEMPERATURES.**
- **KEEP SHARP OBJECTS AWAY FROM THE 2-IN-1 WEDGE TO PREVENT PUNCTURES.**
- **DO NOT PLACE ON ROUGH SURFACES SUCH AS CONCRETE.**