

D Free U1P

Operating Instructions

Toilet Timing Predicting Device

Please keep this document and refer to it as necessary.


Precautions on handling the device

Take following precautions when using the device. Improper usage may cause device failure.


- ⚠ Do not drop or apply strong force.
- ⚠ Do not wash with water or immerse in water.
- ⚠ Do not pull on the cable with excessive force.

How to Charge


Open the power supply port cover on the main unit and insert the micro USB plug (the smaller plug) of the supplied charging cable into the main unit.

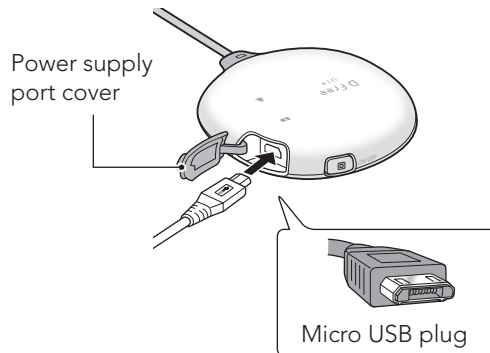

LED B will turn red while charging. Charging is complete once LED B is turned OFF and LED A turns green.

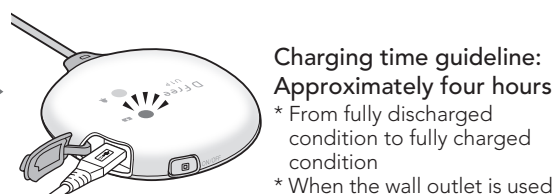

Charging time guideline:
Approximately four hours
 * From fully discharged condition to fully charged condition
 * When the wall outlet is used

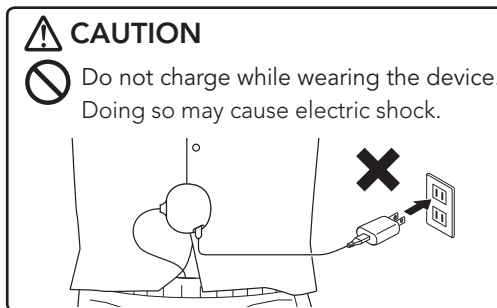


Connect the other side of the charging cable (USB plug) to a PC or connect to a wall outlet using a commercially available power supply adapter.

Power ON/OFF

Press and hold the Power/Operation button for three seconds to switch ON/OFF the device.


Power/Operation button

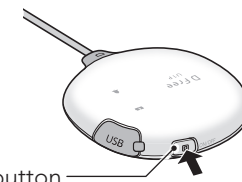

How to Clean


Wipe the sensor with a soft cloth dampened with water or neutral detergent.

* Wipe with a cloth dampened with alcohol or hypochlorite disinfectant only when disinfection is necessary.

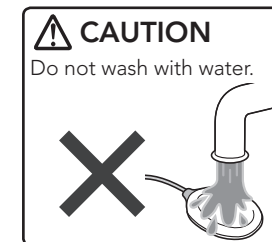


Description of parts


- DFree-U1P x 1

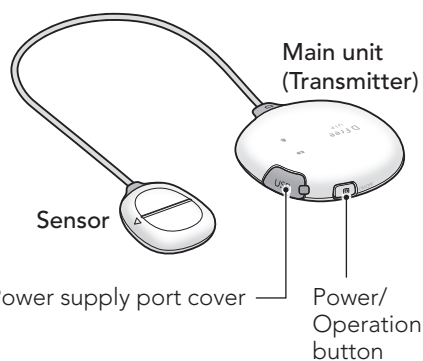

- Charging cable x 1

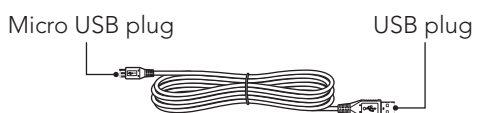

Operation and Notifications

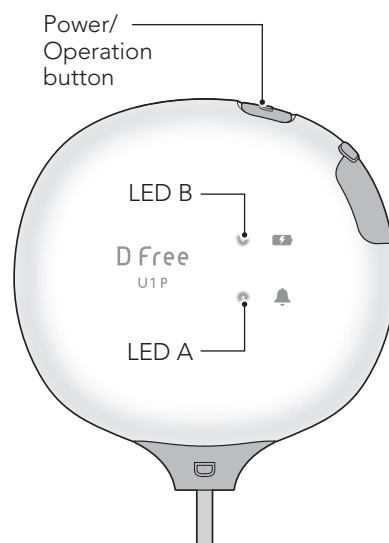


		Button operation	Green	LED A Red	Orange	LED B Red
Power ON/OFF /Pairing	Power ON (while pairing)	Press and hold for three seconds	Flashes alternately in red and green			—
	Pairing completed	—	Flashes twice	—	—	—
	While measuring	—	Flashes (three seconds intervals)	—	—	—
	Power OFF	Press and hold for three seconds	—	Flashes three times	—	—
Charging	While charging	—	—	—	—	Illuminated
	Charging completed	—	Illuminated	—	—	—
	Battery failure	—	—	—	—	Flashes
Notification	Error	—	Flashes sequentially			—




■ To use DFree device, please download and install the dedicated DFree Personal app. The DFree Personal app may be downloaded for free from the App Store and Google Play. The app supports iOS devices (iPhone, iPad, iPod) running iOS version 10 or later and Android devices (phones, tablets) running Android 4.4 or later. Note: Senior care facilities and medical institutions need to download the DFree Professional app.

Safety Precautions


Safety precautions are categorized into three categories: DANGER, WARNING, and CAUTION. Please follow these important precautions as they are for your safety.


Example of pictogram


	Prohibited
	Instruction


	DANGER	May cause imminent danger resulting in death or severe injury.
	WARNING	May result in death or severe injury.
	CAUTION	May result in device or property damage when handled inappropriately.

DANGER


 Never disassemble, repair, or modify this device.
Doing so may cause fire, electric shock, or injury.


 If your eyes are exposed to internal liquid from battery, rinse with water without rubbing and consult a physician immediately. There is a risk of losing your vision if unattended.
There is a risk of losing the vision.


 Immediately rinse with water when the liquid that has leaked from the battery has contacted the skin or clothing.
There is a risk of causing a damage.


 Do not immerse the main unit into water or spill water over it.
Doing so may cause fire, burn, injury, or electric shock.
If the main unit has been immersed in water accidentally, stop using it immediately and consult with Customer Support.


WARNING


 Do not use this device for purpose other than intended.


 Keep this device in places that children cannot reach.
Not doing so may cause an accident.

 Do not operate the button or connect/disconnect the charging cable with a wet hand.
Doing so may cause electric shock.


 Do not charge this device in a location with high humidity such as a bathroom.
Doing so may cause electric shock.


 Do not place this device close to a flame or heating devices.
Doing so may cause fire, electric shock, or failure.


 Immediately disconnect the charging cable and stop using this device when the device is overheating or smoking, or if you notice unusual odor or sound.
Continuing to use the device may cause fire or accident.

 Do not use this device if you notice any abnormalities with the device such as overheating.
Doing so may cause failure or fire.

CAUTION

 Do not use any accessories or parts other than those supplied or sold separately by Triple W.
Doing so may cause failure or injury and void warranty.
Doing so may cause failure or injury.

 Do not use organic solvent such as thinner or cleaners containing abrasive agent.

 Do not charge the device in an environment 104°F or higher (or under direct sunlight).

* Immediately stop using this device when you see symptoms such as itching, rash, etc. Immediately consult with a physician.

Disclaimer

- Do not use this product together with a medical device/ system that supports or sustains human life, or a device/ system requiring a high degree of reliability. This product is not designed to work together with such devices or systems.
- Triple W is not liable for any damages caused by the action taken based on data collected from this product, or not being able to use this product as intended.
- Triple W is not liable for any damages or injuries caused by not abiding to the terms in this Operating Instructions.
- DFree is not a medical device and is not intended to be used for diagnosis, prevention, or treatment of diseases or medical conditions.

FCC COMPLIANCE STATEMENT

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- This device may not cause harmful interference, and
- this device must accept any interference received, including interference that may cause undesired operation.

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment and meets the FCC radio frequency (RF) Exposure Guidelines as this equipment has very low levels of RF energy. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment.

Discarding

When discarding this product, discard in an appropriate method following the instructions of the government and municipality of the region where it is to be discarded.
Do not discard the lithium ion battery with household wastes.



Specifications

Model number	DU-BL-B2	
Power supply	5V, 1A	
Weight	3.18 oz.	
Size	Main unit	3.27" (W) x 3.15" (D) x 1.30" (H)
	Sensor	2.13" (W) x 1.38" (D) x 0.48" (H)
	Cable length	10.9"
Battery	Lithium ion rechargeable battery Capacity: 1150 mAh, Voltage: 3.7 V	
Charging time	Approx. 4 hours	
Continuous operating time	Approx. 24 hours*	
Operating temperature range	41 to 104°F	
Operating humidity range	20 to 80% (no condensation)	

* Based on internal testing. Battery life varies depending on the environment, usage patterns and method of measurement used.

 FCC ID:2APWD-DFU01P

Warranty

Products are warranted to be free from defects in materials or workmanship for one year from the date of purchase. Within this period, Triple W will, at its sole option, repair or replace any components that contain such defects. This warranty does not cover damage caused by accident, improper care, negligence, or normal wear and tear. Such repairs or replacement will be made at no charge to the customer for parts or labor, provided that the customer shall be responsible for any shipping cost.

In addition to the exclusions set forth above, this warranty does not apply to:

- cosmetic damage, such as scratches, nicks and dents;
- consumable parts, such as batteries, and sensor adhesive tapes, unless product damage has occurred due to a defect in materials or workmanship;
- damage caused by water, exposure to water or any similar circumstance;
- damage caused by service performed by anyone who is not an authorized service provider of Triple W;
- damage to a product that has been modified or altered without the written permission of Triple W;
- damage to a product that has been connected to power and/or data cables that are not supplied by Triple W.

Triple W

5795 Kearny Villa Road
San Diego, California 92123 USA
www.dfreeus.biz

Customer Support

Email : sp-us@www-biz.co

Phone: (833) DFREEUS
(833-337-3387)

Monday-Friday 10:00am-5:00pm PST

(Excluding Holidays)

Made in Japan
Ver. 2.0.1