



EASILIFT WALL MOUNTED CHANGING BENCH OPERATING INSTRUCTIONS

IMPORTANT!

Please take time to familiarise yourself with the use and maintenance of this equipment.

MAKE SURE THAT CHILDREN ARE KEPT WELL OUT OF THE WAY AND THAT NO ONE IS LIKELY TO INTERFERE OR TOUCH THE BENCH WHILST IN USE

TO RAISE:

1. Switch unit on at the isolator switch.
2. Release locking plunger and lower bench into horizontal position.
3. Raise or lower to suitable height for transfer using the hand control.
4. Transfer patient onto the bench.
5. Raise safety rail! – Ensuring it is correctly engaged.
6. Buckle and adjust safety straps if fitted.
7. Raise bench to suitable height for operator.

TO LOWER:

1. Lower to suitable height for transfer.
2. Lower safety rail by lifting slightly then swinging outward and downward.
3. Unbuckle safety straps if used.
4. Transfer patient to chair, wheel chair or into standing as convenient.

TO FOLD AWAY INTO STORAGE:

1. Clean and dry the upholstery.
2. Adjust the bench into storage position.
3. Lower safety rail and fold down buckle straps if fitted.
4. Fold up the upholstered bench.
5. Make sure locking plunger is correctly engaged.
6. Switch off electricity at the isolator switch.

HEALTH AND SAFETY:

- NEVER leave a patient unattended.
- Do not allow anyone to play with the bench.
- Do not exceed more than six complete cycles per hour at maximum rated load.
- Always use the safety rail (and straps if fitted).
- Check safe operation of retaining strap and safety rail before use.
- **NEVER EXCEED THE MAXIMUM LOADING OF 100kg.**

MAINTENANCE:

- Check **weekly** that all visible screws and fittings are in place.
- Set up a routine maintenance schedule as recommended in the product manual.

NOTE: For safety reasons to the safety rails the bench must always be lowered with the rails in the up position.

(Please place this card in a convenient position for the operator)