EASyS / EASyS Modular

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Dear EASyS / EASyS Modular user,

The EASyS / EASyS Modular rehabilitation stroller that you purchased is an innovative product made by the company Thomashilfen. After several years of research we have developed a new concept for rehabilitation strollers that is one of the safest and most advanced seating systems available.

In order to make handling of the EASyS / EASyS Modular seating system and chassis safe, practical, and comfortable for you and especially for your child, please read this user manual completely before use.

Should you have any further questions or issues, please contact your specialist supplier or Thomashilfen directly.

For more information about Thomashilfen and our products, please visit our website at www.thomashilfen.com

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Our address for customers in the USA:

ExoMotion LLC
dba Thomashilfen
309 South Cloverdale Street - Unit B 12
Seattle WA 98108
USA
phone: 866 870 2122 (toll free in U.S.)
www.thomashilfen.us

Contents	page
Purpose and use.....	13
Risks and contraindications	13
Safety-related checks and maintenance intervals	14
General safety instructions	14
Technical data	14
Setting up and assembly.....	15
Adjustment options.....	16
Mounting the belts	17
Product- / serial number.....	17
Cleaning, disinfecting, and care instructions.....	17
Terms of guarantee / CE mark.....	17

Purpose and use

The EASyS / EASyS Modular seat unit together with a suitable frame / chassis is appropriate for indoor and outdoor transport (walking pace at the most) of disabled children by their parents or other qualified helpers. The seat unit in combination with the A chassis must not be used for purposes other than intended (e.g. transporting more than one child, attaching shopping bags, etc.). When using the A chassis, EASyS / EASyS Modular is approved for transportation of children in motor vehicles / disabled transportation vans according to ISO 7176-19.

Risks and contraindications

Proper handling may avoid any use-related risks. No contraindications are known. Fabrics used for EASyS / EASyS Modular are flame retardant, tested according to BS 5852 part 1. The extent of resistance against ignition of materials and components has been tested in accordance with the standard and classified as harmless.

Safety-related checks and maintenance intervals

- All operating elements and fastening screws should be subjected to visual inspection every month by a trained expert. The telescopic sliding plate should remain free of dirt and sand.
- Please bear in mind that the screws and release handles (hip angle adjustment) should be secured after each adjustment.

Please observe the following safety instructions:

- Carefully read this user manual prior to first use. If lost, you can download the user manual in the download area of our website: www.thomashilfen.com
- We recommend that you adjust the EASyS / EASyS Modular seat growth settings every 3 months to best fit your child. If necessary, let your therapist help you.
- **CAUTION:** The gap between chassis and release rod of the brake may present a hazard if clothes and extremities get caught in the foot area.
- **CAUTION:** The gap between seat and chassis frame may present a hazard for clothes and extremities getting caught in the seating area.
- **CAUTION:** Solar radiation, electronic heaters, or other heat sources may result in excessive heating-up of the seat / rehab stroller. On contact, this may cause a risk of burns at back and legs! Make sure that the seat / rehab stroller is not exposed to such heat sources at any time when in use or stored, and use it only when it had cooled sufficiently.
- **CAUTION:** Wet or flat wheels may affect the braking effect. If not in use or stopping for a longer period, engage the stroller foot brake in order to prevent accidental rolling.
- Do not exceed the maximum size and weight capacity of the EASyS / EASyS Modular stroller. Please refer to the literature for specification details (s. technical data).
- We recommend that you always secure your child with the 2-point pelvic belt or another retention belt (available as an accessory). Never leave your child unattended in the stroller.
- Using reflective clothing increases visibility of yourself and your child to other road users, especially when strolling after dark.
- Attaching bags, equipment, or other heavy items may cause the stroller to be unstable and tip over. These items are best stowed in the basket below the seat unit.
- When working with the seat unit, please observe the recommendations in the user manual, especially when dismantling it and folding it together.
- Keep packaging material away from children - choking hazard!
- This stroller is intended to transport only one child.
- Existing market experiences do not result in any dangerous combinations of equipment with the seat.

Technical data

EASyS seating system	EASyS Size 1	EASyS Size 2
Seat depth	16 - 28 cm / 6.3 - 11"	24 - 35 cm / 9.4 - 13.8"
Seat width	19 - 31 cm / 7.5 - 12.2"	25 - 36 cm / 9.8 - 14.2"
Back height	40 - 65 cm / 15.7 - 25.6"	53 - 73 cm / 20.9 - 28.7"
Lower leg length	17 - 30 cm / 6.7 - 11.8"	25 - 36 cm / 9.8 - 14.2"
Hip angle	90° to 180°	90° to 180°
Knee angle	90° to 180°	90° to 180°
Foot angle	78° to 100°	78° to 100°
Turning range of seat	180°	180°
Seat tilt-in-space (on A chassis)	-10° to +35°	-10° to +35°
Size of hip guide (w x h)	12 x 23 cm / 4.7 x 9.1"	15 x 30 cm / 5.9 x 11.8"
Size of foot rest (w x d)	29 x 18 cm / 11.4 x 7.1"	33 x 21 cm / 13 x 8.3"
Overall dimensions (l x w x h)	58 x 38 x 61 cm 22.8 x 15 x 24"	63 x 44 x 64 cm 24.8 x 17.3 x 25.2"
Folded dimensions (l x w x h)	73 x 38 x 28 cm 28.7 x 15 x 11"	85 x 44 x 29 cm 33.5 x 17.3 x 11.4"
Weight (seat)	7.7 kg / 17 lb	9.4 kg / 20.7 lb
Carrying capacity (seat)	35 kg / 77.2 lb	40 kg / 88.2 lb

EASyS Modular seating system	EASyS Modular Size 1	EASyS Modular Size 2
Seat depth	16 - 28 cm / 6.3 - 11"	24 - 35 cm / 9.4 - 13.8"
Seat width	20 - 30 cm / 7.9 - 11.8"	26 - 35 cm / 10.2 - 13.8"
Back height *	26 - 42 cm / 10.2 - 16.5"	32 - 52 cm / 12.6 - 20.5"
Lower leg length	17 - 30 cm / 6.7 - 11.8"	25 - 36 cm / 9.8 - 14.2"
Hip angle	90° to 140°	90° to 140°
Knee angle	90° to 180°	90° to 180°
Foot angle	78° to 100°	78° to 100°
Turning range of seat	180°	180°
Seat tilt-in-space (on A chassis)	-10 to + 35°	-10° to +35°
Height adjustment arm support	10 - 14 cm / 3.9 - 5.5"	12 - 18 cm / 4.7 - 7.1"
Size of arm support (w x h)	21 x 10 cm / 8.3 x 3.9"	29 x 12 cm / 11.4 x 4.7"
Size of foot rest (w x d)	29 x 18 cm / 11.4 x 7.1"	33 x 21 cm / 13 x 8.3"
Overall dimensions (l x w x h)	85 x 41 x 57 cm 33.5 x 16.1 x 22.4"	64 x 48 x 77 cm 25.2 x 18.9 x 30.3"
Folded dimensions (l x w x h)	89 x 41 x 38 cm 35 x 16.1 x 15"	90 x 48 x 60 cm 35.4 x 18.9 x 23.6"
Weight (seat) *	7.9 kg / 17.4 lb	9.6 kg / 21.2 lb
Carrying capacity (seat)	35 kg / 77.2 lb	40 kg / 88.2 lb

* without headrest

Setting up and assembly

Adjustments / adaptations shall be carried out by trained or instructed experts (e.g. medical device consultants of a medical device dealer) only.

The seat can be removed from the A chassis without needing any tools. Push the click lever below the seat unit forwards and take the seat unit off the receiver adapter of the chassis forwards. Proceed in reverse order to insert the seat.

CAUTION: Always ensure the click lever engages audibly. Then make sure the seat has clicked into place by pulling it forwards.

Commercially available tools are required for mounting / dismantling the accessories. Separate mounting instructions and user manuals are enclosed with the accessories. If they should be missing, you can download the instructions in the download section of our website www.thomashilfen.com

Mounting the foot rest

The foot rest is to be fixed to the knee joints. Place the grid joints one upon the other and connect them with the screws (from the inside), washers (from the outside), and wing nuts (from the outside). (Fig. 1)

Lateral backrest upholstery / side upholstery (EASyS)

Put the lateral backrest upholstery over the lateral tube and velcro it left and right to the curved section of the back plate (the snap fastener points inwards). Close the Velcro of the lateral backrest upholstery completely. Attach the lateral upholstery to the lateral backrest upholstery with the snap fastener. In addition, velcro it to the back plate. (Fig. 2-4)

Lateral backrest upholstery (EASyS Modular)

Put the lateral backrest upholstery over the lateral tube and velcro it left and right to the curved section of the back plate. Close the Velcro completely – also over the knurled screw. In addition, velcro it to the back plate. (Fig. 5-6)

Receiver for sun and rain cover

Both EASyS and EASyS Modular are equipped with receivers for the sun and rain cover. (Fig. 7)

Seat upholstery

Velcro the seat upholstery to the seat plate. (**EASyS only:** Fix the two lower Velcro fasteners around the tubes of the foot rest).

Inserting the seat unit

Place the seat unit on the rear crossbar and move the seat forward and down. The EASyS click system engages automatically and audibly. (Fig. 8)

Removal of the seat unit

To remove the seat unit, push the click-lever under the seat forward and take the seat unit off the chassis forward. (Fig. 8)

Adjustment options

Seat width

Loosen the hip guide screws (EASyS) / arm support screws (EASyS Modular) from the seat top surface with an Allen key (provided with the user manual) and move the lateral elements accordingly. (Fig. 9) If the seat depth is adjusted to the minimal position, the maximal adjustable seat width depends on the existing space between the lateral back rest upholstery.

Seat depth

Loosen the front Allen screws (same as the hip guide setting) on the seat surface, stand facing the seat towards you and pull out the seat plate. (Fig. 10)

Tip: For easy adjustment of the seat depth grip the frontal edge of the seat with one hand and press against the bottom part of the backrest (concentric) with the other hand.

The hip guides (EASyS) are adjustable in depth, as well. Loosen both screws, set the desired depth, and retighten the screws. (Fig. 11)

Arm support (EASyS Modular)

The arm supports are adjustable both in height and depth. Loosen both wing nuts, set the desired height and depth, and retighten the nuts. (Fig. 12)

Back height

Remove the lateral backrest upholstery slightly from the seat. Loosen both knurled screws at the top of the lateral tubes. (Fig. 13) The back plate is infinitely adjustable in height. The maximum height is achieved when the locking pin clicks in audibly. Velcro fasten the lateral backrest upholstery.

To reduce the height press down the locking pin underneath the right knurled screw and push the back plate down to the desired height. (Fig. 14)

Back height step 2 (EASyS only)

In order to achieve the maximum back height loosen both knurled screws at the top crossbar and pull up the back plate. (Fig. 15)

Tip: The extended body guides offer additional protection to children who are primarily transported in a lying position.

Headrest (EASyS Modular only)

Insert the tube of the headrest into the receiver at the top of the headrest bracket and tighten the wing screw. (Fig. 16) The headrest is infinitely adjustable in height by loosening the knurled screws on the rear of the backrest. (Fig. 17) To adjust the angle of the headrest, loosen the wing screws at the headrest bracket. (Fig. 18) In addition, the headrest can be adjusted in depth by loosening the wing screw on top of the headrest bracket. (Fig. 19) The angle of the headrest surface is adjustable both vertically and horizontally by loosening the screws of the ball joint. (Fig. 20) Make sure to retighten all screws when adjustments are completed.

Back angle adjustment

Loosen the clamping lever at the backrest and pull out the safety locking pin below the lever. Now the backrest can be adjusted in the desired position. The seat unit automatically slides forwards on its rails, shifting its center of gravity to maintain stability in all seat positions. (Fig. 22) Retighten the clamping lever once the position is selected.

To lock back recline based on preset angles: Loosen the clamping lever at the backrest and pull out the safety locking pin below the lever. Now the backrest can be adjusted in the desired position. The seat unit automatically slides forwards on its rails, shifting its center of gravity to maintain stability in all seat positions. (Fig. 22) Engage locking pin and ensure it slides into a factory supplied locking hole. Retighten the clamping lever once the position is selected.

Hint: The seat units have a different number of holes for the safety locking pin at the backrest.

EASyS 1: 2 holes; **EASyS 2:** 4 holes; **EASyS Modular 1:** 1 hole; **EASyS Modular 2:** 2 holes

During transport of the rehab stroller in a vehicle, the safety locking pin has always be locked in the bottom hole (90° back angle) The other holes can be used to adjust further commonly used back angles.

Lower leg length

Loosen the threaded pins to the side of the foot rest with the Allen key. (Fig. 23) The foot rest can be adjusted without fixed increments. Retighten the threaded pins when adjustments are completed.

Foot angle

Loosen the 4 screws on the right and left on the bottom of the foot rest, adjust the desired angle, and retighten the screws. (Fig. 24)

Knee angle

Set the knee angle by loosening the side wing nuts on both sides. (Fig. 25)

Tip: The foot rest can be flipped up to facilitate transfers

Mounting the belts

EASyS

Pass the two upper belts through the slots in the upholstery (choose the appropriate slots depending on the size of the child) and through the slots in the front back plate. (Fig. 26) Insert the belts into the cam buckles and adjust the desired belt length. (Fig. 27)

The two bottom belt ends can be attached with the cam buckles to either the back plate or to the side of the plastic element. (Fig. 28-30) Guide the belts left and right next to the back upholstery to the corresponding cam buckle and fix them.

The front belts of the 4-point pelvic belt are attached right and left at the plastic element with a cam buckle, as well. Guide the belt through the slot in the hip guide, insert it into the cam buckle, and fix it. (Fig. 29-30)

EASyS Modular

Pass the two upper belts depending on the size of the child to either both inner or both outer loops of the back upholstery and attach them to the rear of the backrest with the cam buckles. (Fig. 31)

The two bottom belt ends can be attached with the cam buckles to either the back plate or to the side of the plastic element. (Fig. 32-33) Guide the belts left and right next to the back upholstery to the corresponding cam buckle and fix them.

Tip: The belt length can also be adjusted from the front. Pull the D-ring to shorten the belt.

Product / serial number (conclusion on construction year)

To make sure you receive correct spare parts or for potential technical recalls we always require the serial / batch number and exact item code. If you give us the serial / batch number, we can inform you about the construction year of our EASyS / EASyS Modular.

You can find the serial / batch number of your EASyS / EASyS Modular on the barcode label, underneath the seat.

Cleaning, disinfecting, and care instructions

Upholstery

The upholstery can easily be removed. The 100% PES fabric and the interlaced yarns are made of high-quality cloth and can be washed at up to 40°C (104°F). The material is printed with non-toxic dyes which are free of heavy metals. Please be aware that even high-quality materials may fade as a result of long-term and intensive solar radiation or due to frequent washing. The upholstery is flame retardant (BS 5852 part 1).

Velcro fasteners

In order to maintain functionality of the Velcro fasteners, brush them down from time to time. Always close the Velcro fasteners during washing.

Terms of guarantee / CE mark

Thomashilfen offers you a guarantee of 3 years for all frame parts, commencing on the date of purchase. This guarantee incorporates all claims which influence functionality. Damages due to improper use (e.g. overloading), natural wear and tear are excluded.

EASyS / EASyS Modular meets the requirements of the European standards EN 12182 and 12183 and is provided with a CE marking.

Enjoy using your EASyS / EASyS Modular rehabilitation stroller!

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Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG