



**Fig.5**

---

## **Clip Clamp #3706**

---

## **USER'S GUIDE**

A great new item for holding eye-talks, photographs or icons. You can easily mount this to wheelchair trays, tabletops or bedrails. Size: 27"H.

**Operation:**

1. Using the mounting clamp, attach the Clip Clamp to a Wheel Chair, Table or Desk. Tighten the clamp enough to hold the Clip Clamp securely, but do not over-Tighten the clamp.
2. The mounting clamp can be mounted vertically (currently) or horizontally. To change the clamp to horizontal, first loosen the clamp from the gooseneck by turning the clamp counter-clockwise (see figure. 1)



**Fig. 1**

3. Once the clamp has been removed save the lock washer, the lock washer will not be used for mounting the clamp horizontally. It is only needed for vertical mounting. Next slide the gooseneck into the clamp as shown in (figure. 2) the gooseneck's flared end should be flush with the side of the clamp. Using the include 3/16 Allen wrench tighten the threaded post as shown in (figure. 3), do not over tighten. If you do not see the Allen slot in the post, unscrew the post from the clamp by hand and turn it around as shown in (figure. 4).



**Fig. 3**



**Fig. 4**

**Troubleshooting:**

**Problem:** The clamp seems to loosen from the gooseneck when used vertically.

**Action:** Make sure you are using the supplied lock washer (shown in figure. 5) between the clamp and the gooseneck. If you still have problems you can add a drop or two of thread lock such as Loctite Medium strength No. 242 or 243.

**Care of Unit:**

Clip Clamp can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

**Do not use abrasive cleaners,** as they will scratch the surface of the unit.