
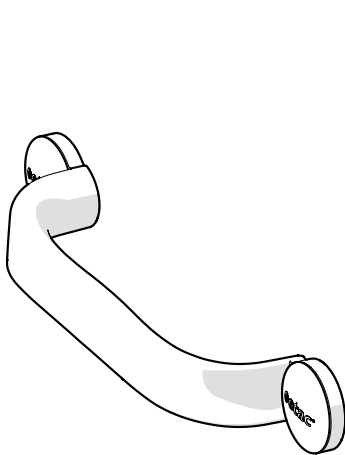
Etac Flex


Manual

etac®


75339A 17-08-29

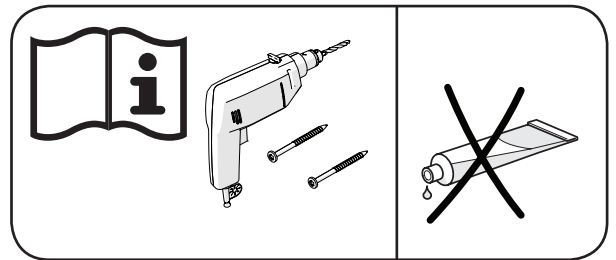30 cm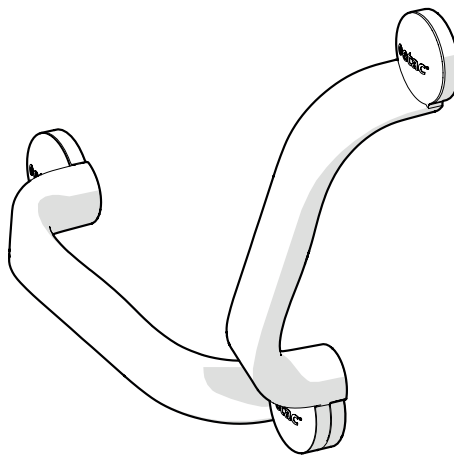

60 cm

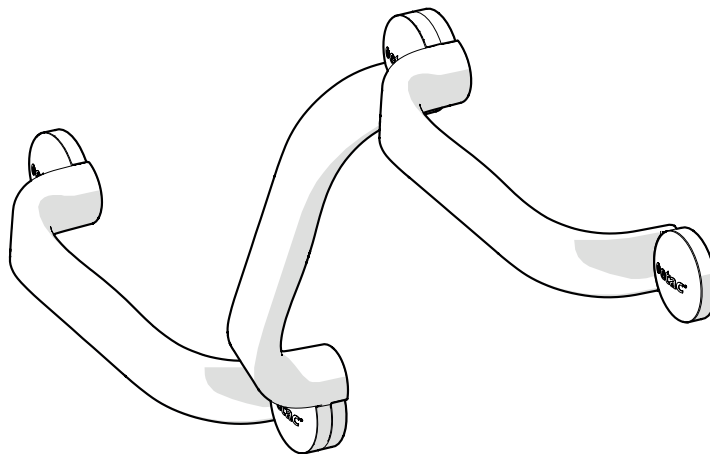
90 cm

Manual


sv Svenska 4	de Deutsch 5	it Italiano 7
no Norsk 4	fi Suomi 6	es Español 7-8
da Dansk 4-5	fr Français 5	pt Português 8
en English 5	nl Nederland 6-7	

A


30 cm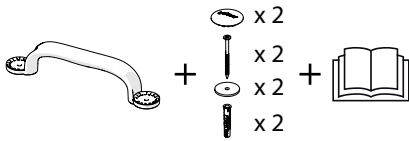


60 cm

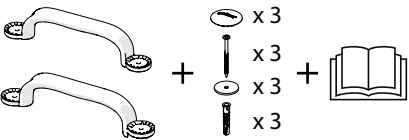90 cm

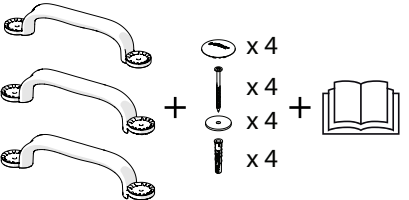B



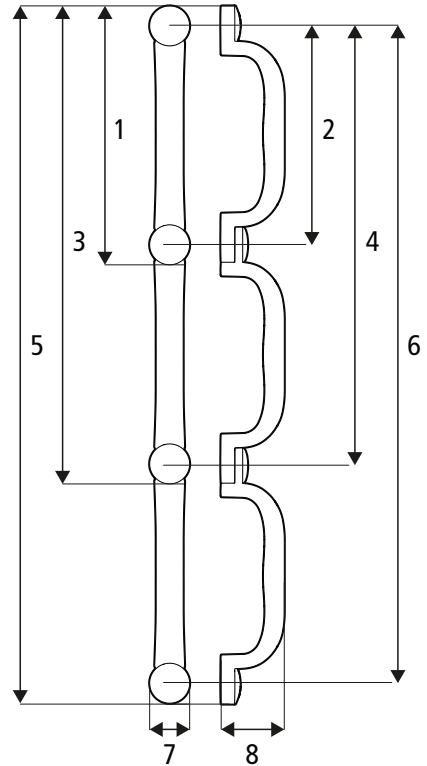
B1: 355 mm (14")	B2: 300 mm (11¾")
B3: 655 mm (25¾")	B4: 600 mm (23½")
B5: 955 mm (37½")	B6: 900 mm (35½")
B7: 55 mm (2")	B8: 83 mm (3¼")

max kg 100 kg (220 lbs)	>PP GF<	KG 0,35 kg (0,77 lbs)
----------------------------	---------	--------------------------

C

1 	2 	3 	4 	5
-------	-------	-------	-------	-------

D

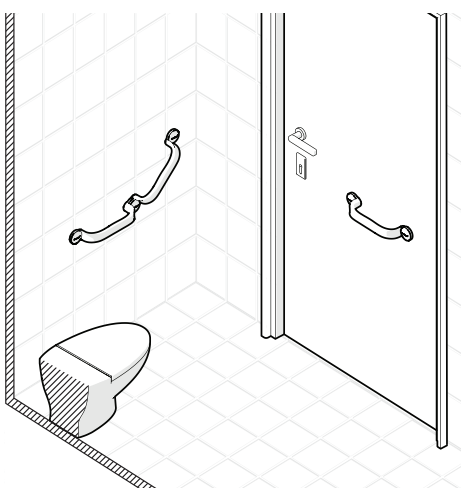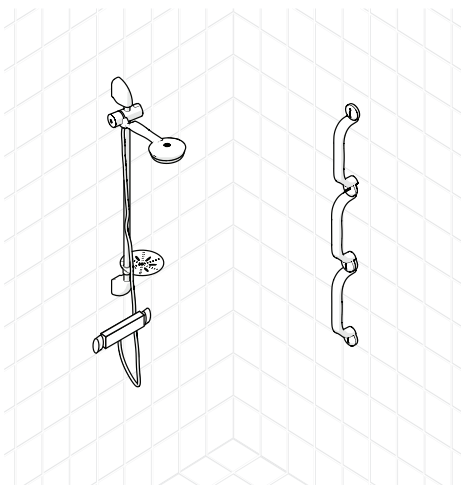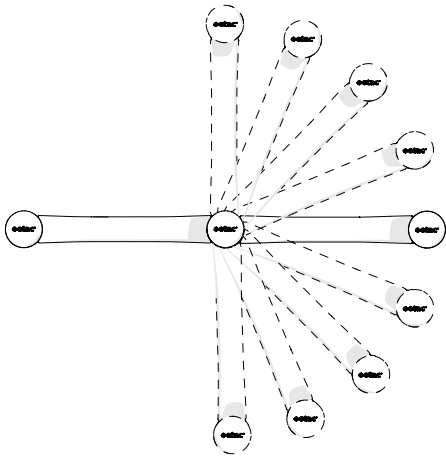E

E1

1.2

1.3 75 mm
Ø 10 mm

1.4

E2

2.2

2.3 min. 24 mm
Ø 3 mm

2.4 min. 4Nm

E3

3.2

3.3 min. 4Nm

3.4

3.5

E4

4.2 min. 4Nm

4.3

**Svenska**

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en

English

Thank you for choosing an Etac product. To avoid injury during installation, handling and use, you should read this manual and keep it for future reference. The manual can also be found on www.etac.com, together with other documentation including prescriber information.

Intended use

Etac Flex is a wall-mounted support handle intended for use as a transfer and standing support for people with impaired mobility. Etac Flex is intended for use in home environments and in institutions, i.e. hospitals and nursing homes.

Etac Flex is not intended to be sat on, or to be used in any other way or in any other environment than those stated above.

Etac Flex has been tested and complies with the requirements set out in EN ISO 10993-1 and SS EN 12182.

Etac Flex is CE-marked in accordance with Medical Device directive 93/42/EEG.

This manual applies for Etac Flex when mounting it with screws. When mounting it with glue, see separate manual under www.etac.com.

Content of the delivery Figure A

Technical data Figure B

This product should be recycled in accordance with national regulations. See material specifications for sorting (figure B).

Maintenance and safety Figure C

C1. Clean/disinfect the product using a standard non-abrasive cleaning agent with a pH between 5 and 9, or with a 70% disinfectant solution. Rinse and dry. The product requires no other maintenance.

C2. The product can be safely cleaned in a cabinet washer-disinfector at 85°C for 3 minutes.

C3. Max. user weight 100 kg (220 lbs).

C4. Check the screw fastener. A defective or incorrectly mounted product should not be used. In the event of a fault, contact your prescriber or retailer.
C5. Heated products can cause burns.

Constituent materials are corrosion-resistant. Constituent materials are resistant to standard disinfection agents.

Guarantee, service life

5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com.

Service life 10 years. For complete information regarding the service life of the product, see www.etac.com.

Positioning, use Figure D

Etac Flex can be mounted in different positions. The additional handle can be mounted at angles of 22.5° between 0 and 90°. Etac Flex can be extended by an unlimited number of additional handles.

Mounting options

Etac Flex has been designed to withstand a maximum user weight of 100 kg (220 lbs) and should only be mounted onto a wall with sufficient load-bearing capacity. Ensure that you use suitable fixtures.

Screws: Etac recommends the accompanying screw set for mounting on wood/plywood or concrete walls.

If in doubt about the wall's load-bearing capacity or the suitability of the fixtures, an assessment should be carried out by a qualified person.

In the case of other wall materials, or if the load-bearing capacity of the wall is deemed insufficient for any other reason, the wall can be reinforced by a qualified person.

Mounting on a wall with insufficient load-bearing capacity or unsuitable fixtures poses a danger to the user!

Mounting with screws Figure E

• Follow the instructions under sections **E1 & E4** for mounting on uncracked concrete C20/25 to C50/60. Always mount using Fischer UX R 10 plugs.

• Follow the instructions under sections **E2 & E4** for mounting on wood or plywood (min. 12 mm) with partition wall (min. 12 mm).

• Follow the instructions under sections **E1 or E2 & E3-E4** for mounting of Flex 60/90.

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R82 UK Limited.
R82, Inc.
Etac Supply Center AB