


FirstClass Sleep-to-Stand Lift Chair


ASSEMBLY and OPERATION MANUAL



CHAIR POSITION REMOTE

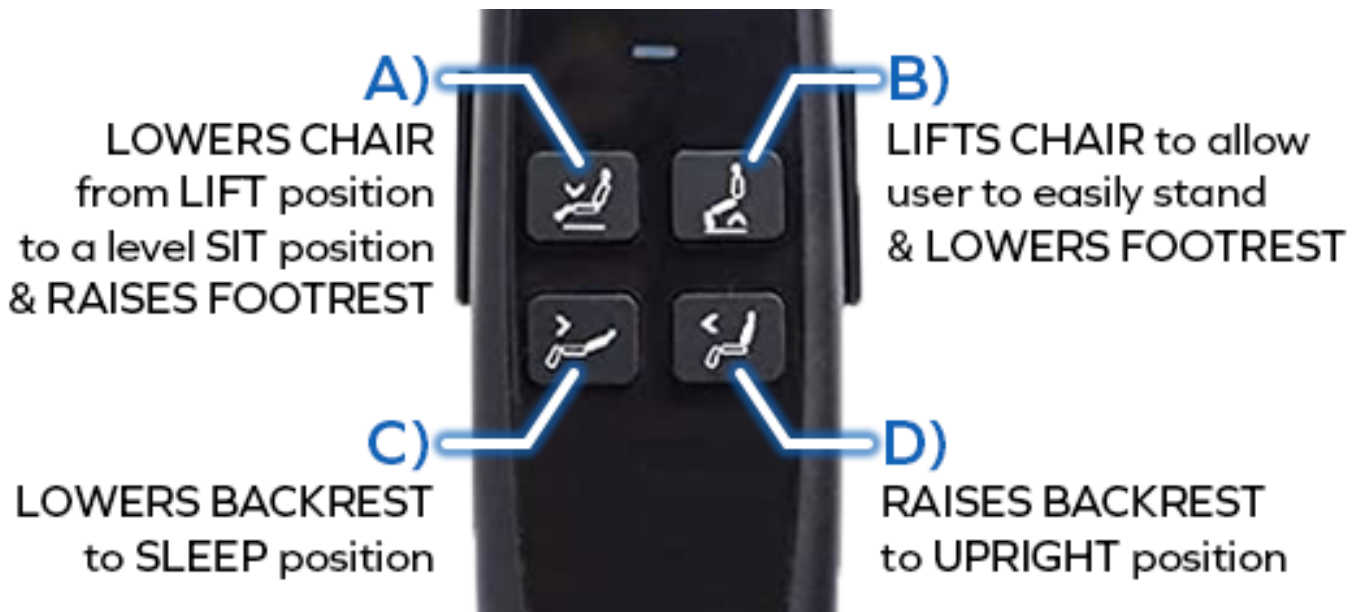




MASSAGE REMOTE

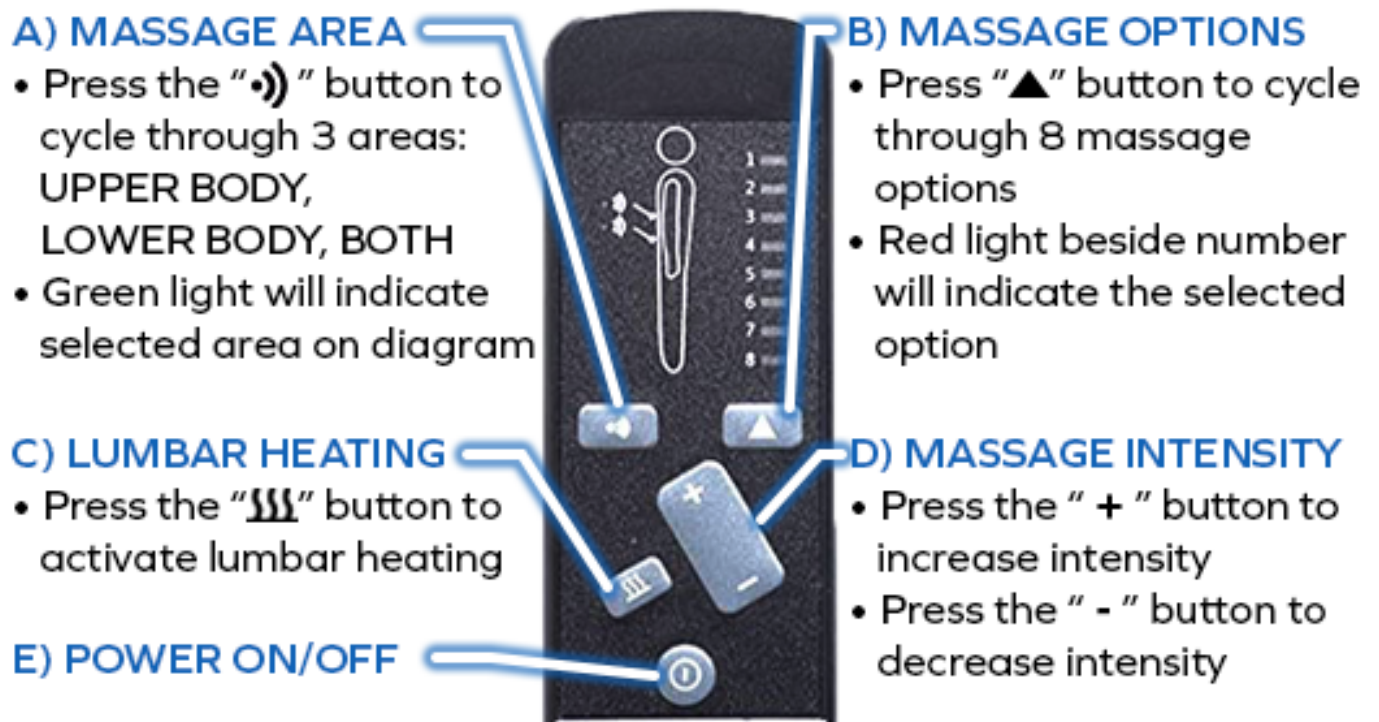

ASSEMBLY INSTRUCTIONS

PARTS LIST:

- A. Chair Base with Motor
- B. Right Armrest with Loop & Pocket for Remotes
- C. Left Armrest
- D. Backrest
- E. Chair Position Remote (should be attached to the chair base)
- F. Motor Power Cord
- G. Massage Remote & Massage Power Cord
- H. Pillow
- I. Wrench

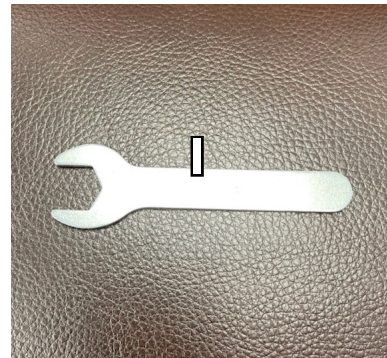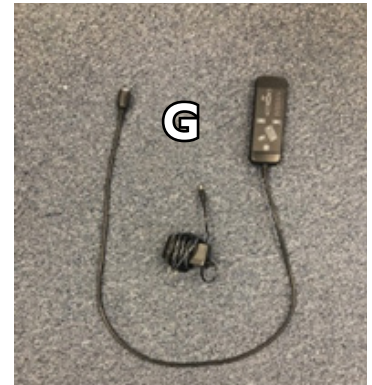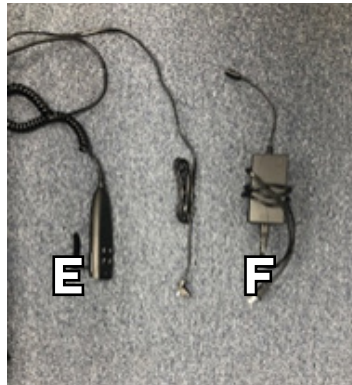

1) LOCK THE WHEELS/CASTORS ON THE BOTTOM OF YOUR CHAIR USING THE WRENCH.



2) PLUG IN THE MOTOR POWER CORD INTO A WALL OUTLET AND LEVEL CHAIR BASE.



Using the buttons on the CHAIR POSITION REMOTE make sure the base of the chair is completely flat and level on the ground. Make sure the inserts for the backrest attached are in a comfortable angle to slide on the backrest.

3) ATTACH THE BACKREST.

Align the two metal slots on both sides of the backrest to the inserts on the base of the chair. Once aligned, press down firmly until you hear them click in place. Note: Attaching the backrest before putting the arms on gives the user a clearer view for assembly.



4) PLACE ARMRESTS ON.

Lower the backrest using the Chair Position Remote so the backrest is level and flat.





Note: BEFORE ATTACHING ARMRESTS ENSURE THE RIGHT SIDE HAS THE LOOP AND POCKET FOR BOTH REMOTES.

Attach each armrest by aligning them with the two metal slots on each side of the chair.



**Note:
RESET THE BACK OF YOUR CHAIR TO THE
UPRIGHT POSITION FOR THE FINAL STEP.**

5) ATTACH THE MASSAGE REMOTE.

Plug the Massage Remote to the cord under the leather flap on the back of the chair attached by Velcro then plug the Massage Power Cord into a wall outlet.



6) YOUR FIRSTCLASS SLEEP-TO-STAND LIFT CHAIR IS READY TO USE!

Please contact us with any questions.



 **PlatinumHealth.**

888-550-2410

info@platinumhealthllc.com

platinumhealthllc.com