


912410



914504

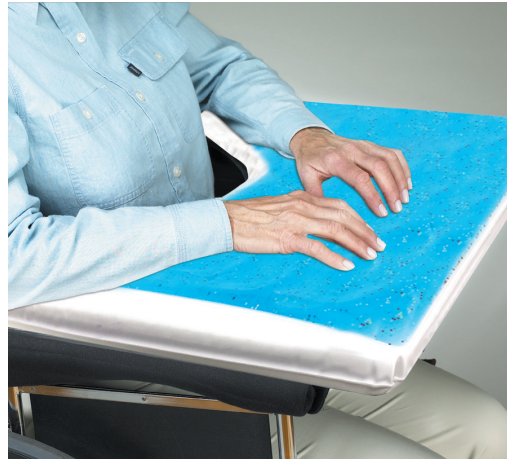

912416

ReOrder #	Description	Wheelchair Size	Unit
912410	Gel-Top Sensory Tray, Black Vinyl, Lift-Away	16" - 18"	Each
914504	Gel-Top Sensory Tray, White Vinyl, Lift-Away	16" - 18"	Each
912415	Gel-Top Sensory Tray, Black Vinyl, Hook & Loop Straps	16" - 18"	Each
912416	Gel-Top Sensory Tray, Black Vinyl, Hook & Loop Straps ( <i>Pediatrics</i> )	14"	Each

**Purpose**


Skil-Care’s Gel-Top Sensory Tray provides both tactile and visual stimulation to reduce agitation and restlessness on the part of wheelchair-bound Alzheimer’s and other diminished capacity patients. It also reduces pressure on arms and elbows when the patient uses the tray for postural support and positioning.

**Easy to Use**

The Gel-Top Sensory Tray is available in to two models: A Lift-Away version for restraint-free care and a hook and loop strap version.

**Lift-Away Model (#912410 & 914504)**

1. Wrap padded armrest covers around armrests and secure hook & loop (Illustration A)  
Make certain that the small straps attached to the armrest covers are positioned to the rear of the armrests.
2. Wrap the small strap around the armrest support bar, just behind the wheelchair armrest

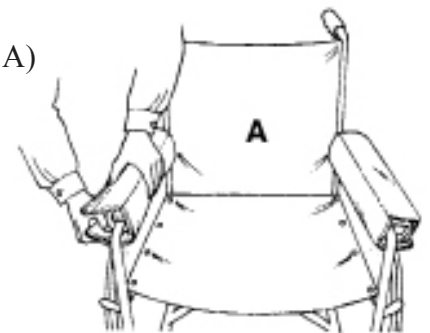

**Hook & Loop Closure Model (#912415 & 912416)**

1. Place tray on wheelchair arms. Attach hook and loop straps around the chair. Secure the bands by pressing the “hook” material.

**Easy to Care**

All trays may be cleaned with sprayed-type cleanser and a damp cloth or sponge. The armrest covers can be laundered and dried at temperatures up to 180°F (82°C). Do not use bleach.



Manufactured by: Skil-Care Corporation • Made in the U.S.A