



#### New Sta-Dri Instructions

##### Notes:

When all the desiccant beads are green, it is time to reactivate. To do this, place desiccant, screen side up, in microwave and heat for 1 minute on high. Let cool for 20 minutes. Most beads will have a yellowish tint which means it is ready to use.

In a conventional oven, remove desiccant from jar and bake, screen side up, for 1 hour at 200 degrees F or until beads are yellow. Allow desiccant to cool before handling.

##### **Do Not Place Hearing Aid in Microwave or Oven**

##### Directions:

1. Before retiring for the night, clean your hearing aid (and ear molds, if you have them) with Tech Care Clean Wipes disinfectant wipes.
2. Open Battery doors
3. Place the hearing aid on the screen side of desiccant.
4. Replace the lid and close tightly.