# Healing/Off-loading Shoe

## Instructions For Use

| OrthoWedge™      | HeelWedge™

**EN**

### Indications

- > Postsurgical healing
- > Wounds or ulcerations
- > Any situation that requires a pressure off-load of the forefoot (OrthoWedge™) or rearfoot (HeelWedge™)
- > Forefoot trauma (OrthoWedge™) or rearfoot trauma (HeelWedge™)

### Contraindications

- > Open fractures

### Cleaning Instructions/Maintenance

- > When the shoe is slightly dirty, clean it with a damp cloth or a soft brush.
- > Allow the shoe to air dry only.

### Instructions for use

The DARCO® healing/off-loading shoe promotes faster healing when wounds or ulcerations are present. The shoe fits right and left equally.

1. Open the shoe by means of the closures.
2. Please ensure that the foot is properly positioned in the shoe.
3. Secure the foot in the shoe with the closures.

### Caution

Please always follow these instructions for use. In case of pain or swelling, discontinue use and consult your medical provider immediately. This device is designed to assist recovery and should be used exclusively under the instruction of and in consultation with a healthcare professional. DARCO® International Inc. does not assume any responsibility for misuse of the product. It is advisable to wear dressings with the shoe. Do not drive a vehicle while you are wearing this device. Use extreme caution when walking on all surfaces when using this device. Walk more slowly than normal. This shoe is vulnerable to breakdown when excessive weight is applied and is not designed to be worn in pairs. This device is for single-patient use only.

*To the patient/user: This device has been designed and manufactured by DARCO® International,*

**EC REP**

Manufactured for:  
DARCO® International, Inc.

HOS-IFU-21\_Rev0321

to be state of the art for treatment of a specific medical condition. Should any serious incident occur in relation to this device it should be reported to the manufacturer at the contact information listed on this document and the proper authority of the country or state where you are located. When treatment is complete, dispose of this device according to local laws and ordinances.

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