



# southpaw enterprises, inc.®

Store these instructions with the enclosed maintenance checklist in a safe place. You may also access them on our website.

## Instruction Sheet

## Cuddle Swing 120030

## Adult Cuddle Swing 120035



Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

**IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.**

Warning: Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

### MATERIALS LIST:

- 1 - Cuddle Swing
- 1 - Instruction Sheet
- 1 - Maintenance Checklist

**WORKING LOAD:** 200 lbs. Adult Cuddle Swing 300 lbs.

### INSTRUCTIONS FOR USE:

To use the Southpaw Cuddle Swing, attach a Southpaw Safety Snap (# 0100) to the loop at the top of the swing then attach the snap to your suspension point. A Southpaw Height Adjuster (# 5000) may be needed to adjust the swing to the proper height for your client. A Southpaw Safety Rotational Device (# 2000) must be used for any rotational activity. The Cuddle Swing will provide full and even support for the client seated in the swing. Client may need assistance to enter and exit the swing.

**SPACE REQUIREMENTS:** Use caution to keep a minimum of 6 foot clear distance from walls and other obstacles.

### MAINTENANCE CHECKLIST:

All of the following wear points should be checked on a monthly basis and recorded on the checklist enclosed. Check the loop for wear at the point the loop rubs against the Safety Snap. If wear is found, discontinue use until repaired. Check seams for loose or missing stitches. If loose or missing stitches are found, discontinue use until repaired.

### MATTING PRECAUTIONS:

Always use appropriate matting.  
Crash mats and wall mats must be used with all hanging equipment.

### STORAGE PROCEDURES:

Store out of direct sunlight.  
Store where not accessible by clients without proper supervision.

### CLEANING INSTRUCTIONS:

Check with your infectious disease department before using these procedures.  
Check for client allergies first.  
Machine wash warm water, mild detergent. Tumble dry, low heat.

## CAUTION

- ◆ Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.
- ◆ Floor mats of proper thickness should be positioned properly under and around all suspended equipment.

**Safety Precautions** - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

**This sheet is for example only.** It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



# Southpaw Enterprises Inc.®