


KRATOS POSEIDON with SafetyGrip®

KPSG780

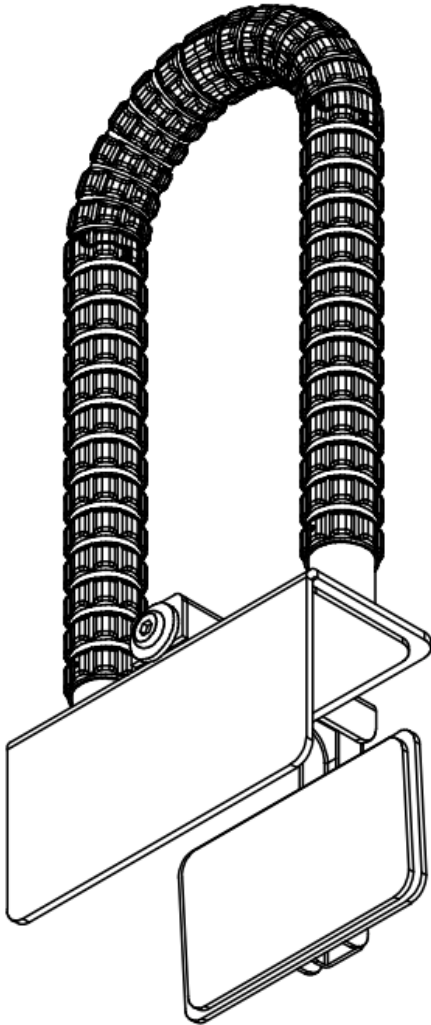


Safety Information

! **IMPORTANT:** If you are unsure that you can install this product correctly, hire a qualified professional.

KIT CONTENTS

The Hex Key needed for this installation is included in the kit.

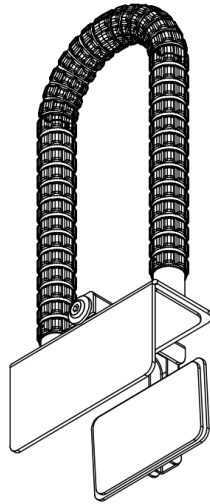

TOOLS



HEX KEY

PRODUCT INFORMATION

The KPSG780 Bath Hand Rail comes with our patented Interlocking SafetyGrip®. The KPSG780 is protected with AlphaSan® technology, which provides lasting protection against microbial contamination to all Hartmobility products, delivering unrivaled anti-microbial performance.

- This KPSG780 is designed to assist with balance and stability for persons weighing up to 350lbs (158kg) and is not designed to bear the full weight of the user.
- **NOT DESIGNED FOR FIBERGLASS SURFACES.**

INSTALLATION

Installer Note: Make sure that all surfaces where you want to clamp KPSG780 Bath Hand Rail are clean and dry.


STEP 1: Position the Bath Hand Rail

Position the Bath Hand Rail over the edge of the bath tub where you want it located.

STEP 2: Securely Tighten The Clamp

Securely tighten the clamp onto the side of the bath tub using the Hex Key. There are 3 non-slip surfaces on this Bath Hand Rail. Make sure that all 3 are in full contact with your bath tub surfaces where you are mounting this unit. Finally, make sure that the Bath Hand Rail does not move when you push and pull on it.

Installer Note: Keep the Hex Key because you will need it to maintain the safety of your Bath Hand rail

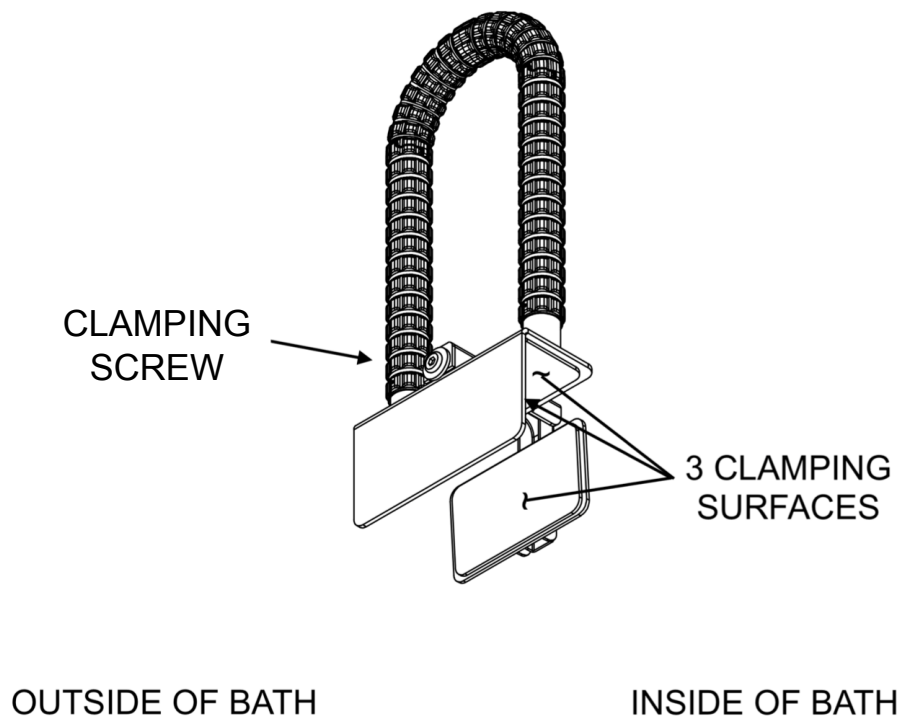

MAINTENANCE

- The Bath Hand Rail is installed on a smooth surface and can loosen with use. Check the installation and the Clamping Screw weekly.
- To clean your Bath Hand Rail, simply wipe with a clean cloth and any commercially available cleaning solution.

LIMITED WARRANTY

Hartmobility Inc. product(s) are covered by a limited lifetime parts warranty. This warranty covers against defects in design and workmanship. Hartmobility Inc. will correct any defects in parts or workmanship at no charge.

The warranty does not cover product failure due to installation error, owner misuse, abuse, negligence or normal wear and tear.

