

OVERVIEW


We are constantly answering questions and recording helpful videos to make using your Vive Bed Assist Foot Bar as easy as possible. Check out the included links and QR codes to help you through the process.



To see all FAQs in one place visit **vhealth.link/ojh**

WHAT'S INCLUDED

- Rear Base Attachment
- Front Base Attachment
- Foot Bar
- C-shaped Locking Pin
- Securing Strap

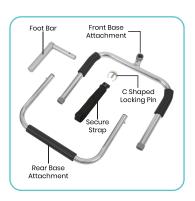

INSTRUCTIONS



For a video demonstration, check out **vhealth.link/4tk**

Slide the Rear Base
Attachment into the
Front Base Attachment
until the locking pins
on both sides
have engaged.



2. Slide the Foot Bar into the top opening of the Front Base Attachment and Use the C-shaped Locking Pin to lock it into place.





3. Use the securing strap to lock the assembled Bed Assist Foot Bar down at the front rail to the side of your bed. Tighten down as best you can. (You will notice there is probably extra length than needed for your strap as every bed is different and we wanted to accommodate all fits. Please tuck the extra strap under the bed or out of the way so it is not a hazard.)





WARNING 🗥

- Do not use the Foot Bar until you have used the strap to secure the product to the bed.
- This product should work universally for all beds but there may be a few beds it might not work for as it is

not recommended for electric beds. If you have a bed that is not typical of most beds feel free to call us for help in where to place the strap to secure the item.

 Check periodically to make sure the strap is secured and all locking pins are properly locked into place before use.

HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at **vhealth.link/ojh** for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website.



service@vivehealth.com



1-800-487-3808



vivehealth.com

Distributed by



8955 Fontana Del Sol Way Naples, FL 34109 Made in China