LIFETIME LIMITED WARRANTY

Your Medline brand product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, which are subject to normal wear and replacement. Non-durable components are warranted for 6 months. If you have a question about your Medline product or this warranty, please call us at 1-800-MEDLINE. This warranty gives you specific legal rights, and you may have other rights, which vary from state to state.

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Three Lakes Drive, Northfield, IL 60093 USA. 1-800-MEDLINE

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual.

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**Do not install this product
without first reading
and understanding this
instruction manual.**

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**Failure to follow
instructions may result
in serious injuries.**

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IMPORTANT SAFEGUARDS

Read all instructions before using.
Important information is highlighted by these terms:

WARNING– Important safety information for hazards that might cause serious injury.

CAUTION– Information for preventing damage to the product.

NOTE– Information to which you should pay special attention.

READ ALL INSTRUCTIONS BEFORE USING.

WARNINGS:

Please read carefully before use. Failure to comply may result in severe injury.

DO NOT exceed the 250 lbs (113 kg) maximum weight capacity of this safety frame.

Evenly distribute weight between the armrests during use.

Ensure all detent buttons are properly engaged in position prior to use.

FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURIES.

SAVE THESE INSTRUCTIONS.

ASSEMBLY INSTRUCTIONS

Before beginning assembly, identify all parts (Figure 1)

1. Remove existing toilet seat assembly, including bolts and nuts.
2. Place toilet safety frame bracket between toilet fixture and seat assembly with rounded part of bracket closest to the back of toilet (Figure 2).
3. Replace toilet seat assembly and reinsert the original mounting bolts through front holes of the bracket and toilet fixture (Figure 3). Tighten the nuts.
4. Depress the built-in snap buttons and tighten the locking collar on each side of the safety frame (Figure 4).
5. Slide each side frame into desired width position. Insert black adjustments hand knobs to the bracket and tighten to secure into place (Figure 5).
6. Adjust foot pieces to desired height (Figure 6). Make sure both foot pieces are fully engaged and adjusted at equal heights.
7. Make sure mounting hardware is tightened and all snap buttons are fully secure before use.

Figure 1

Figure 2

Figure 3

Figure 4

Figure 5

Figure 6

Product fully assembled

CLEANING

Your safety frame can be easily cleaned with regular household cleaning products. **DO NOT** use abrasive powders or scouring pads.

Grip cleaning instructions

Clean the grips with a mild soap, detergent or household cleaner. **DO NOT** use an organic solvent (such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause the grip to loosen.

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©2018 Medline Industries, Inc. All rights reserved/tous droits réservés/todos los derechos reservados. Medline is a registered trademark of/est une marque déposée de/es una marca comercial registrada de Medline Industries, Inc.