



# MJM International Corporation

## 196-26-BAR – 196-30-BAR

"We Do Care - We Do More"

### Assembly Instructions



#### IDENTIFY COMPONENTS TO START ASSEMBLY

- SEAT SECTION.
- ELEVATED LEGREST WITH CUSHION
- RECLINING BACKREST WITH CUSHION
- ARMS.
- PUSH PULL HANDLE
- COMMODE
- SLIDING FOOTREST
- MESH SLING WITH PAD
- 16 (SIXTEEN) 1/2" SCREWS.
- BOX WITH CASTERS AND SCREWS
  - 2 CASTERS WITH BRAKES.
  - 3 CASTERS NON BRAKE.

**NOTE: A PHILLIPS SCREWDRIVER OR A CORDLESS HAND DRILL IS REQUIRED TO TIGHTEN/SECURE ALL SCREWS.**

#### STEP 1 CASTERS.




- INSERT CASTER LEG STEMS INTO BASE SECTION.
- DRILL THROUGH LEG PIPES USING 1/8 DRILL BIT AND PILOT HOLES AS REFERENCE.
- SECURE CASTERS USING 1/2" RETAINING SCREWS.

#### STEP 2 BACKREST.



- INSERT RECLINING BACKREST.
- SECURE USING 1/2" RETAINING SCREWS. USING PILOT HOLES AS REFERENCE.

#### STEP 3 ARMS.

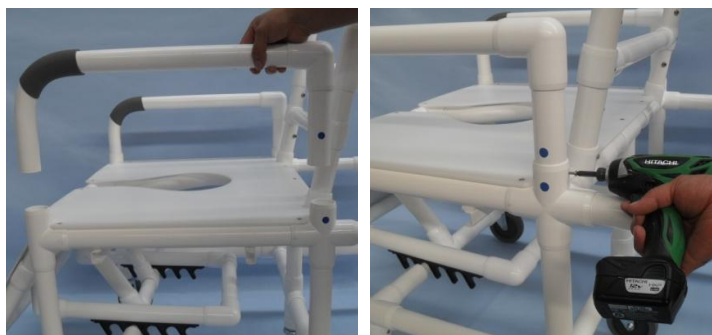

- INSERT ARMREST INTO SEAT SECTION.
- SECURE USING 1/2" RETAINING SCREWS. USING PILOT HOLES AS REFERENCE.

#### STEP 4 PUSH-PULL HANDLE.




- INSERT PUSH PULL HANDLE INTO BACK SECTION



# MJM International Corporation

"We Do Care - We Do More"

## STEP 5 PUSH-PULL HANDLE

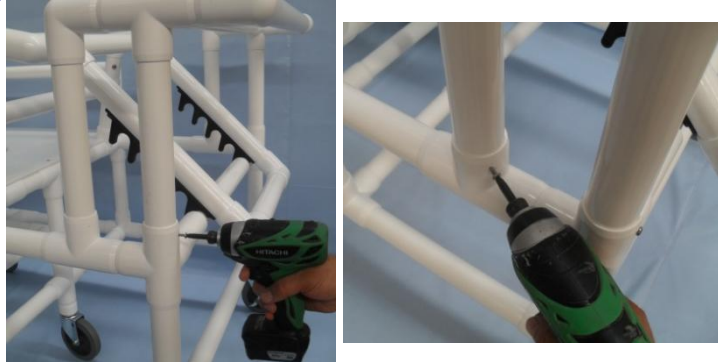

- SECURE USING 1/2" RETAINING SCREWS. USING PILOT HOLES AS REFERENCE.

## STEP 6 SLIDING FOOTREST.




- CENTER SLIDING FOOTREST UNDER FRAME AND INSERT PVC PIPE INTO 4-WAY FITTINGS.

## STEP 7 SLIDING FOOTREST



- SECURE USING 1/2" RETAINING SCREWS AND PILOT HOLES AS REFERENCE..

## STEP 8 LEGREST CUSHION.

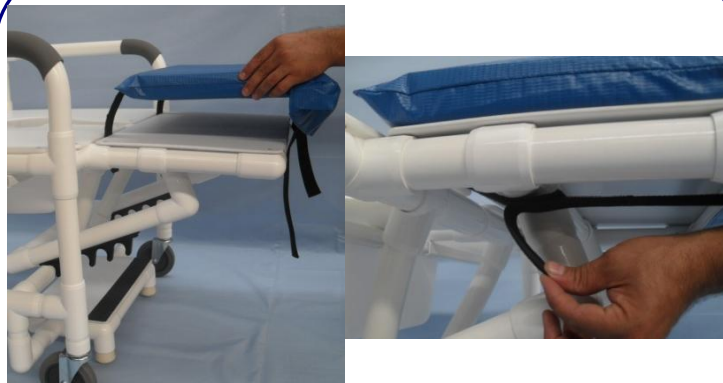

- PLACE CUSHION ONTO LEG REST
- SECURE WITH VELCRO

## STEP 9 LUMBAR CUSHION, COMMODE.



- SNAP ON LUMBAR CUSHION ONTO RECLINING BACK.
- INSERT COMMODE FROM EITHER SIDE ONTO RAILS UNDER SEAT.

## STEP 10.



- PRODUCT COMPLETELY ASSEMBLED

**NOTE: CASTERS, RECLINING BACKREST, SLIDING FOOTREST, PUSH PULL HANDLE MUST BE CHECKED AND SECURED PRIOR TO USING SHOWER CHAIR.**

**WARNING NOT SECURING ALL THE ABOVE CAN CREATE A SERIOUS SAFETY HAZARD.**

Rev: 1.3

323 East Owassa • Suite 4 • Edinburg, TX 78539 • 1-800-270-6199 • 956-781-5000 • Fax 956-781-5552

[www.mjminternational.com](http://www.mjminternational.com) • [hmoroles@yahoo.com](mailto:hmoroles@yahoo.com)



# MJM International Corporation

## CARE INSTRUCTIONS

*"We Do Care - We Do More"*

### **PVC PIPING AND CASTERS**

Wipe down piping with non-abrasive cleanser. **DO NOT** use solvents. Restore luster with Johnson's Pledge or similar spray wax. If casters are used in wet environment, keep lubricated with white lithium paste or similar lubricant; remove hair and mop- thread build up. Provide a weekly maintenance to avoid premature rusting. Rust Proof or Plastic caster need the same care to get the maximum usage.

### **MESH FABRIC (SLING)**

If laundering is required, use only **COLD** water in large machine, with regular detergent and one-cup bleach. Do not use softeners or solvents. Spin-dry only; machine heat will shrink mesh fabric. Hang damp fabric on frame and allow to air dry, returning to its original shape.

### **SOLID FABRIC (PRIVACY SKIRTS)**

Wipe down panels with sponge using solution of mild detergent, one-cup bleach and three gallons water, **DO NOT** launder in machine. To keep vinyl flexible, occasionally spray with furniture polish (e.g. Pledge) or vinyl formula (Armor All). This will help drying out and cracking.

### **FOOTREST AND DROP ARMS**

Remove soap build-up from sliding surface; with a soft bristle brush and soapy solution. Lubricate sliding mechanism with furniture spray wax to restore ease in movement.

When lifting drop-arm, use two hands, one at each end of the section. Lifting only at hand grip will put undue pressure against the sliding fitting.

### **SOFT SEAT**

Wipe with non-abrasive disinfectant wash. Avoid contact with sharp or rough surfaces.

**CAUTION:** **DO NOT** use chair if casters are rusted and fail to move easily or if cracks occur in fittings.

**THE GLOBAL LEADER IN PVC MEDICAL EQUIPMENT!**