



vive®

FOREARM CRUTCHES


Owner's Manual
MOB1014

OVERVIEW

We are constantly answering questions and recording helpful videos to make using your Vive Forearm Crutches as easy as possible. Check out the included link and QR code to help you through the process.


FIND CORRECT HEIGHT

1. Let your arm hang naturally by your waist. Adjust the forearm crutch height so the handle touches your wrist.

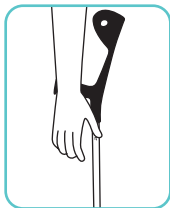

CANE HEIGHT ADJUSTMENT

1. Press in the button snap.
2. Slide extension tube up or down to desired height. Button snap should secure cane into place.

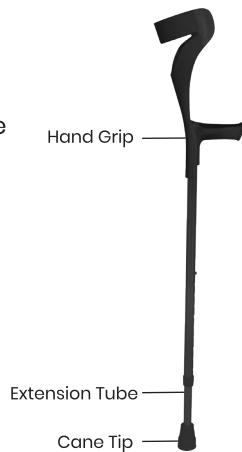

WARNING

- Make sure the brass pin is locked into place before use.
- Weight Capacity: 300 LBS