



ATLAS

WHEELCHAIR STAIRLIFT

Instruction Manual





- 1.0** | **Introduction**
 - 1.1 - Indication for Use
 - 1.2 - General Description / Intended Use
 - 1.3 - Disclaimer
 - 1.4 - Unpacking the Device

- 2.0** | **Symbols and Definitions Used in the Manual**
 - 2.1 - Warning/Cautions/Note Definitions
 - 2.2 - Safety Precautions

- 3.0** | **Diagram of Chair**
 - 3.1 - Side View of the Atlas

- 4.0** | **Setting up the Mobile Stairlift Atlas**
 - 4.1 - Using the Atlas

- 5.0** | **Care and Cleaning Instructions**

- 6.0** | **Maintenance**

- 7.0** | **Troubleshooting**

- 8.0** | **FAQs**

- 9.0** | **Warranty**

- 10.0** | **Device Label**

1.0 Introduction

Thank you for purchasing a Mobile Stairlift Atlas Wheelchair Lift! We aim to provide the safest and most adaptable mobility equipment on the market. Please read this manual in its entirety before operating or maintaining this product. To ensure safe use, all operators of this device should be properly trained.



CAUTION

Improper usage of this device can cause injury to the rider or operator. The Atlas Wheelchair Stairlift must only be operated as described in this manual. Do not modify the Atlas Wheelchair Stairlift or any of its components unless you are a Certified Mobile Stairlift Technician, as modifications can result in unpredictable performance and result in injury to the rider or operator.

NOTE: This manual should remain with the Atlas Wheelchair Stairlift at all times.

For customer service or technical assistance,
please contact us via email Info@ClimbingSteps.com

1.1 Indications for Use

The Mobile Stairlift Atlas Wheelchair Lift is a manual transport device that transports the rider in a seated, upright position on the ground as well as up or down stairs in a safe and efficient manner. The Atlas Wheelchair Stairlift is capable of moving adult, adolescent, or child occupants who are able to remain seated and secured. The Atlas Wheelchair Stairlift's four wheels also allow the operator to transport the occupant on flat surfaces when the tracks are in an upright and locked position.



WARNING

DO NOT use this stair chair lift to transport occupants who may have cervical, spinal, or fracture injuries.

1.2 General Description / Intended Use

The Mobile Stairlift Atlas Wheelchair Lift is intended as a transport device for a human occupant in a seated position to be taken up or down flights of stairs. The occupant's chair should be secured in the chair by restraints for their safety and the safety of the operator. The operator must read this manual and be properly trained on this product before use. The operator must practice with the Atlas Wheelchair Stairlift and an unoccupied wheelchair before using it to transport a rider.

1.3 Disclaimer

This user manual contains general instructions and guidelines for the use, operation, and care of this product. These instructions are not all-inclusive. The safe and proper use of this product is solely at the discretion of the user. Safety information is included as a service to the user. All other safety measures taken by the user should be within and under consideration of applicable regulations. It is recommended that a complete training on how to properly use this product should be provided before actual use.

1.4 Unpacking the Device

Unpack your brand new Mobile Stairlift Atlas Wheelchair Lift, remove all the shipping and packing materials, and check all the items for proper operation. If anything is missing or noted not to be correct, or any type of damage is present due to shipping, please contact Mobile Stairlift.

Items Included:

- 1 x Atlas Wheelchair Stairlift
- 1 x Battery & Key Set
- 1 x Charger
- 1 x Instruction Manual



2.0 Symbols and Definitions Used in this Manual



Consult instructions for use.



General Warning of any possible damages, bodily injury, or even death resulting from incorrect use or carelessness. (See Section 2.1 for more information)



Caution to alert the reader to hazardous situations and if instructions are not properly followed could result in minor or moderate injury to the user or occupant. (See Section 2.1 for more information)



Serial Number of the device.



Date of Manufacture



Product should not be disposed of with your other household waste.



Check sign explains the items that you should comply with upon operation, repair and maintenance of this device.



Do not lubricate the belts or tracks of this device.

2.1 Warning / Caution / Note: Definitions

These words carry special meanings and should be carefully reviewed.



This symbol alerts the reader about a situation which, if not avoided, could result in death or serious injury. It may also describe potential serious adverse reactions and safety hazards.



This symbol alerts the reader of a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage the product or other property. This includes special care necessary for the safe and effective use of the device and the care necessary to avoid damage to a device that may occur as a result of use or misuse.

NOTE: This provides special information to make maintenance easier or important instructions clearer.

2.2 Safety Precautions



1. Read this entire user manual before using the Mobile Stairlift Atlas Wheelchair Lift, and use the Atlas Wheelchair Stairlift only as described in this manual.
2. Always make sure that the chair is locked in the unfolded position before use. An unlocked chair may fold during use causing injury.
3. Do not use the chair with suspected cervical, spinal, or fracture injuries.
4. Do not leave an occupant unattended. Maintain a grip on the Atlas Wheelchair Stairlift while an occupant is seated and secured to the Atlas Wheelchair Stairlift.
5. Always use the restraint straps to secure the occupant's chair to the lift. An unrestrained occupant may fall from the lift.
6. While on flat surfaces, do not push the chair with the upper control handle in the fully extended position. Pushing the chair with the handle extended may cause the chair to tip when you encounter obstacles.
7. Always lock the chair in position before you transport an occupant.
8. Make sure the occupant keeps hands inside the chair when moving up or down a staircase.
9. Avoid getting dirt inside the tracks.
10. Do not attach any additional items to this chair unless certified by Mobile Stairlift.
11. The Atlas Wheelchair Stairlift may not work the same on all stair surfaces and in all environments and conditions.
12. Never lubricate the belts or tracks of this product. Lubricating the track system can result in injury to the patient and /or operators.
13. Do not attempt to transport occupant loads greater than what you can safely lift.
14. To clean the Atlas Wheelchair Stairlift, wipe with a towel and clean water. Dry after cleaning. Please note that cleaning products may cause corrosion and damage to the Atlas Wheelchair Stairlift.
16. Use only parts that are specifically made for this Stairlift. If replacement parts are needed, please contact Mobile Stairlift for assistance.
17. Maintain and operate the Atlas Wheelchair Stairlift only as described in this manual. Mobile Stairlift can be reached for all service, maintenance and questions regarding this chair and its operation.

3.0 Diagram of Chair



3.1 Side View of Mobile Stairlift Atlas Wheelchair Lift



Foot brakes are located behind the two back wheels. When at rest, please engage the brakes to prevent unwanted movement.



Tension controlled tracks make controlled descents safer, and durable rubber ensures safe transport for years to come.

4.0 Setting Up The Mobile Stairlift Atlas Wheelchair Lift

NOTE:

It is recommended that the operator begin training by practicing using the Atlas Wheelchair Stairlift without a rider. This will help them get familiarized with the working mechanisms of the Atlas Wheelchair Stairlift and how to safely move on flat surfaces, as well as how to ascend and descend staircases with a rider.

Before each use of the chair, make a visual check to see that all parts are in place and that there are no loose or moving parts. If you see anything out of ordinary issues with the chair, do not use it until you verify with our customer service rep that it is safe to use.

STEP 1 | After taking your Atlas out of the box, lay the device down with the tracks on the ground.



STEP 2 | Locate and remove the two wing bolts on the black frame near the large back wheels.




STEP 3 | After removing the wing bolts, lift the black base frame vertically and reinsert the wing bolts into the black frame—securing the frame vertically.




STEP 4 | Stand the Atlas up and lock the wheels on both sides by pressing on the red levers on the wheels.



STEP 5 | Slide the red wheel supports in either side by lining up the pieces while pulling the linch pins.

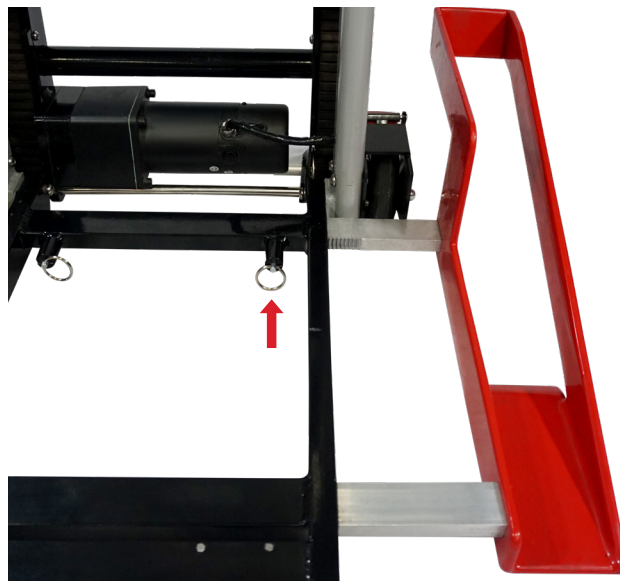

STEP 6 | Similarly, slide the wheelchair frame supports on either side inward towards the handlebar while pulling the linch pins.

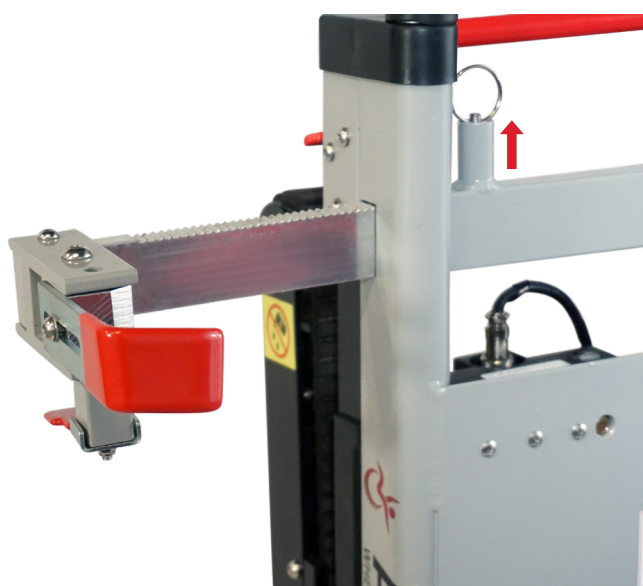

STEP 7 | Your Atlas is fully assembled and ready to be adjusted for a wheelchair.




4.1 Using The Mobile Stairlift Atlas Wheelchair Lift

STEP 1 | Slide the wheelchair into the Atlas, making any minor adjustments if needed, and lock the wheelchair's wheels.



STEP 2 | Press on the red lever of the wheelchair frame supports to close and lock the handle around the frame on either side.

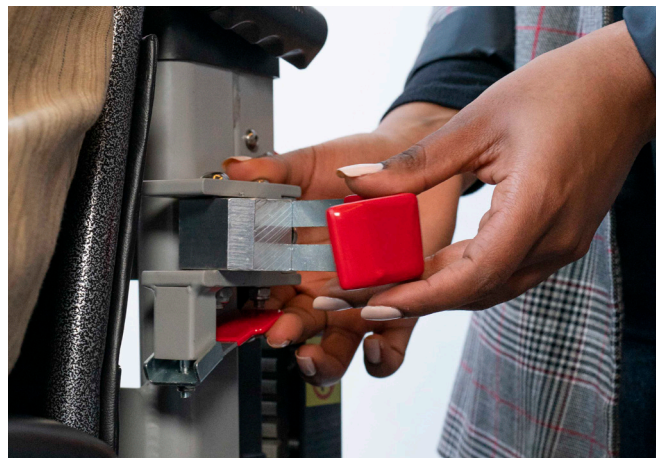


STEP 3 | Secure the occupant into the Atlas by buckling the seatbelt around them and the wheelchair.




STEP 4 | Pull on the red horizontal bar to unlock the adjustable handlebar. Set to a comfortable height.



STEP 5 | Locate the red lever between the tracks and press while pulling downwards to release the tracks until they lock into place.

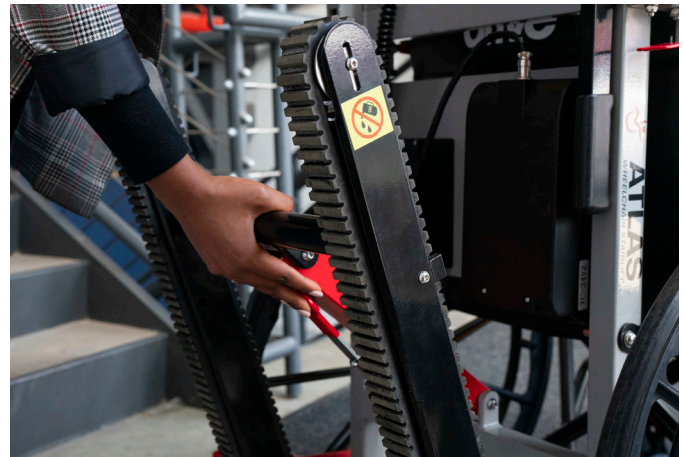

STEP 6 | Pull the push handles up by pressing downwards on their red release lever and move them upwards until they also lock into place.

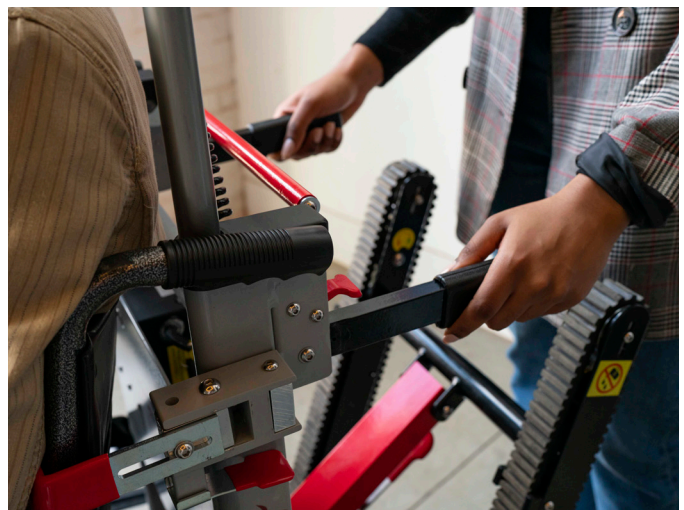

STEP 7 | Using the push handles, align the Atlas with the edge of the stairs. Put the push handles in their retracted position before proceeding.



STEP 8 | Power the device on, tilt the Atlas backwards to the stairs and hold the “up” button to ascend, or the “down” button to descend.



STEP 9 | Pull the push levers up by pressing downwards on their red release lever and move them upwards until they also lock into place.



STEP 10 | After arriving at the staircase landing, keep moving backwards or forwards until there is adequate space to set the Atlas down on all four wheels safely.

NOTE: To descend a staircase, tilt the Atlas backwards and use the “down” button on the device until the tracks catch on the stairs.



5.0 Care and Cleaning Instructions

- To clean the chair, use a clean cloth or soft brush with warm clean water.
- Dry completely before use.
- Do not use products that have bleach or phenolic cleaners.
- Avoid over-saturation or leaving the chair wet for any amount of time.
- To clean the track casing, use SD-20 All Purpose Cleaner and a cloth to remove any grease or other substance.
- Thoroughly dry before use.
- In a medical setting, we recommend you disinfect the chair between riders with basic disinfectant for patient and operator safety.

6.0 Maintenance

To obtain parts and service call Mobile Stairlift information in the front of this manual.

Please contact your representative for prior approval before returning this product

7.0 Troubleshooting

If you have any questions about the Stairlift, please call +1 (212) 366-5483 for customer service assistance. The device label will also show the serial number and the date manufactured

8.0 FAQs

Q: Can the rider operate the Mobile Stairlift Atlas Wheelchair Lift alone?

A: No.

Q: Can the chair be used in more than one location?

A: Yes, the chair folds up small enough to fit in most car trunks, and it is meant to be transported wherever it is needed!

Q: Will this work on carpeted stairs? How about Wood, Cement, etc.?

A: Yes, the Atlas Wheelchair Stairlift is made to work on all surfaces. On carpeted stairs, the carpet must be fully attached to the stairs. It will not work with unattached carpets: the tracks will not be able to grip properly and may slip during use.

Q: So the Atlas Wheelchair Stairlift will work on cement, wood, iron and all types of stair surfaces, but what about different shaped staircases, like round staircases?

A: The Mobile Stairlift Atlas Wheelchair Lift does not work on round staircases.

9.0 Warranty

Thank you for your interest in the products and services of Climbing Steps, Inc.

This Limited Warranty applies to physical goods, and only for physical goods, purchased from www.MobileStairLift.com

What does this Limited Warranty cover?

This Limited Warranty covers any defects in material or workmanship under normal use during the Warranty Period. During the Warranty Period, Climbing Steps, Inc will repair or replace, at no charge, products or parts of a product that proves defective because of improper material or workmanship, under normal use and maintenance.

What will we do to correct problems?

Climbing Steps, Inc will either repair or replace the Product at no charge, using new or refurbished replacement parts or products.

How long does the coverage last?

The Warranty Period for products purchased from Climbing Steps, Inc is Lifetime on the Frame Welds and 180 days on all parts from the date of purchase. A replacement product or part assumes the remaining warranty of the original product or 180 days from the date of replacement or repair, whichever is longer.

What does this Limited Warranty not cover?

This Limited Warranty does not cover any problem that is caused by:

- Conditions, malfunctions, misuse, accidents, physical damage, modifications, or damage not resulting from defects in material or workmanship.

What do you have to do?

To obtain warranty service, you must contact us to determine the problem and the most appropriate solution for you.

10.0 Device Label



Indications for Use: The Mobile Stairlift Atlas Wheelchair Stairlift is a manual transport device. The Stairlift transports the body in a seated, upright position up or down a set of stairs. The chair holds adult, adolescent, or child occupants who can remain seated and secured in the chair. The chair stair lift is for home use with proper training. This device can also be used in hospital environment and emergency medical service and for medical first responders. The front and rear wheels also allows the operator to transport the occupant on flat surfaces.



SN



th



MOBILE STAIRLIFT
ATLAS WHEELCHAIR STAIRLIFT

