

## OTHER PRODUCTS AVAILABLE

---


Your Serial #: \_\_\_\_\_



For more models like this, call 800-475-8122.

Ask about our Protzet-a-bed — a machine washable, waterproof mattress protector!  
Now available in all sizes!

Turn your bed into a luxury adjustable bed with our independent dual-massage system that fits right under your mattress!

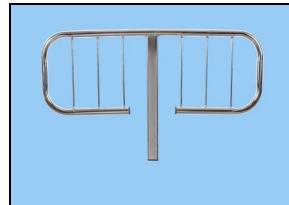

Call in today to order your battery back-up, headboard, footboard, or guard rails! Also ask for our hand control clips!

505 W. Williams • PO Box 917 • Postville, IA 52162

563-864-7364 • Fax: 563-864-7674

[www.transfermaster.com](http://www.transfermaster.com)

---



\*See note.

**Transfer Master** Products Inc. ©  
"America's Leading Hi-Low Beds" Since 1993

**Transfer Master Products, Inc.**  
**New Valiant Adjustable Bed**  
**Model 311N**

**An Owner's Manual**

505 W. Williams • PO Box 917 • Postville, IA 52162

563-864-7364 • 800-475-8122 • Fax: 563-864-7674

[www.transfermaster.com](http://www.transfermaster.com)

\*Appearance of upholstery may differ from picture.

## YOU MADE THE RIGHT CHOICE

---



“The bed is great!! Thank you so much for helping me get my life back to normal after Katrina.

***Pamela Ortega  
Baton Rouge, LA***

“I have been disabled since 1997 from a spinal cord injury. For the first time in almost ten years, I am able to sleep through the night—eight hours plus. I love my Transfer Master bed!”

***James White  
Hyde Park, MA***

“...We purchased two of your beds...and found them durably constructed and maintenance free and a great relief to my osteoarthritis. We highly recommend your products and wonder how we ever lived without them!”

***B. Clark  
Marstons Mill, MA***

## FULL - ONE YEAR WARRANTY

---

Transfer Master Products, Inc. warrants this bed, to the original consumer only, against defects in material or workmanship for a period of one year from the date of purchase by the original consumer. Transfer Master Products, Inc., through its authorized distributor/service organization, will repair or replace, at its option, any bed which evidences, within the warranty period, a defect in material or workmanship.

This warranty does not include any damage caused to fabric, electrical cables or cords, external or internal finish. Nor does this warranty apply to the foundation cover or mattress that are warranted under a separate warranty by the specific manufacturer. Contact your dealer for service on those items. Transfer Master Products, Inc. will, at its option, repair and/or replace the defective parts and/or components provided that the warranty parts have not been subject to misuse or improper service by the consumer, have not been damaged in transit or handling and/or have not been altered or repaired by unauthorized representatives. This warranty does not include costs for unnecessary service calls, i.e., in-home service calls solely for the purpose of educating the consumer about the product, or finding an unsatisfactory power connection. This warranty extends only to the original purchaser. Proof of purchase is required. If Transfer Master Products, Inc.’s original purchaser (if different from the consumer) gives a warranty to their customer which in any way differs from the warranty extended therein, Transfer Master Products, Inc. is in no way to be held liable to indemnify Transfer Master Products, Inc.’s original purchaser or any subsequent purchaser, for any portion of the warranty which in any way is different from, modifies or alters the warranty written above. Transfer Master Products, Inc. shall not be liable for any incidental or consequential damages or for any other damage or loss not contained in this warranty. This warranty gives you specific legal rights. You may also have other rights that vary from state to state. This warranty is expressly in lieu of all other warranties expressed or implied.

## SLEEP TIPS



Sleep is one of the body's best ways to re-build. We all need 8-10 hours of sleep per night. Good nightly sleep requires you to relax your body so you can sleep. Your

New Valiant allows you to do that, every night.

You can position your bed to relax, reposition to read, reposition to watch TV, reposition to sleep and reposition again during the night if you so desire.

### FREQUENTLY ASKED QUESTIONS

#### **If I have a problem after I have my bed, what should I do?**

Call the location where you purchased your bed. Someone will be more than happy to help you.

#### **Can this adjustable bed help me get out of bed?**

Yes, raise your head section all the way up, and lower your foot section flat. Then you can easily slide your legs to the floor while still having the support of your bed.

#### **Do I need specialty sheets for my bed?**

Many of our beds are approximately the same size as most beds so there is usually no need for specialty size sheets. In the event you have purchased an 84" Long bed, we can supply you with the name and number of a company which specializes in custom size bedding.

#### **Can I or should I turn my mattress like I did my old one?**

In order to extend the life of your mattress and insure it wears uniformly, we recommend you turn the mattress four times a year— Spring,, Summer, Fall & Winter.

#### **Is there any maintenance I should do to my bed?**

Lower the head, back, and leg sections in down position when not in use. We recommend applying Silicone Spray in the pivot points of your bed. This should be done every 4-6 months.

#### **Do I need special wiring for my adjustable bed?**

No. The beds use standard, grounded, household outlets and require very little electricity. Please do not plug bed into an extension cord.

## Basic Use Instructions

---

NEVER PLACE HANDS IN THE LIFTING MECHANISM WHILE THE BED IS PLUGGED IN.

DO NOT ALLOW CHILDREN TO OPERATE THE BED.

THIS BED IS NOT A TRANSPORTATION DEVICE, THE BED SHOULD NOT BE MOVED ANY SIGNIFICANT DISTANCE WITH A PERSON ON THE BED.

DO NOT OPERATE THE LOWER SECTION WHEN THERE ARE TWO PEOPLE IN THE BED.

KEEP ALL CORDS AND LINENS CLEAR OF THE LIFTING MECHANISM.

DO NOT SIT ON OR TRANSFER FROM THE FOOT OF THE BED - TO DO SO WILL CAUSE DAMAGE.

BED CAN ONLY BE OPERATED WITH CASTERS OR GLIDES INSERTED. TO OPERATE WITHOUT WILL CAUSE DAMAGE AND WILL NOT BE COVERED UNDER WARRANTY.

**TO PREVENT INJURY, FLIP BED ON ITS SIDE WHEN CHANGING LIFT MOTOR(S).**

## Basic Assembly Instructions

Step 1: Carefully examine to determine if there is any shipping damage. If so, notify the carrier immediately and contact our office (800-475-8122) to inform us of the claim.

Step 2: When handling components of the bed, use work gloves to protect your hands.

Step 3: Make sure hand-held control is in safe area so as to not get damaged when turning base deck over.

Step 4: There are up to 3 mattress retainer bars on each bed, unless you have ordered half rails for the foot section. These retainer bars keep the mattress in place when adjusting bed positions. Locate mattress retainer bars, which are clipped together and bubble wrapped.

Step 5: Bolts have already been installed in the deck boards. Using a 7/16" wrench, remove the bolts from the deck boards. Install the mattress retainer bar/bars by lining up retainer bar/bars with holes on the underside of the deck, using supplied bolts.

Step 6: Tighten hardware with a 7/16" wrench.

Step 7: Place mattress on the bed.

### Half Length Guard Rail Instructions

Use 7/16 socket or open end wrench.

1. Locate mounting hardware on the underside of the deck.
2. Remove nuts from all 4 threaded systems.
3. Position bracket on underside of bed and tighten hardware in place.
4. Place half rail into bracket and clip into holding place.

## Guard Rail and Retainer Bar Assembly Diagram

