

OPERATION INSTRUCTIONS

LOW BED MODELS:

250# CAPACITY - LB76, LB80

OPTIONS:

**LB HEAD & FOOT BOARD
LB ELEVATING HEADREST
LB CASTERS**

**WARRANTY - ATTACHED
ASSEMBLY INSTRUCTIONS – ATTACHED
PARTS LIST – ON REVERSE SIDE**

PURPOSE:

This low bed was designed for confused, at risk users. The bed is low to the floor to avoid serious injury when a user unwittingly climbs out. The low bed serves as a restraint-free solution for wandering, at risk users.

USAGE RECOMMENDATION:

The low bed is recommended for users who are at risk of injury when climbing out of a regular bed.

OPERATING INFORMATION – If bed has optional casters, lock all locking casters. Place user in bed. There are no side rails or restraints. If user climbs out of bed, repeat above.

SAFETY/MAINTENANCE INFORMATION:

**Make certain the bed is assembled according to the enclosed instructions.
Optional casters – lock all four locking casters when transferring.
Do not use bed for transporting patients.
Check monthly for hairline fractures in the pipe and fittings.
Do not use casters larger than 2".
Use only 35" x 76" or 35" x 80" mattresses.**

CLEANING INSTRUCTIONS:

Clean the pipe and fittings with in-house disinfectant. Optional fabric is machine washable at low temp. Air dry. Avoid using solvent based or abrasive cleaners.