## EN - 4-POINT PELVIC BELT

Installation & User Instructions

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## DE - VIERPUNKT-BECKENGÜRTEL

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- P-PB500LTS
- P-PB500LTM
- P-PB500LTL
- P-PB600LTM
- P-PB500PBXS
- P-PB500PBS
- P-PB500PBM
- P-PB500PBL
- P-PB600PBXS
- P-PB600PBS
- P-PB600PBM
- P-PB500SRXS
- P-PB500SRS
- P-PB500SRM
- P-PB500SRL
- P-PB600SRXS
- P-PB600SRS
- P-PB600SRM
- P-PB700SRXS
- P-PB700SRS
- P-PB700SRM
- P-PB700SRL

Please note that the Poziform should be used with the hardware supplied and **must not** be used as a transportation safety belt in a moving vehicle.

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## EN - ASSEMBLY INSTRUCTIONS

**BEFORE** fitting the pelvic belt to any wheelchair/seating system, please consult the wheelchair/seating system manufacturer's manual (or contact them) to find out if it is acceptable to do so.

### Clamp Assembly

1. Clip the appropriate size Frame Adapter if required (Fig. 1) around the frame tubing, as close to the rear of the seat base as possible.

Figure 1a

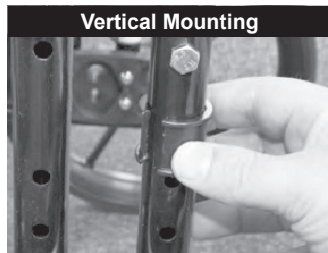
Figure 1b

2. Clip the Frame Clamp around the frame tubing or the Frame Adapter if using the adapter (Fig. 2) making sure the threaded holes are facing towards the wheelchair.

Figure 2a

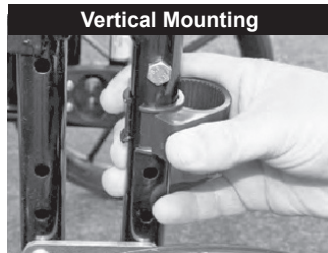
Figure 2b

3. Mount the Strap Guide Plate to the Frame Clamp with the provided washers and screws (Fig. 3). Position the guide plate in the orientation required and fully tighten the screws.

Figure 3a

Figure 3b

## EN - ASSEMBLY INSTRUCTIONS

### Belt Adjustment

- When the user is properly seated in the wheelchair, position the belt across the pelvis and attach the strap to the Guide Plate as shown in Fig. 4.

Figure 4

- Tighten the strap in place making sure the belt is comfortably sitting around the pelvic area (adjust each strap evenly so the buckle is in the middle) and that there is no more than 3" (75mm) of strap coming from the buckle to the 'D' Ring.
- Grab the two ends and buckle the belt.
- Attach both of the 1" straps to the harness (Fig. 5) and fit the strap to the Guide Plate as shown in Fig. 4 so that the straps are not too tight.

Figure 5

Figure 6

- Tension the belt by pressing on the end of the buckle (Fig. 6) and pull on the 'D' Ring until the belt is not too loose to be ineffective nor too tight to cause discomfort.

### Cleaning Instructions

- Clean with a damp cloth and a mild detergent.
- When wet, allow to air dry.
- **DO NOT** dry in any drying machine.
- **DO NOT** wash in a washing machine.
- **DO NOT** iron.

### Maintenance Instructions

- **CHECK** for any signs of fraying and that buckles work correctly.
- **CHECK** and make sure that all hardware is secured and not damaged, especially after making any adjustments.
- **PERIODIC** cleaning will help prevent damage to the buckles.

Every Inspired product comes with our one year manufacturer's warranty. This warranty ensures that when you receive our products, they will arrive free of defects in material and workmanship. We will repair or replace any part or entire product at our discretion for a period of up to one year from the date of delivery to the original purchaser. This warranty does not include cost of inconvenience, shipping, property damage, misuse, abuse, accident, or similar incidents. The warranty will not apply if the product has been modified, tampered with, repaired by unauthorized individuals or assembled incorrectly. Custom orders are non-refundable and non-returnable.

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© 2016 Inspired by Drive™ - A Drive DeVilbiss Healthcare Affiliate