premium seat rizer
with removable arms

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item # 12402

installation instructions

1. Remove product from carton and READ ALL INSTRUCTIONS CAREFULLY before attempting to install product.
2. If installation of arms is desired:
 - a. Place one arm at a time into the arm holes on the raised toilet seat (the brass buttons on the arm should be facing out, away from the toilet seat.)
 - b. Depress the brass button in the front of the arm while sliding the arm into position. You will hear a “click” when the button is positioned properly. (You will see the brass button pop through the position hole.)
 - c. Repeat process for back of arm.
 - d. Repeat process for the other arm.
 - e. To remove the arm, depress the brass button while pulling up on the arm. Repeat for the back of the arm.
3. Remove existing toilet seat and lid from toilet.
4. Position raised toilet seat correctly on rim of toilet bowl and center as necessary.
5. Attach original toilet seat to toilet bowl so it is positioned over raised toilet seat.
6. Check for secure fit routinely before using product. User’s weight should be centered over toilet seat, not too forward or too far to one side, as unit may tip.
7. If you have a style of commode that this product will not fit properly, it is recommended that you discontinue use.

cleaning instructions

1. Use a non-abrasive detergent or household cleaner with warm water.
2. Rinse with warm water and wipe dry.
3. DO NOT USE AN ABRASIVE CLEANER OR CLOTH with this product, as damage may ensue.

caution

1. Always be sure that the seat is correctly and securely in place before using.
2. Physically challenged users should be supervised or assisted whenever using this product.
3. This product is designed for use with the most common types of commodes. It will not fit all commode types; be sure that the unit is correctly installed, fits properly and is stable before use.

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lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

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