MOBILITY STATUS AND SUPPORT DETERMINATION GUIDE



MOBILIZATION TASK	EVALUATE THE RESIDENT'S ABILITY TO:	RESIDENT R	RESIDENT MOBILIZATION SUPPORT NEEDS
TRANSFER IN AND OUT OF BED	MOVE FROM A RECLINED POSITION TO THE SIDE OF THE BED AND FROM A SEATED POSITION ON THE SIDE OF THE BED TO A RECLINED POSITION IN THE BED	/	O STAFF TRANSFER RESIDENT IN/OUT OF BED WITH SIZE OPRIATE, TYPE OF CLOTH SLING WITH TOTAL LIFT (TL) TO TRANSFER RESIDENT IN/OUT OF BED GO TO NEXT MOBILIZATION TASK
MOBILIZATION TASK	OBSERVE RESIDENT	RESIDENT ABILITY	RESIDENT MOBILIZATION SUPPORT NEEDS
DEMONSTRATE LEG STRENGTH	FROM A SEATED POSIITON, LIFT AND KICK LEGS WITH RESISTANCE APPLIED WITH CAREGIVERS'S HAND		AFF TRANSFER RESIDENT FROM ONE SURFACE TO ANOTHER E AND TYPE APPROPRIATE OF CLOTH SLING WITH TOTAL LIFT GO TO NEXT MOBILIZATION TASK



MOBILITY STATUS AND SUPPORT DETERMINATION GUIDE



MOBILIZATION TASK	OBSERVE RESIDENT	RESIDENT ABILITY	RESIDENT MOBILIZATION SUPPORT NEEDS
	STAND FROM A SEATED POSITION		E OR TWO STAFF TRANSFER RESIDENT FROM SURFACE TO ACE WITH APPROPRIATE SIZED HARNESS AND SIT TO STAND
TRANSFER FROM SURFACE TO SURFACE		STAND D	ETERMINE IF ASSISTANCE IS NEEDED FOR AMBULATION

TWO OR MORE STAFF USE TOTAL LIFT AND SIZE AND TYPE APPROPRIATE CLOTH SLING

Transfers From Floor











CLOTH SLING TYPE AND SIZING GUIDELINES

HIGH BACK SLING



- 1. Sling should extend from the top of the head to just above the coccyx (3)
- The base of the sling should measure approximately the same as the width from one hip to the other.

FULL BODY SLING



Sling should extend from the top of the head to just above the knee bend.

When lifting resident fabric should extend outside the body by about 2 inches

SIT TO STAND HARNESS



Sit to Stand Harness is to be placed on the lower back; the cushion should be centered under the arm, safety belt should comfortably reach around the resident's lower back

