

Contents:

Partially assembled shower bed
Six (6) casters (4 locking, 2 non-locking)
Push/pull handles
Close cell mattress
Ten (10) ½" stainless steel screws

Assembly Instructions:**Step 1**

Installing Casters:

1. Install the four locking casters into the female fittings on the base of the head and foot sections of the shower bed. Secure each caster to the fitting with one ½" stainless steel screw.
2. Install the two non-locking casters to the female fittings on the base in the center of the frame. Secure with stainless steel screws.

Step 2

Install one push/pull handle to each bed end. Secure with ½" stainless steel screws.

Safety rails can be held in place by one safety locking pin.

Close cell mattress serves as a buffer/comfort for patient when using the shower bed.

Instructions for Use:

The main purpose for the shower bed is to bathe residents/patients that are confined to a bed. The easy rolling casters allow the shower bed to transfer bed ridden residents/patients to and from the shower room.

- Roll the shower bed next to the individual's bed
- Drop the side rail facing the bed down, keeping the opposite side rail up
- If it's possible and the resident/patient has no medical requirements, transfer the resident/patient with head elevation in down/flat position.
- Have a minimum of 2 persons (nurses, aids, etc.) transfer the resident/patient onto the shower bed.
- Once the resident/patient is safely transferred and in a comfortable position, the head elevation can be raised if needed. Both side rails must be up and locked prior to the shower bed being moved.
- When using the shower bed, direct the drain hose to a drain system. All excess water must drain properly.

