# Littmann® CORE Stethoscope

Connects to Eko Software

Eko

3M

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# 3M™ Littmann® CORE Digital Stethoscope

## Quick Start Guide

Download the Eko app from the Apple App Store® or Google Play Store®.

The Eko logo is a rounded square with a thin black border, containing the word "Eko" in a bold, sans-serif font.


Turn on Bluetooth® on your phone or tablet, launch the app, and turn on the CORE Stethoscope.

### **In the Box**

- Eko CORE Digital Attachment and 3M™ Littmann® Cardiology IV™ Stethoscope
- Additional set of 3M Littmann Snap Tight Soft-Sealing Eartips (small)
- Nonchill Bell Sleeve
- USB Charging Cable
- Quick Start Guide

## Eartips

Insert eartips in a forward direction.

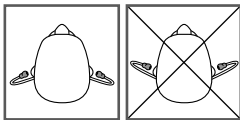## Volume

Use the (+) and (-) buttons to adjust volume.  
There are 7 volume steps.

## Turn On and Off

When the CORE stethoscope is off, it functions as an unamplified stethoscope.

To turn the CORE stethoscope on, press the power toggle in. The light will begin flashing. You will now hear amplified and filtered sound in the eartips.

## Telemedicine and AI Features

Eko offers telemedicine and AI features that require a software subscription (where available, sold separately).

You can setup a subscription online at [ekohealth.com](https://ekohealth.com)

### Active Noise Cancelling

The CORE stethoscope includes active noise cancelling to remove ambient noise from the surrounding environment. Active noise cancelling can be turned off or on through the Eko App.


# Notifications

CORE  
Stethoscope is  
on & seeking  
device

CORE  
Stethoscope is  
off & charging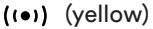


CORE  
Stethoscope is  
low on battery

CORE  
Stethoscope is  
on & connected


CORE  
Stethoscope is  
fully charged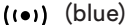

CORE  
Stethoscope is  
recording

# Recording

When connected to the Eko app, press the (o) button to begin recording.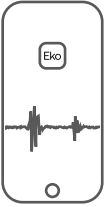


## Placement

The CORE Stethoscope is designed to be used in all traditional auscultation positions for the heart, lungs, bowels, and arteries.

## Charging

Connect the micro USB cable to the CORE Stethoscope and use a UL-certified or CE-marked USB wall charger. The CORE Stethoscope will take about 2 hours to fully charge. The light will turn green when it is fully charged.

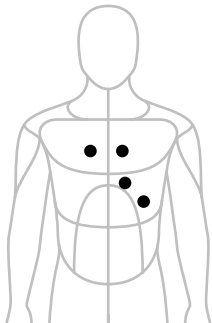## User Manual

Review the user manual before using your Littmann CORE Stethoscope. To view the manual visit:  
[ekohealth.com/getstarted](http://ekohealth.com/getstarted)  
[Littmann.com/COREmanual](http://Littmann.com/COREmanual)

## Service and Support

In the USA, please visit [littmann.com/service](http://littmann.com/service) or call 1-800-228-3957. If you are outside of the USA, please visit [littmann.com](http://littmann.com) for the contact information of your local 3M office.

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©2020 3M; Eko Devices Inc. All rights reserved.
Eko trademarks, including CORE, are marks and/or registered marks of Eko Devices Inc.
© 3M, 2020; Eko Devices Inc. Tous droits réservés.
Les marques de commerce Eko, y compris CORE, sont des marques et/ou des marques déposées d'Eko Devices Inc.