

# STATIONARY OXYGEN CONCENTRATOR FILTERS

The LM5BA 5L Stationary Oxygen Concentrator has 3 filters. For optimal performance and longevity of the unit and to preserve the Manufacturer's Warranty, Rhythm Healthcare provides the following guidance for filter cleaning and replacement.



## AIR INTAKE (SPONGE) FILTER



The Sponge Filter is located on the back of the unit. It should be removed and cleaned on a weekly basis using warm soapy water. Use caution to **COMPLETELY** dry the filter before replacing. The Sponge Filter should also be replaced between patients. The filter should be replaced if it is torn or damaged.

## INLET FILTER (HEPA FILTER)



The Inlet Filter (HEPA) is also located on the back of the unit. This filter should be inspected regularly and replaced if it is damaged or cracked. This filter should also be changed in between patients.

## BACTERIAL FILTER



The Bacterial Filter is located inside of the LM5BA Concentrator. This filter should be changed with every compressor servicing and in between patients.

