


TITAN POLE ACCESSORY with SafetyGrip®

TSGA601

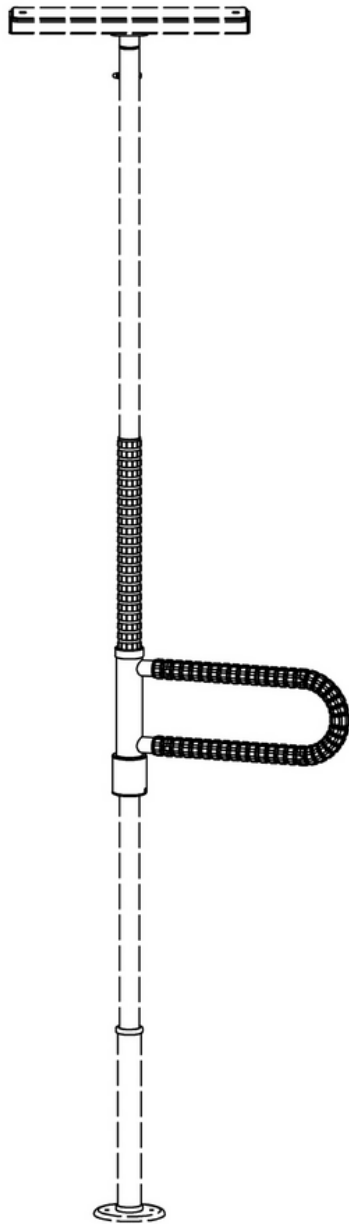


Safety Information

! **IMPORTANT:** If you are unsure that you can install this product correctly, hire a qualified professional.

If you have any questions, please contact Customer Care.

KIT CONTENTS

All tools are included in the kit.

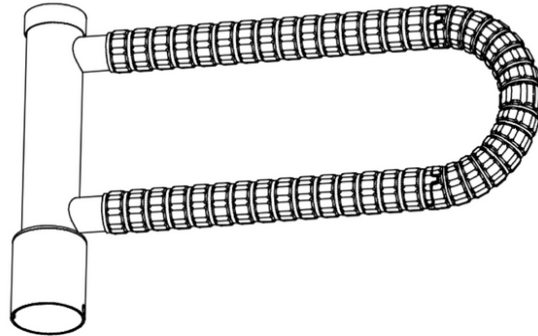

① ROTOBAR



② BOTTOM COGS COLLAR



③ PLASTIC STOP RING



HEX KEY

PRODUCT INFORMATION

This TSGA601 SafetyGrip® RotoBar comes with our patented Interlocking SafetyGrip®. The TSGA601 is protected with AlphaSan® technology, which provides lasting protection against microbial contamination to all Hart Mobility products, delivering unrivaled anti-microbial performance.

- The TSGA601 Safety Grip RotoBar is designed to be used with the TSG610; TSG615 & TSG650 Poles.
- The TSGA602 is designed to assist with balance, sitting up, positioning and stability for persons weighing up to 350lbs (158kg) and is not designed to bear the full weight of the user.

INSTALLATION


NOTES

1. If you purchased the TSGA601 RotoBar and the Titan pole together follow the Steps 1 through 4 below.
2. If you already have the Titan pole and it is installed you will need to loosen it and lay it on the floor. Remove the Base and remove the Bottom Cover Sleeve. Now follow Steps 1 through 4 below.

STEP 1: Slide the Rotating Grab Bar and parts onto the Pole

Slide the RotoBar assembly and the other parts onto the Titan pole from the bottom.

Follow the sequence shown on the Step 1 drawing.

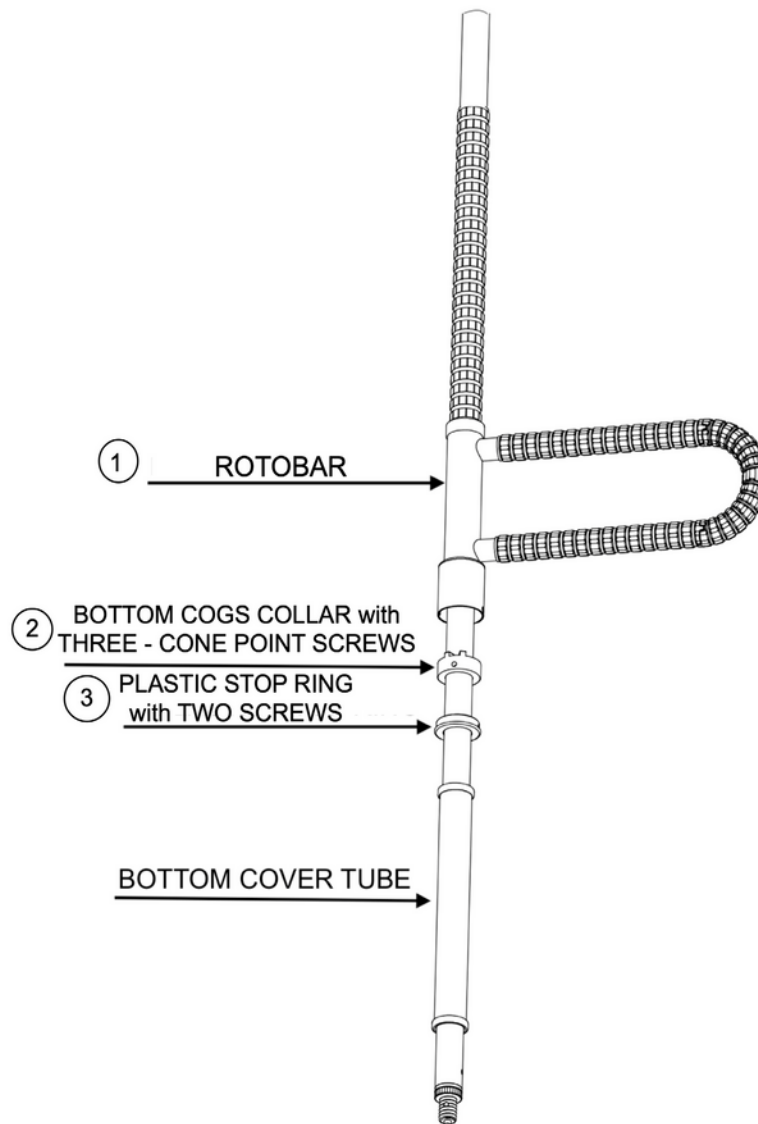

ASSEMBLY of PARTS SEQUENCE
STEP 1

STEP 2: Finish Installing the Pole

Finish installing the Titan pole by following the Pole Assembly Instructions provided with the Titan pole.

STEP 3: Decide how high you want the RotoBar

When the Titan pole is assembled and standing between the floor and ceiling you can decide how high that you want the RotoBar to be from the floor.

Move the bottom Cogs Collar up the Pole to this height.

Installer Note: This bottom Cogs Collar will be the height of the bottom horizontal RotoBar.

Securely tighten the three Security Screws in the bottom Cogs Collar with the Hex Key provided.

STEP 4: Install the Plastic Stop

Install the Plastic Stop Ring into the bottom of the RotoBar with the two screws provided.

Test that the RotoBar lifts off the internal cogs, rotates smoothly and drops down into the cogs.

MAINTENANCE

- The TSGA601 Grab Bar can loosen with use. Check the Security Screws in the bottom Cogs Collar weekly.
- To clean your RotoBar, simply wipe with a clean cloth and any commercially available cleaning solution.

LIMITED WARRANTY

Hartmobility Inc. product(s) are covered by a limited lifetime parts warranty. This warranty covers against defects in design and workmanship. Hartmobility Inc. will correct any defects in parts or workmanship at no charge.

The warranty does not cover product failure due to installation error, owner misuse, abuse, negligence or normal wear and tear.

