drive™

shower stool

banco para ducha

banc de douche

item #s:

RTL12004KD

12004KDRB-1

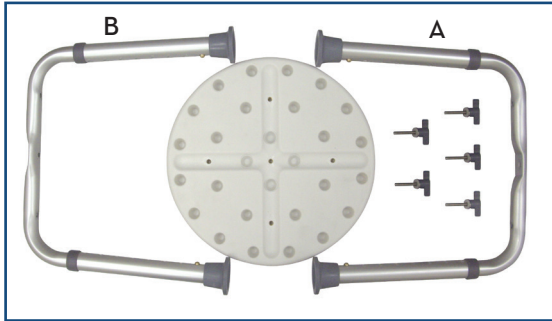
12004KDRT-1

EU Authorized Representative
Drive Medical LTD
Atinley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

drive™

assembly instructions

1. Remove contents from box.
2. Place seat upside down on a flat surface so holes are showing.
3. Align leg section A (the leg section that has the concave area on the underside) with the holes in seat. Place and tighten the two handscrews in the outer holes, making sure to put on a metal washer. Do not affix the middle handscrew.
4. Align leg section B (the leg section that has the concave area on the top side) with the holes in the seat. Place and tighten the two outer holes using the handscrews and washers.
5. Place the middle handscrew and tighten through the two legs.

NOTE: Periodically check the handscrews to make sure they remain tight as loosening may occur.

Weight capacity: 300 lbs (136 Kg)

lifetime limited warranty

Your Drive branded product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

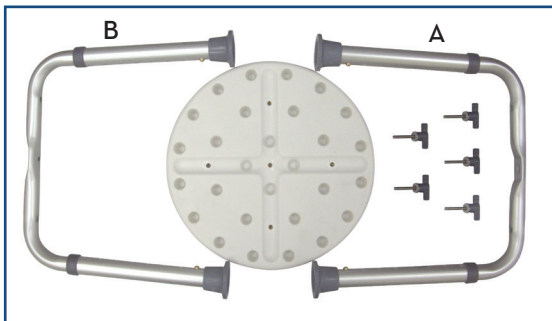
This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

The logo for Drive, featuring the word "drive" in a bold, blue, lowercase sans-serif font. A small yellow square is positioned above the letter 'i'. A trademark symbol (TM) is located at the top right of the logo.

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The logo for Drive, featuring the word "drive" in a bold, blue, lowercase sans-serif font. A small yellow square is positioned above the letter "i". A trademark symbol (TM) is located at the top right of the logo.